
# **FAUTEUIL RELEVEUR NEST**

## **LIFT CHAIR NEST**

**REF :**  
**821159.TISSU**  
**821159.CUIR**

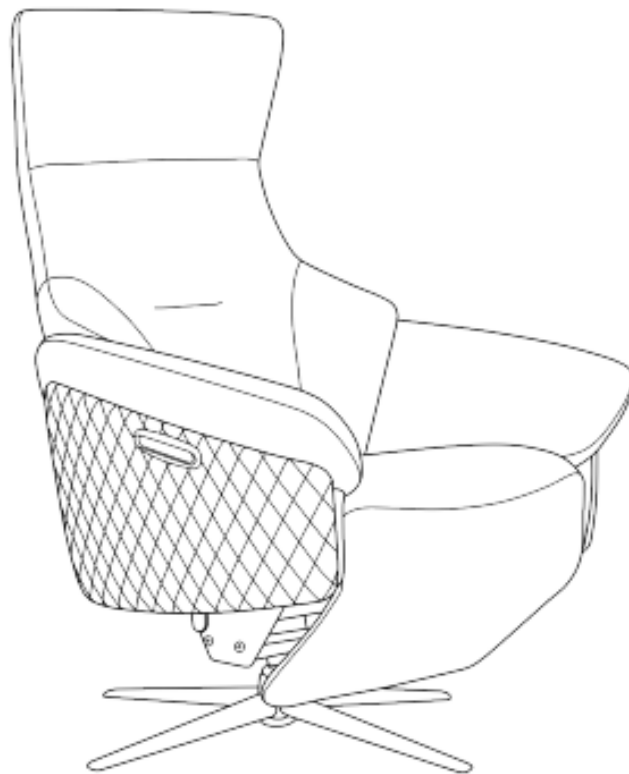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## **INTRODUCTION :**

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## Safety precautions:

### Attention

**Please follow the safety instructions below to avoid damage, electric shock, fire or other injury**

- Before use, be sure to read this "Safety Precautions" carefully and use it correctly.
- The precautions shown below are intended for the safe and proper use of this product to prevent harm or damage to the user and others.

### Attention

- Please use the leisure chair on a level floor. If you need to move the chair, it is recommended to be lifted by two adults, and do not move it by pushing or pulling it so as not to scratch the floor, carpet or wear the footrest.
- When using a power supply, do not pull the power wire, keep it away from fire, water, and keep it out of reach of children and the elderly. Turn off the power switch after use, and store the power wire away to avoid safety accidents caused by children playing with it.
- When using a rechargeable battery pack, pay attention to battery indicator: when the battery pack is fully charged, five blue lights will be on, and when the battery runs low to three blue lights or less, the USB interface and Type-C interface will not be able to charge mobile device.
- If you need to retract the foot rest, you should observe the bottom of the chair first to make sure that there are no pets or other objects down there before operating it safely.
- Before use, wipe the dust on the surface of the chair with a clean soft cloth and keep it always clean.
- Do not allow children to jump and play on the chair alone or those who cannot express their will to use it alone to avoid electric shock or fall.
- Avoid devices with sharp angles or knives from touching the chair surface to prevent scratching the fabric.
- Do not use the chair outdoors for a long time. Avoid exposure to sunlight for a long time to avoid partial fading or bleeding of the color of the product.
- Please keep the chair at least 30 cm away from the heat sources. Do not use it near the open flames such as stove or on a warmer such as electric blanket to avoid fire.


### Prohibition

- Except for professional maintenance personnel, it is forbidden to disassemble, repair or modify the chair at will to avoid malfunction leading to electric shock or fire.
- When using the plug to charge the product, it is forbidden to insert and unplug it with wet hands. When inserting or pulling the plug, you must hold the plug body tight, and it is forbidden to pull the power wire to prevent damage and electric shock.
- When not in use for a long time, please unplug and put away the power wire.


## Installation Method:

### Components include:

- ① Chair base X1
- ② Swivel X1
- ③ Seat X1
- ④ Back cushion X1
- ⑤ Power supply X1
- ⑥ Battery X1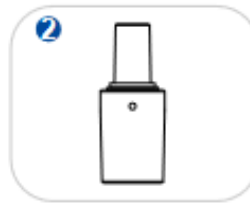


### Installation(Assembly) procedure description:

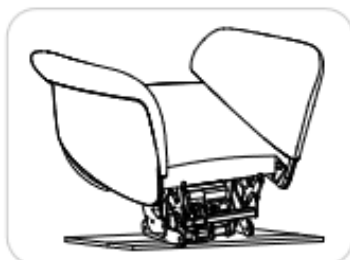- 1** Remove the seat from the carton, lay it flat on the ground, and cut off the 4 ties at the bottom.


- 2** Fix the Swivel shaft on the cross bar at the bottom of the frame with screws (M6\*12mm/4pcs).

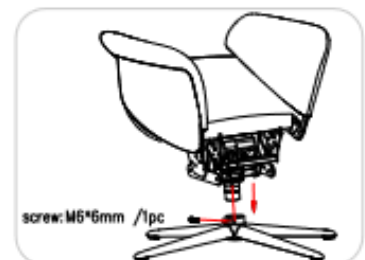- 3** Insert the Swivel shaft vertically into the center hole of the legs base, the hole on the outside of the legs base aligned with the hole on the side of the swivel shaft with screws (1 / M6 \* 6MM) tighten.

- 4** Align the locks of back cushion with the latch of the seat and press the latch firmly. When you hear a "click", the installation is successful.

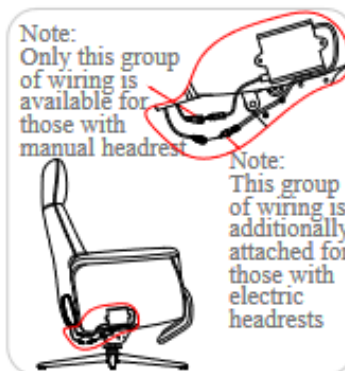- 5** Connect the control box with the outgoing wire of back cushion

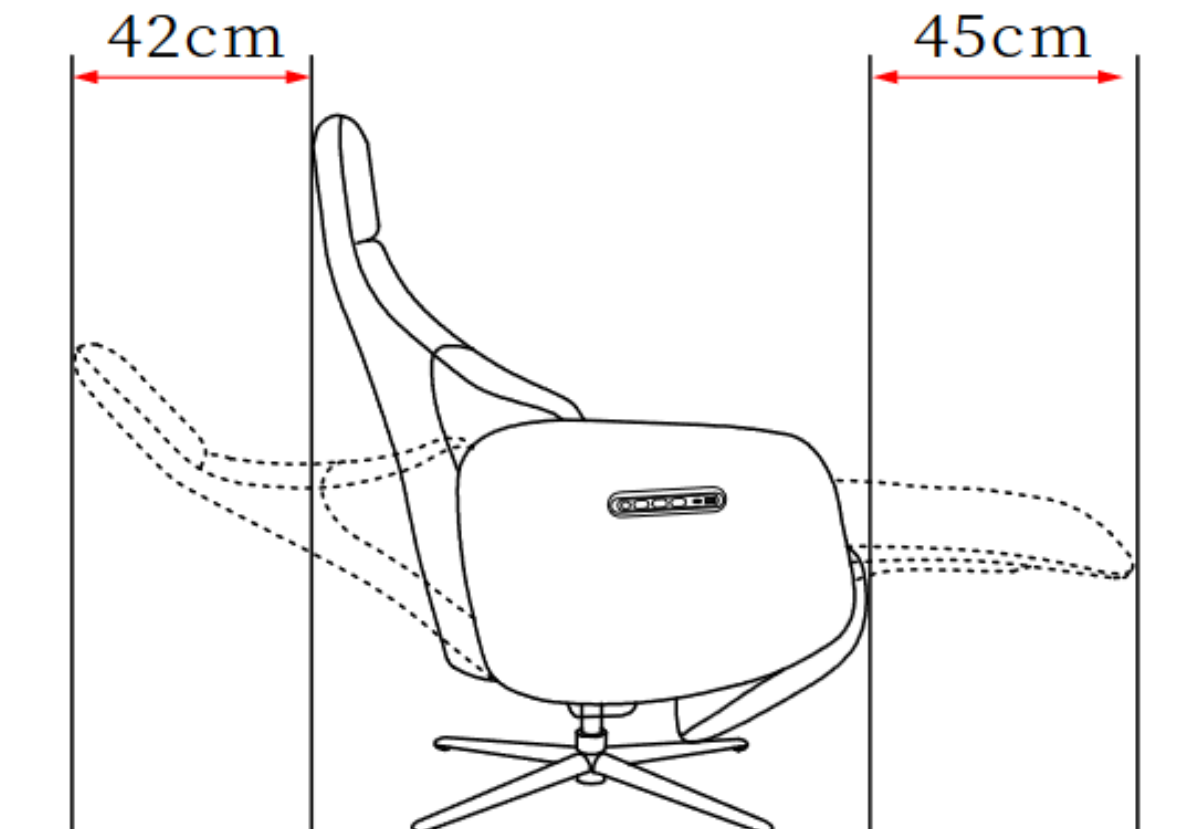
- 6** ① Insert the battery pack in the direction of the card slot.  
 ② Rotate 90 degrees clockwise to complete the installation.

## Environment in use and Application Method:

### Environment in Use:

It is recommended to place it in a flat place, with a space around it that can allow the chair back and foot rest to unfold, so as to avoid collision.

	<p>When lowering the back rest or extending foot rest, in order to avoid collision with surrounding objects, a space of more than 45 cm in front and 42 cm in the rear should be reserved in advance.</p>
	<p>Do not use the product outdoors. Avoid exposure to sunlight for a long time to avoid local fading or bleeding of colors of the product and rapid aging of the fabric.</p>
	<p>Please keep the chair at least 30 cm away from the heat sources. Do not use it near the open flames such as stove or on a warmer such as electric blanket to avoid fire.</p>


## Environment in use and Application Method:

### Application Method:

#### • Turn on the power (power supply method)

Insert the round port at the outlet end of the power supply into the back round outlet hole, and insert the plug into the corresponding matching socket (now, the green light on the surface of the power supply box lights up)
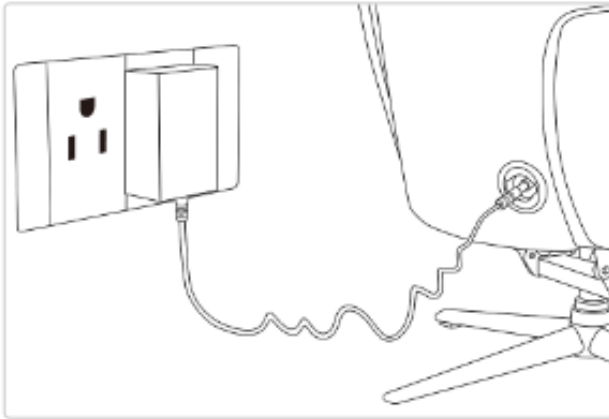


#### Please check before using:

- Check whether hair or dust adhere to the power plug.
- Check whether the power wire and power plug are damaged.
- Do not insert or pull out with wet hands to avoid electric shock accidents.

#### • Turn on the power (battery pack use method)

- ① Insert the battery pack in the direction of the card slot
- ② Rotate 90 degrees clockwise to complete the installation
- ③ Rotate 90 degrees counterclockwise to remove

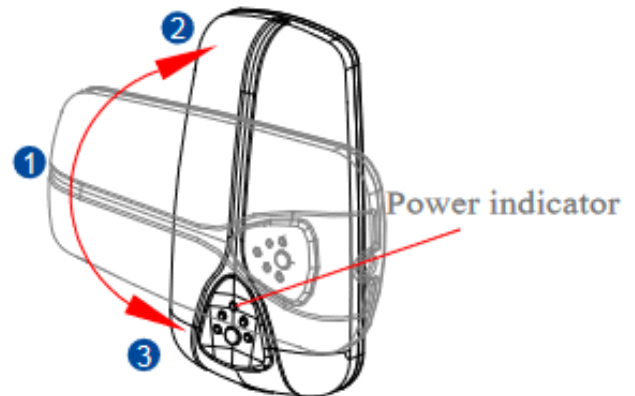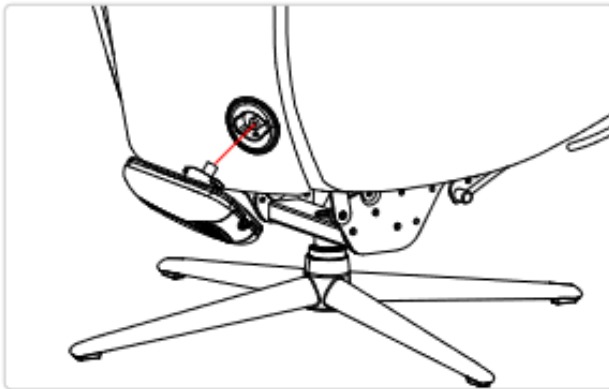#### Cautions for use of battery pack:

#### Attention


When using a rechargeable battery pack, pay attention to battery indicator: when the battery pack is fully charged, 5 blue lights will be on, and when the battery runs low to three blue lights or less, the USB interface and Type-C interface will not be able to charge mobile device.

## Environment in use and Application Method:

Use method of the switch button:

- **Use method of the switch button [applicable to chairs with the electric headrest function]**

The switch buttons ⑦/⑧ are used to control the retraction/opening of the headrest, the buttons ③/④ are used to control the backrest reset, and the buttons ①/② are used to control the opening/retraction of footrest: The buttons ⑨ are used to control the seat lift, Button ⑤/⑥ is an automatic one-key reset (When this button is pressed, the headrest, seat and back will be opened or retracted at the same time)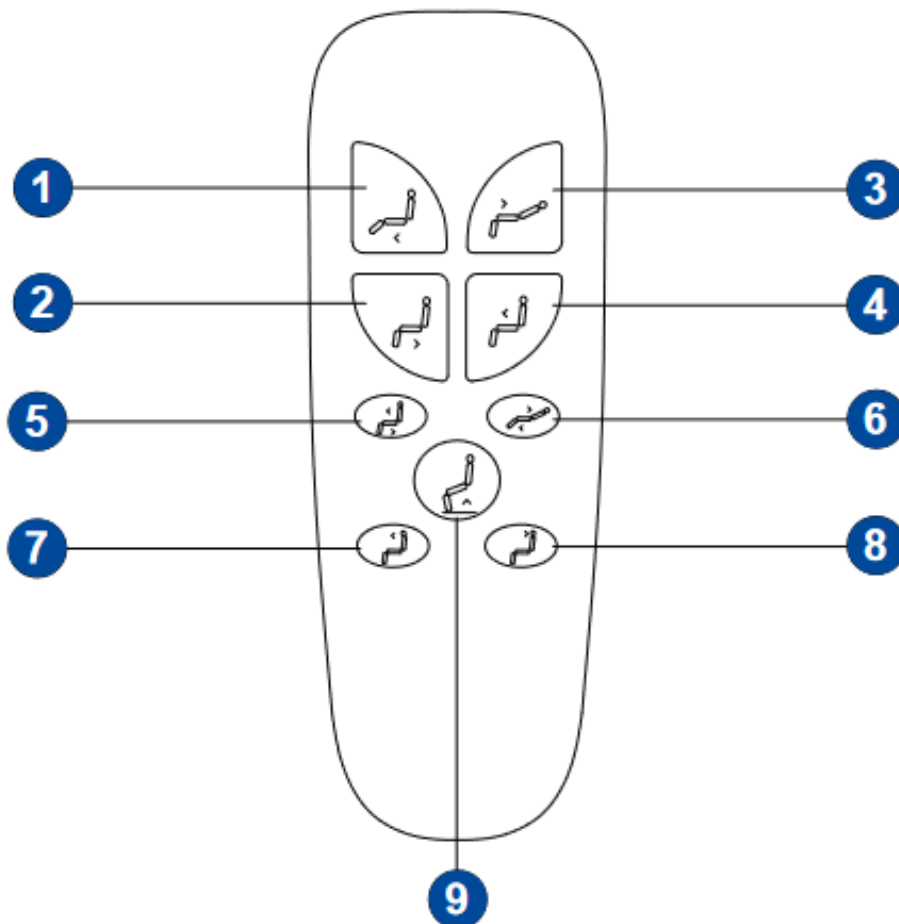

- |                       |                                 |                                |
|-----------------------|---------------------------------|--------------------------------|
| ① Footrest opening    | ② Footrest retraction           | ③ Backrest leaning             |
| ④ Backrest retraction | ⑤ One key to reset all function | ⑥ One key to open all function |
| ⑦ Headrest protrudes  | ⑧ Headrest retraction           | ⑨ Seat lift                    |

## Attention:



- Clean the chair surface with a clean soft cloth before use, and keep it clean frequently.
- Teach children not to jump on the seat surface of the chair, so as not to deform after partial compression and affect the use.
- Avoid devices with sharp angles or knives from touching the chair surface to prevent scratching the fabric.
- If you need to retract the foot rest, you should observe the bottom of the chair first to make sure that there are no pets or other objects before operating it safely.

### Attention

When moving, please note:

- If you need to move it, it is recommended that two adults support the bottom of the armrest to lift the chair to move.
- Do not move the chair when it is unfolded to avoid damage to the chair parts.
- Do not directly push, pull and use other ways to move the chair to avoid scratching the floor and carpet or scuffing the footrest.
- Do not carry people or objects, and do not allow children to handle it to avoid danger.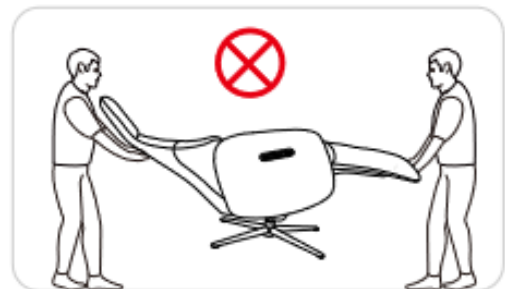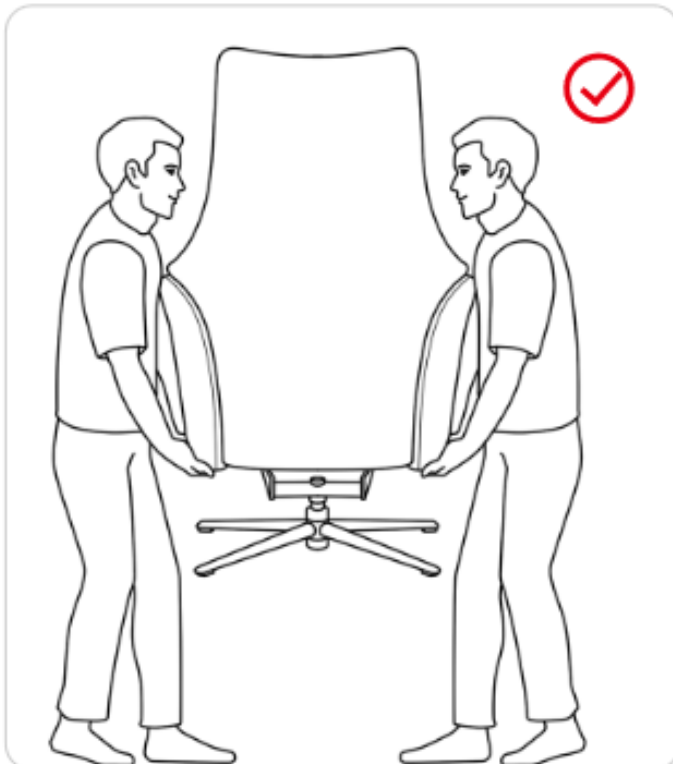

# Maintenance procedure:

## • Daily maintenance procedure:

Use a clean soft cloth to clean fabric dust or stains frequently. The gap at the junction of the back, armrest and seat surface can be cleaned with a vacuum cleaner.

### Attention

#### Special attention:

- Please place the chair in a place with a suitable temperature to avoid staining the fabric by moisture and dust, and avoid discoloration and aging of the fabric caused by strong light.
- Be sure to pull off the power wire 's power plug from the power outlet when performing maintenance, and be careful not to pull it out with wet hands to avoid electric shock and injury
- It is forbidden to use wet cloth, hard objects or chemicals such as acid and alkaline to touch the fabric, so as not to affect the surface quality and service cycle.
- Do not iron the fabric to avoid damage to the fabric.

## • Regular maintenance procedure:

Generally, the maintenance shall be carried out once a quarter. After the chair surface is wiped with a clean soft cloth, the leather fabric can be evenly coated with special leather cleaner or polishing agent, and then cleaned with a clean soft cloth later. When it is not in use for a long time, please put on a protective cover to prevent dust on the surface. .

### Attention

#### Special attention:

- Avoid children stabbing with sharp objects, and avoid children jumping and playing on the chair to avoid injury.
- Children should avoid scribbling on the chair with colored pens, markers or color powder and other pollutants. Once the pollution is found, it must be cleaned up in time.
- Children should not play with the power supply, pull the power wire and remove the battery pack alone.
- When sitting or lying on the sofa, it is not advisable to carry hard objects or metal products on the body, so as to avoid scratching the surface and making it difficult to repair, thus affecting the beauty.

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