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INSTRUCTIONS MANUAL EN

IMPORTANT

Read the instructions carefully before using this product and keep them if necessary.

User: If you are unable to read or understand any warnings, precautions or instructions, please contact your health care professional or distributor before using this product or risk and damage may result.

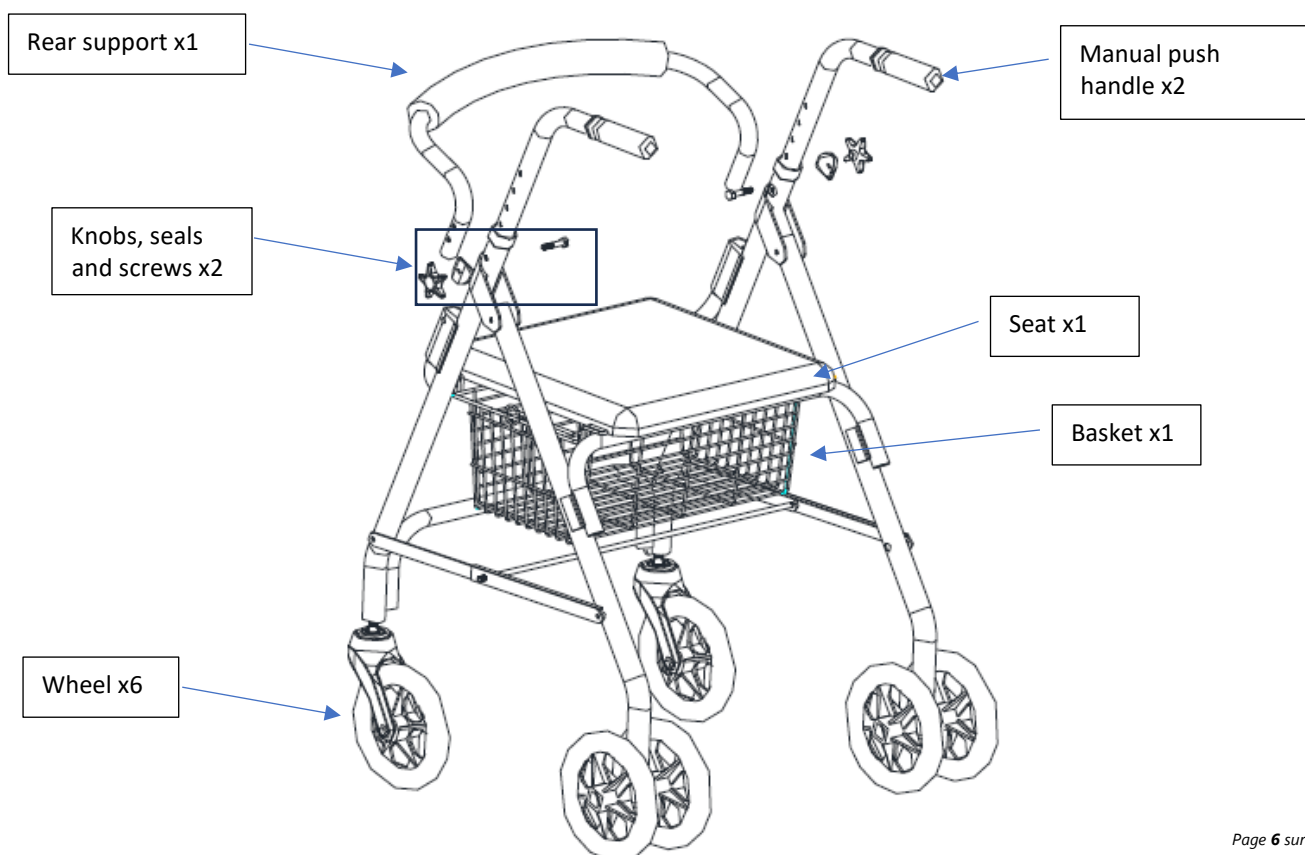
Consult your doctor or health care professional before using a walking aid. He/she will advise and inform you on the most appropriate walking technique for you.

Distributor: this leaflet must be given to the users of this product.

PRODUCT DESTINATION

6-wheel pressure-braked rollators are walking aids designed for people with reduced mobility. These products are designed for indoor and outdoor use and are fitted with brakes on the rear wheels.

PRODUCT DESCRIPTION

SAFETY PRECAUTIONS

Failure to follow the safety precautions may result in the risk of serious injury.

Do not exceed the maximum weight limit of 136 Kg.

Do not exceed the maximum load limit in the basket (5kg). Loads must only be carried in the basket. Do not attach heavy objects to the rollator as this could affect its stability.

Make sure that all parts are securely fastened and in good working order.

Check the rollator's screws and fastenings regularly and retighten if necessary.

Make sure the handles are securely locked.

Make sure the rollator is fully open and the seat is securely fastened.

Make sure both handles are tight and at the same height.

Test the stability of the rollator before applying the full weight to the handles.

Do not use the seat when the rollator is on sloping or uneven ground. Only use the rollator on flat, solid surfaces.

Replace the wheels if they are badly worn or damaged.

DO NOT MOVE THE ROLLATOR WHEN SEATED. It is only a walking aid and must not be used as a transport device.

Test the brakes before using the Rollator.

When the locks are released, the wheels must be able to turn freely.

Beware of the risk of pinching when folding.

Do not use the rollator on stairs or escalators.

Only use the accessories and parts supplied by the manufacturer.

Do not make any adjustments to the rollator while it is in use.

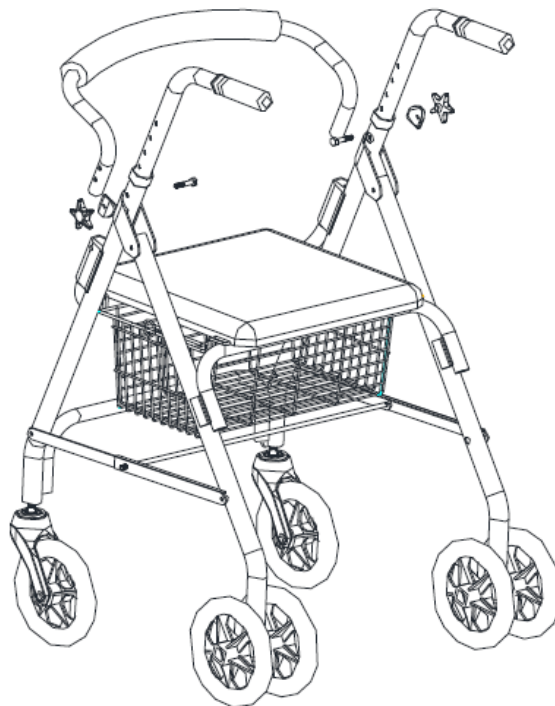
All wheels must always be in contact with the ground.

Take care when climbing kerbs or using sloping, uneven or slippery surfaces. Failure to do so may result in a serious risk of falling or injury.

Do not use the walker on wet, icy or snowy slopes.

If any parts are damaged, broken or worn, contact your dealer.

MOUNTING & USE

Lock all fasteners before each use.

CARE AND MAINTENANCE

Failure to care for and monitor your product may result in the risk of serious injury.

- Clean the rollator regularly with warm water and a soft cloth. Do not use abrasive products, detergents or solvents.
- Dry with a clean, dry soft cloth.
- Make sure there is no accumulation of dirt or debris around the axle, brakes or wheels.
- Do not use cleaning products containing phenol or chlorine, as this could damage aluminium and polyamide materials.
- Lubricate the wheels with a little oil if they squeak.
- Periodically check the wear and strength of all screws and bolts. Replace all broken, damaged or worn parts immediately.
- Check that the brakes are working before each use.
- If there is a malfunction, do not repair the rollator yourself. Consult an authorized dealer.
- **Checklist :**
To keep the rollator in good condition, check before each use, and check regularly as follows :

| Check the item | Anytime | Weekly | Monthly | Yearly |
|-------------------------------------|---------|--------|---------|--------|
| Clip and knob for height adjustment | | X | | |
| Structural weld at wheels | | X | | |
| Cleanliness | X | | | |
| Screws and fasteners | | X | | |
| Stability of frame and backrest | | | X | |
| Condition of seat | | | | X |
| Brakes | X | | | |

CHANGE OF OWNERSHIP

The product can be reused. It must first be cleaned and disinfected in accordance with hygiene instructions. When transferring the rollator, remember to hand over all the necessary technical documents to the new user. Beforehand, the product must be inspected by an authorised specialist.

RECYCLING

When the product has become unusable and you have to throw it away: please contact your specialist dealer. If you wish to carry out recycling for yourself, ask your local waste management company for disposal guidelines.

STORAGE

Failure to observe the storage conditions may lead to deterioration of the product and thus to the risk of serious injury.

Do not store your rollator for an extended period of time near a heat source or in the sun (e.g. behind a window or near a radiator) or near a cold source.

Keep away from all flames and sources of sparks.

Observe the conditions for storing and stowing your rollators.

- In a dry and temperate place
- Protect your rollator by packaging it from dust, corrosion (e.g. abrasive elements, sand, sea water, salty air).
- Store all removed parts together in the same place (or locate them if necessary) to avoid mixing them with other products when reassembling.

- All components must be stored without any load (do not place any objects that are too heavy on the rollator elements, do not jam anything...).

TECHNICAL SPECIFICATIONS

| | |
|------------------------|--|
| Overall Dimensions | Length 70 x Width 55 x Height 78-90 cm |
| Folded Dimensions | Length 58 x Width 21 x Height 79 cm |
| Seating Dimensions | Length 35 x Width 32 x Height 40 cm |
| Distance between grips | 45 cm |
| Weight | 7kg |
| Max. User Weight | 136 Kg |

PERIOD OF USE

The life expectancy of this product is 5 years under normal use, safety and maintenance conditions. Beyond this period, the product can be used for as long as it is still in good condition.

WARRANTY

This product is guaranteed for two years from the date of purchase by the user. IDENTITIES will provide replacement parts for any part that has broken due to a manufacturing, design or material defect. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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