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INSTRUCTIONS MANUAL EN

IMPORTANT

Read the instructions carefully before using this product and keep them in case of need.

User : If you are unable to read or understand the warnings, precautions or instructions, please contact your healthcare professional or distributor before using this product, as this may cause risks and damage. They will be able to advise and inform you on the most appropriate use.

Distributor: this leaflet must be given to users of this product.

PRODUCT DESTINATION

Positpro range cushions :

Technical positioning aid with memory effect for lying patients. This product is designed for patients at risk of bedsores. It prevents bedsores in immobilised patients lying down. The cushions in the POSITPRO range can be used to create positioning combinations for patients lying down to prevent pressure sores. They act by reducing pressure points at the head, elbow, sacrum and heel support points, as well as friction points on the knees, malleoli, calves and at-risk areas.

This device is intended for carers and must be prescribed by a doctor. Read each property of each cushion before use.

Gutter cushion for Positpro chair, Hemi Arm Positpro cushion:

POSITIONING aid for patients in wheelchairs. This product is designed for patients with hemiplegia of the arm.

Its curved shape supports the arm and relieves pressure on the forearm. It provides comfort when positioning the patient in a wheelchair.

This device is intended for use by carers and should be used on a case-by-case basis depending on the type of patient diagnosed (total, partial or proportional hemiplegia). It is monitored by the care team as part of the patient's care.

Positpro grey knee cushion :

Technical positioning aid with memory effect for lying patients. This product is designed for patients at risk of developing pressure sores. It prevents bedsores in immobilised patients lying down. It acts by reducing pressure and friction points on the knees.

This device is intended for carers and must be prescribed by a doctor. Read each property of each cushion before use.

Positpro grey heel release :

Technical positioning aid with memory effect for lying patients. This product is designed for patients at risk of developing pressure sores. Allows the heels to be relieved while the patient is lying down, in a lateral decubitus position or on their back. Attaches to the mattress with a strap.

PRECAUTIONS FOR USE

This product must be used on the advice of a healthcare professional following a medical prescription.
If the patient does not accept the product, do not use it. If the patient does not tolerate the product, do not use it.
If the patient experiences pain when using this product, do not use it.
Follow the instructions in the medical prescription and the planned treatment protocol.
Do not use a soiled or undecontaminated cushion.
Do not use the product if it is damaged - Check the seams before use.
Failure to observe the precautions for use may result in discomfort and injury to the hemiplegic patient.
If the patient does not accept the product, do not use it.

USE

Positpro range cushions/ Positpro grey knee cushion / Positpro grey heel release:

It is advisable to position the patient every 2 hours in a 30° supine position (semi-fowler) and in a 30° left and right lateral position. Read each cushion specification and possible combinations. Ask a healthcare professional for advice.
Clinical monitoring involves ensuring that the patient is properly hydrated and eating properly. The product is available in grey and aubergine (excluding 829118 and 829119)

Gutter cushion for Positpro chair, Hemi Arm Positpro cushion:

The arm cushion is used for patients with hemiplegia of the arm. It is placed on the armrests of the right or left armchair. It relieves pressure on the arm and forearm on the left and right. It provides comfort for the patient. It is used according to the degree of disability of the patient and the state of the muscles (stiff or flaccid). Ask a healthcare professional for advice.

Re-education is essential in the clinical follow-up of hemiplegic patients.

CHANGE OF PATIENT

When the product is to be re-used on another patient: it must be decontaminated and used in perfectly hygienic conditions.

When changing patient, remember to give this leaflet to the new user.

CARE AND MAINTENANCE

Positpro range cushions/ Gutter cushion for Positpro chair:

The STERITEX cover for the POSITPRO cushion can be decontaminated with a non-chlorinated disinfectant (spray, wipe). The cover should be machine-washed at 95°C with the zip closed. Disinfect the cushion after each use. Dry with a soft, clean, dry cloth. Do not use abrasive products, detergents or chlorinated products. In the event of wear or tear, do not attempt to repair the product. Dispose of it or contact your retailer to find out if it is covered by the after-sales service.

STERITEX cover

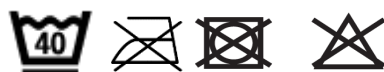
Cushion core

Knee cushion Positpro grey/ Heel pad Positpro grey:

When the product is to be re-used on another patient: it must be decontaminated and used in perfectly hygienic conditions. When changing patient, remember to give this leaflet to the new user. The product should be machine-washed at 40°C. Disinfect the cushion after each use. Dry with a soft, clean, dry cloth. Do not use abrasive products, detergents or chlorinated products. In the event of wear or tear, do not attempt to repair the product. Dispose of it or contact your retailer to find out if it is covered by the after-sales service.

Microbead cushions 829130 to 829142, 829146:

When the product is to be re-used on another patient: it must be decontaminated and used in perfectly hygienic conditions. When changing patients, remember to give this leaflet to the new user. Disinfect the cushion after each use. Dry with a soft, clean, dry cloth. Do not use abrasive products, detergents or chlorinated products. In the event of wear or tear, do not attempt to repair the product. Dispose of it or contact your retailer to find out if it is covered by the after-sales service.

		Machine wash at 95°C - Care and decontamination with a non-chlorinated bactericide			
		Bleaching prohibited			
		Moderate drying programme			
		Do not iron			
		Do not dry clean			

RECYCLING

When the product has become unusable and you need to dispose of it, contact a local waste disposal company.
If you wish to recycle the product, please enquire locally.

STORAGE

Failure to comply with storage conditions may lead to deterioration of the product and therefore a risk of injury.
The product can be stored between 0°C and 50°C.
Keep away from sources of ignition.
Observe the following conditions when storing your products

- In a dry, temperate place
- Protect your product with dust-proof packaging
- Do not place heavy objects on the cushion: the foam could settle and deteriorate.

TECHNICAL CHARACTERISTICS

Positpro range cushions :

Assembly: polyester-cotton thread stitching

Filling: Comfortable TECHNI GEL fibre, exceptionally soft and supple to the touch. 100% polyester with memory effect.

Core cover : Made from 100% cotton warp and weft fabric.

Cover in contact with the patient: Made from STERITEX technical textile, polyurethane coating, soft and comfortable to the touch (190g/m²).

STERITEX polyurethane is breathable and waterproof thanks to its 100% polyester bi-elastic knitted base. (190g/m² - Composition: 55% polyester - 45% polyurethane)

This product contains biocides in accordance with EU Standard 528/2012. Protection against microbiological deterioration.

Active ingredients: silverglass complex 398477-47-9 + zinc - pyrethione

The covers are fitted with a ZIP with flap for better protection of the cushion.

Gutter cushion for Positpro chair:

Assembly: polyester-cotton thread stitching

Filling: A combination of rigid polyether foam for attachment to the chair and visco-elastic foam for cushioning the arm.

Cover in contact with the patient: Made from a STERITEX technical textile with a polyurethane coating, soft and comfortable to the touch (190g/m²).

STERITEX polyurethane is breathable and waterproof thanks to its 100% polyester bi-elastic knitted base. (190g/m² - Composition: 55% polyester - 45% polyurethane). The cover is fitted with a zip for removal.

This product contains biocides in accordance with EU Standard 528/2012. Protection against microbiological deterioration.

Active ingredients: silverglass complex 398477-47-9 + zinc - pyrethione

Knee cushion Positpro grey/ Heel pad Positpro grey :

Assembly: polyester thread stitching

Filling: 100% polyester with memory effect.

Cover: 54% polyester - 46% polyurethane

This product contains biocides in accordance with EU Standard 528/2012. Protection against microbiological deterioration.

Active ingredients: silverglass complex 398477-47-9 + zinc - pyrethione

Microbead cushions (829130 to 829142, 829146) :

Assembly: by polyester thread stitching

Filling: Polystyrene

Textile composition: 56% polyester - 44% polyurethane

This product contains biocides in accordance with EU Standard 528/2012. Protection against microbiological deterioration.

Active ingredients: silverglass complex 398477-47-9 + zinc - pyrethione

Overall dimensions	829006 - 43 x 17 x 24 cm 829033 829033.A 829033.P - 190 x 30 cm
---------------------------	--

	829074 829074.A 829074.P - 190 x 30 cm 829075 829075.A 829075.P - 55 x 40 cm 829076 829076.A 829076.P - 27 x 40 cm 829077 829077.A 829077.P - 55 x 18 cm 829078 829078.A 829078.P - 30 x 18 cm 829079 829079.A 829079.P - 40 x 15 cm 829080 829080.A 829080.P - 50 x 25 cm 829081 829081.A 829081.P - 78 x 35 cm 829082 829082.A 829082.P - 30 x 15 cm 829084 829084.A 829084.P - 50 x 17 cm 829086 829086.A 829086.P - 57 x 23 cm 829098 829098.A 829098.P - 50 x 25 cm 829099 829099.A 829099.P - 50 x 25 cm 829118 - 35 x 22 cm 829119 - 90 x 20 cm 829130 - 40 x 55 cm 829131 - 40 x 30 cm 829132 - 70 x 17 cm 829133 - 30 x 17 cm 829134 - 60 x 30 cm 829135 - 110 x 60 x 24 cm 829136 - 130 x 18 cm 829137 - 30 x 20 x 15 cm 829138 - 34 x 24 x 10 cm 829139 - 200 x 35 cm 829140 - 190 x 35 cm 829141 - 70 x 48 cm 829142 - 85 x 20 cm 829146 - 55 x 35 cm
Weight	829006 - 0,300 kg 829033 829033.A 829033.P - 2,740 kg 829074 829074.A 829074.P - 2,000 kg 829075 829075.A 829075.P - 0,700 kg 829076 829076.A 829076.P - 0,720 kg 829077 829077.A 829077.P - 0,600 kg 829078 829078.A 829078.P - 0,720 kg 829079 829079.A 829079.P - 0,250 kg 829080 829080.A 829080.P - 1,300 kg 829081 829081.A 829081.P - 0,450 kg 829082 829082.A 829082.P - 0,350 kg 829084 829084.A 829084.P - 0,500 kg 829086 829086.A 829086.P - 0,600 kg 829098 829098.A 829098.P - 1,300 kg 829099 829099.A 829099.P - 1,900 kg 829118 - 0,250 kg 829119 - 0,700 kg 829130 - 0,400 kg 829131 - 0,400 kg 829132 - 0,300 kg 829133 - 0,500 kg 829134 - 0,900 kg 829135 - 0,500 kg 829136 - 0,400 kg 829137 - 0,300 kg 829138 - 0,300 kg 829139 - 1,600 kg 829140 - 1,200 kg 829141 - 1,200 kg 829142 - 0,500 kg 829146 - 1,200 kg

PERIOD OF USE

The expected life of this product is 2 years under normal use and maintenance. After this period, if the product has lost its memory properties, if the cover is damaged or if the Velcro fasteners are damaged: do not use this product. It can be used as long as it is still in good condition.

The lifespan of the product will vary according to the weight of the patients who use it and the number of times it is washed. The flattening of the product will be an indicator of its age.

WARRANTY

This product is guaranteed for two years from the date of purchase by the user against defects in stitching and workmanship. Contact the distributor from whom you purchased the product should you need to make a claim. Failures resulting from accidental damage, misuse, improper storage, improper unpacking, modifications or wear and tear are not covered by this warranty.

Contact the retailer from whom you purchased the product to apply the guarantee.

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