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INSTRUCTIONS MANUAL EN

IMPORTANT

Read the instructions carefully before using this product and keep them if necessary.

User: If you are unable to read or understand any warnings, precautions, or instructions, please contact your health care professional or distributor before using this product or risk and damage may result.

Consult your physician or health care professional before using any assistive device. He/she will advise and inform you on the most appropriate use in your case.

Distributor: this instruction manual must be given to the users of this product.

PRODUCT DESTINATION

The Samoa toilet chair is a medical device. It is a technical aid for toileting that is designed for people who are elderly, have reduced mobility, are disabled, or injured, or who can easily lose their balance.

SAFETY PRECAUTIONS

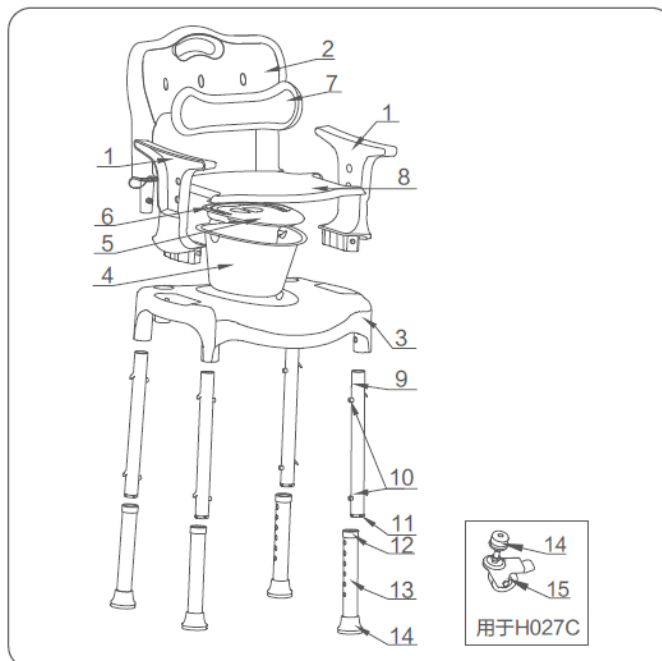
Failure to follow the safety precautions may result in the risk of serious injury.

- Never exceed the maximum supported weight of 130 kg. Make sure all components are secure before use.
- Ensure that the product is in good working order. The appliance is not suitable for users who cannot sit or stand normally.
- Visually inspect the product. Check the stability of the chair before use. Use only on level ground. Never bend over when using the chair. Do not adjust the height when the user is seated.
- Please adjust the height before use.
- If any parts are damaged, broken, worn or missing, contact your retailer.
- Before use, please check that all the product's fasteners are tight and that there is no play.
- It is recommended to have an accompanying person when using the toilet chair.
- When adjusting the height, make sure that all four legs are the same height and that the positioning pins are tight.
- When installing the backrest armrest and the top of the legs: make sure there is an audible "click" and check that they are securely fastened, to ensure safety.
- The foot caps on the 811192 toilet chair are made of plastic and should be replaced in good time if worn, to avoid accidents.
- Warning: People with mobility problems should not use this toilet chair.

PRODUCT DESCRIPTION

Parts list :

1. Armrest (including structural functions of armrest and armrest tube)
2. Backrest (including structural functions of backrest and backrest tube)
3. Chair seat (including the structural functions of the toilet chair support and toilet chair frame)
4. Toilet chair bucket
5. Toilet chair bucket cover
6. Toilet seat bucket handle
7. Back cushion
8. Seat cushion (including toilet chair cover structure function)
9. Upper leg tube
10. Positioning pin
11. Inner tube plug
12. Tube sleeve
13. Telescopic tube
14. Foot caps

MOUNTING & USE

The toilet chair has removable armrests and backrests, and its height can be adjusted in six stages.

Height adjustment: Press the protruding head of the positioning pin (10) with one hand and hold the telescopic tube (13) with the other hand, move it up and down to adjust it to a suitable position, and make sure that the protruding heads of the positioning pins (10) of the 4 legs are firmly attached to the holes of the same height to avoid any danger.

Installing and using the toilet chair 811192

- ① Insert the 4 upper leg tubes (9) into the holes in the seat legs (3) until they "click" into place. When the protruding head of the pin (10) pops out of the locating hole, this means that the leg tubes are installed.
- ② Insert the armrest (1) and backrest (2) into the seat fixing holes (3) in the same way as step ① until you hear a "click". When the protruding head of the single pin pops out of the locating hole, this means that the leg tubes are installed.
- ③ Place the toilet bucket components (4)(5)(6) in the center hole of the seat (3), then fit the seat cushion (8).
- ④ Fit the backrest cushion (7) into the 3 small oval holes in the backrest (2). Installation is now complete, and it can be used normally.
- ⑤ When using the toilet function, please remove the seat cushion (8) and toilet chair cover (5) and replace them after use to prevent unpleasant odors from feces and urine; when using the bath function (except for toilet chair 811192, it is recommended to remove the toilet bucket components (4)(5)(6).
- ⑥ When using the 811192 toilet chair, first tighten the four castors (15). Once the user is seated securely, release the foot brake, and the carer/guardian can push it to move it.

Removing the armrest and backrest: Two people are required. One person presses down on the armrest (1) and the pin protruding from the bottom of the backrest (2) with both hands, while the other person holds and slightly lifts the armrest (1) and backrest (2) to remove them.

Please follow all the instructions stated in this user manual when using the SAMOA toilet chair. This is for your security.

CARE AND MAINTENANCE

Failure to care for and maintain your product may result in the risk of serious injury.

- Regularly clean by hand the product (all castors, joints and fixings) with a damp cloth and disinfectant. Do not use abrasive products, detergents, or solvent-based products.
- Dry with a clean, dry soft cloth.
- The chair must be completely reviewed every year, with compulsory lubrication.
- It is recommended that the device is serviced annually. If the product is subject to intensive use and/or use in a multi-user environment, we recommend that it be serviced more frequently.
- In case of malfunction, do not repair the product yourself. Consult an authorised dealer.

Check the item	Any time	Weekly	Monthly	Yearly
Condition and operation of armrests, backrest and seat	X			
Condition and operation of bucket	X			
Positioning pin	X			

CHANGE OF OWNERSHIP

The product can be reused. It must first be cleaned and disinfected in accordance with hygiene instructions. When transferring the product, remember to give all the necessary technical documents to the new user. Beforehand, the product should be inspected by an authorized specialist.

RECYCLING

When the product has become unusable, and you must throw it away: please contact your specialist dealer. If you wish to carry out recycling for yourself, ask your local waste management company for disposal guidelines.

STORAGE

Failure to observe the storage conditions can lead to deterioration of the product and thus to the risk of serious injury.

- Do not store your product for prolonged periods near a source of heat or in direct sunlight (e.g. behind a window or near a radiator) or near a source of cold.
- Keep away from open flames and sources of sparks.
- Respect the conditions for storing your product.
- In a dry, temperate place
- Protect your product with packaging from dust and corrosion (e.g. abrasive elements, sand, sea water, salty air).
- Store all removed parts together, in the same place (or mark them if necessary) to avoid mixing them with other products when reassembling.
- All components must be stored without any load (do not place heavy objects on the product components, do not jam anything, etc.).

PERIOD OF USE

The life expectancy of this product is 5 years under normal use, safety and maintenance conditions for a user. Beyond this period, the product can be used for as long as it is still in good condition.

If the device is used in a multiuser environment (such as a nursing home or hospital) then service life may be reduced.

FEATURES TECHNICAL

Overall dimensions	Width 54 x depth 47 x height 75 to 87.7 cm
Seat dimensions	Width 47 x depth 43 x height 39 to 52 cm in 6 positions
Width between armrests	46 cm
Weight	5,1 kg
Maximum authorised weight	130 kg

GUARANTEE

This product is guaranteed for two years from the date of purchase by the user. IDENTITIES will provide replacement parts for any part that has broken due to a manufacturing, design or manufacturing deficiencies. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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