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EN

IMPORTANT

Read the instructions carefully before using this product and keep them if necessary.

User: If you are unable to read or understand any warnings, precautions, or instructions, please contact your health care professional or distributor before using this product or risk and damage may result.

Consult your physician or health care professional before using any assistive device. He/she will advise and inform you on the most appropriate use in your case.

Distributor: this instruction manual must be given to the users of this product.

PRODUCT DESTINATION

The Moovy adjustable toilet/shower and transfer chair is a medical device. It is a technical toilet aid for the elderly, people with reduced mobility, disabled, injured or who can easily lose their balance.

SAFETY PRECAUTIONS

Failure to follow the safety precautions may result in the risk of serious injury.

- Never exceed the maximum supported weight of 125 kg.
- Ensure that all components are securely in place before use.
- Ensure that the product is in good working condition.
- The device is not suitable for users who cannot sit and stand normally.
- Visually inspect the product.
- Before each use of the product, check the brakes and wheels.
- Check the stability of the chair before use.
- Use only on a flat surface.
- When stationary, be sure to engage the wheel brakes.
- Ensure that the brakes are unlocked before moving the chair again.
- For any transfers or when the chair is stationary, also engage the brakes.
- Never lean when you are on the chair.
- Check the wheel rotation and brakes.
- When the user sits or stands, keep the feet on the floor and not on the footrests.
- Please do not adjust the height while the user is seated.
- Adjust the height before use.
- Ensure that the wheels are locked when the patient sits in or gets out of the chair.
- Ensure that the wheels are locked when the patient is bathing.
- If it is necessary to leave the chair on an outdoor ramp, before pushing the chair, make sure that the folding handles are completely in place.
- Ensure that the folding handle is secured before pushing the chair.
- In case of damaged, broken, worn, or missing parts, contact the dealer.

PRODUCT DESCRIPTION

Product information

Dimensions :

Length : 770mm

Width : 590mm

Height : 930mm

Weight : 34kg

Braking system : Foot brake

MOUNTING & USE



MOUNTING :

1. The product that the user receives is a fully pre-installed finished chair. The user only needs to make appropriate adjustments according to the user's height and direction before using.
2. To use the commode, first close the left and right brackets, then put the support frame in place by inserting the two ends of it into the mounting holes under the seat and then align the two brinks of the bedpan with the slots on the support frame.
3. Installed into the infusion support socket, it can be disassembled and replaced freely.


USE :

1) Instructions of moving the patient:

1. Turning the manual handle to adjust the seat to a height of 1 CM higher than the furniture where the user is uprightly sitting, open the chair from behind and then maneuver it to the front of the user ; Place the user's feet on the footrests and lock the brakes before the user is seated in the chair.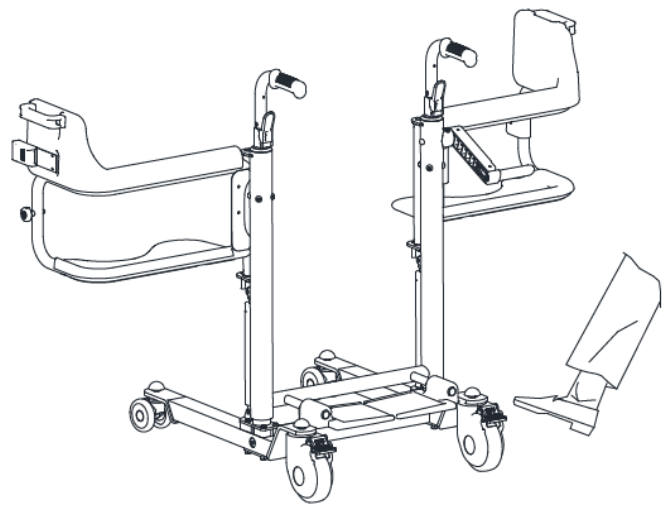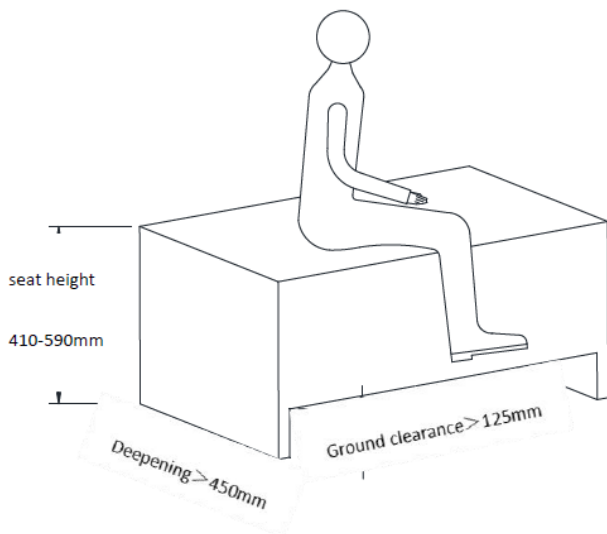

- Help the user sit in one half and then the other of the seat by leaning to the opposite side while closing the according half of seat assembly;
When the user is seated in the chair, close it from behind and lock the auto buckle and safety buckle.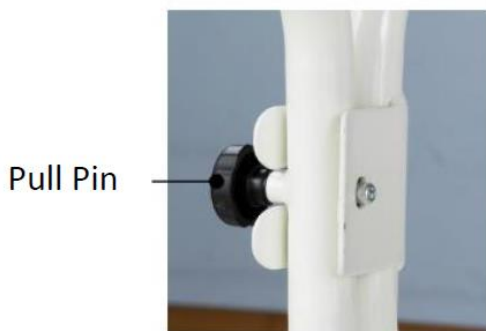

Auto buckle lock


Safety buckle lock

- Unlock the brakes when the patient is seated in the chair with double buckles locked. At this time, can move the patient by front handle.
- When the user is transferred to the desired spot, turn the manual handle to adjust the seat to a height of 1 CM higher than the desired furniture and then lock the brakes, unlock and open the chair from behind so as to allow the user sit in the desired spot smoothly.

2) How to adjust the height :

Turning the manual handle in a clockwise direction, and the transfer chair will go up; Turning in a counterclockwise direction, it will go down.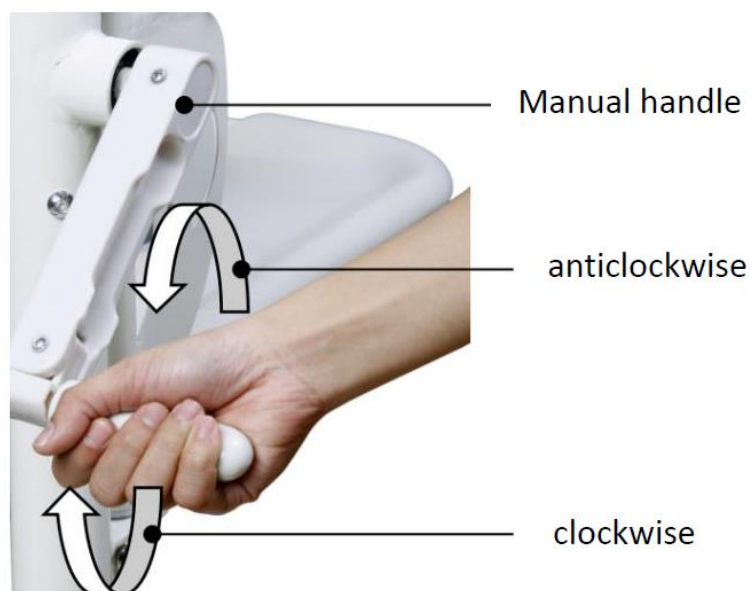

Manual handle

3) How to use the folding handle :

Hold the backfolde and bend down to fold; In the contrary, push it up, fixed when you hear the "click" sound.

Folding handle

Please follow all the instructions stated in this user manual when using the Moovy adjustable toilet/shower and transfer chair. This is for your security.

Notes : For safe and smooth using, the bedpan and bedpan holder should be installed only when needed.

CARE AND MAINTENANCE

Failure to care for and maintain your product may result in the risk of serious injury.

- Regularly clean by hand the product (all wheels, seats, and fixations) with a damp cloth and a mild detergent and disinfectant. Do not use abrasive products, detergents, or solvent-based products.

- Dry with a clean, dry soft cloth.
- The chair must be completely reviewed every year, with compulsory lubrication.
- It is recommended that the device is serviced annually.
- In case of malfunction, do not repair the product yourself. Consult an authorised dealer.

Check the item	Any time	Weekly	Monthly	Yearly
Four wheels	X			
Belt of protection	X			
Cleanliness	X			
Screws and fasteners		X		
Lifting column			X	
Frame stability			X	
State of the seat				X
Brakes			X	

CHANGE OF OWNERSHIP

The product can be reused. It must first be cleaned and disinfected in accordance with hygiene instructions. When transferring your product, remember to hand over all the necessary technical documents to the new user. Beforehand, the product must be inspected by an authorised specialist.

RECYCLING

When the product has become unusable, and you have to throw it away: please contact your specialist dealer. If you wish to carry out recycling for yourself, ask your local waste management company for disposal guidelines.

STORAGE

Failure to observe the storage conditions can lead to deterioration of the product and thus to the risk of serious injury.

- Do not store your product for an extended period of time near a heat source or in the sun (e.g. behind a window or near a radiator) or near a cold source.
- Keep away from all flames and sources of sparks.
- Respect the conditions for storing your product:
 - In a dry and temperate place
 - Protect your product from dust, corrosion (e.g. abrasive elements, sand, sea water, salty air).
- Store all removed parts together in the same place (or locate them if necessary) to avoid mixing them with other products when reassembling.
- All components must be stored without any load (do not place any objects that are too heavy on the product elements, do not jam anything...).

PERIOD OF USE

The life expectancy of this product is 5 years under normal use, safety and maintenance conditions for a user. Beyond this period, the product can be used for as long as it is still in good condition.

If the device is used in a multiuser environment (such as a nursing home or hospital) then service life may be reduced.

FEATURES TECHNICAL

Overall dimensions	Width. 59 cm x Depth. 77 cm x Height. 93/ 111,5 cm
Clearance width	41 cm
Weight	34 kg
Maximum authorised weight	125 kg

Tubes	Reinforced steel with standard thickness, compliant with EN standard
Seat surface	Supported by a steel plate and upholstered in medical ABS plastic
Backrest	PE blow moulded / PU foam moulded, environmentally friendly and waterproof
Pedal	Robust, durable engineering plastics
Handle	The articulated part of the folding handle is made of aluminium alloy
Patient support	Environmentally friendly rubber

GUARANTEE

This product is guaranteed for two years from the date of purchase by the user. IDENTITIES will provide replacement parts for any part that has broken due to a manufacturing, design or manufacturing deficiencies. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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