**COMPRESSE POIGNET
BUBBLE**

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**BUBBLE WRIST
COMPRESS**

User Manual

REF. 835169

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ENG

TECHNICAL SPECIFICATIONS

Unfolded length	31 cm
Unfolded width	16 cm
Compartments	6
Color	Green

DESCRIPTION AND USE

Description

Made from composite polyester fiber fabric, elastic band, and adhesive buckle.
External fixation for wrist fractures or soft tissue injuries.

Usage

1. Open the plastic bag and take out the wrist strap.
2. Place the upper round hole of the thumb fixing belt on the thumb, with a wider part wrapped around the wrist, and the other end wrapped around the lower part of the wrist and secured firmly with an adhesive buckle.
Ensure the tightness is comfortable.
3. If necessary, please follow the doctor's advice.

Dressing Procedure

1. Attach the wrist strap to your thumb.
2. Secure the Velcro firmly.
3. Overlap and wrap from your wrist to the base of your palm.
4. Start training.

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Precautions and Contraindications

1. Keep away from fire.
2. Fixation time should be determined according to the patient's needs. Please follow the doctor's advice for home use.
3. Do not use on patients with wrist skin lesions.

TRANSPORT AND STORAGE

Use general means of transportation. Ensure cleanliness during use, and when not in use, place it in a dry, ventilated area, being mindful of moisture.