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## **IMPORTANT**

Read this leaflet carefully before using this product and keep it for future reference.

User: If you are unable to read or understand any of the warnings, precautions or instructions, please contact your healthcare professional or distributor before using this product, as this may cause hazards and damage.

Consult a physician before using any medical device. He or she will advise and inform you on the most appropriate use of this product.

Distributor: This leaflet should be given to users of this product.

## **PRODUCT DESTINATION**

Technical positioning aid with memory effect for lying patients. These products are intended for patients at risk of pressure sores. They act to prevent pressure sores in patients who are lying down and immobilised. The cushions of the POSECO range allow to realize positioning combinations of the lying patient for the prevention of the pressure sore. They act by reducing the pressure points at the level of the head, elbows, sacrum, heels...but also the friction points of the knees, malleoli, calves and the areas at risk. This device is intended for caregivers and is subject to a medical prescription. Read each property of each cushion before use.

## **SAFETY PRECAUTIONS**

This product must be used upon medical advice and following a medical prescription.

Failure to follow the usage precautions may lead to injuries and deterioration to the pad properties.

Please follow the instructions of the medical prescription and care protocol.

- Please follow the doctor's instructions on the type of pad to use, which will vary according to the patient's condition.
- Do not use a soiled, contaminated pad.
- Do not use a damaged product – Check seams before use.
- If the patient does not tolerate the product, do not use it.

## **USE**

It is advisable to position the patient every 2 hours in a 30° supine position (semi-fowler) and in a 30° left and right lateral position. Read the individual characteristics of the cushions and the possible combinations. Seek advice from a health professional. Clinical monitoring includes proper hydration and dietary management of the patient.

## **CARE AND MAINTENANCE**

The Clinicare POSECO cushion can be decontaminated with a non-chlorinated disinfectant (spray, wipe).

Machine washable at 40°C.

Disinfect the cushion after each use. Dry with a clean, dry soft cloth. Do not use abrasive products, detergents or chlorinated products.

If the product is worn or torn, do not attempt to repair it. Dispose of it or contact your dealer to find out if it is covered by the After Sales Service.

## **CHANGE OF PATIENT OR BED**

The product can be reused. It must first be cleaned and disinfected in accordance with the hygiene and decontamination instructions. When changing patients, remember to give the instructions to the new user. Beforehand, the product should be inspected to check its good condition.

## **RECYCLING**

When the product has become unusable and you need to destroy it, contact a local waste disposal company.

If you want to recycle this product: ask your local authorities.

## **STORAGE**

Failure to observe the storage conditions may lead to deterioration of the product and thus to the risk of serious injury.

The product can be stored between 0°C and 50°C.

Keep away from flames and sparks.

- Respect the conditions for storing your products
- In a dry and temperate place
- Protect your product with a dust cover
- Do not place heavy objects on the cushion: the fiber could settle and degrade.

## TECHNICAL SPECIFICATIONS

**Assembly:** by polyester thread stitching

**Filling:** Comfortable TECHNI GEL fibre with an exceptionally soft and supple feel. 100% polyester with memory effect.

**Cover in contact with the patient:** Made of Clinicare, polyurethane coating, soft and comfortable to the touch (120g/m<sup>2</sup>) The polyurethane is breathable and waterproof thanks to its bi-elastic knitted base made of 100% polyester. (120g/m<sup>2</sup> - Composition: 42% polyester - 58% polyurethane). This product contains biocides in accordance with EU Standard 528/2012. Protection against microbiological deterioration.

Active ingredients: 3iodo-2-propynyl butylcarbamate

Overall Dimensions	POSECO universal cushion - 829110– 55 x 40 cm POSECO cylindrical cushion - 829111 –55 x 18 cm POSECO decubitus cushion - 829112– 190 x 30 cm POSECO half-moon cushion - 829113 – 190 x 30 cm POSECO buoy cushion - 829114– 50 x 17 cm
Weight	POSECO universal cushion - 829110 – 0,700 kg POSECO cylindrical cushion - 829111 – 0,600 kg POSECO decubitus cushion - 829112 – 2,740 kg POSECO half-moon cushion - 829113 – 2 kg POSECO buoy cushion - 829114 – 0,500 kg

## DURATION OF USE

The life expectancy of this product is 2 years under normal use, safety and maintenance conditions. Beyond this period, the product can be used for as long as it is still in good condition.

La durée de vie prévue de ce produit est de 2 ans en condition normale d'utilisation et d'entretien. After this period, if the product has lost its shape memory properties, if the cover is damaged or the velcro : do not use this product anymore. It can be used as long as it is still in good condition. The life span will vary according to the weight of the patients who will have used this product and according to the number of washings of the product. The flattening of the product will be the indicator of the product's age.

## WARRANTY

This product is guaranteed for two years from the date of purchase by the user, on the defects of sewing and confection. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, incorrect storage, incorrect unpacking, modifications, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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