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## INSTRUCTIONS MANUAL EN

## IMPORTANT

Read the instructions carefully before using this product and keep them if necessary.

User: If you are unable to read or understand any warnings, precautions or instructions, please contact your health care professional or distributor before using this product or risk and damage may result.

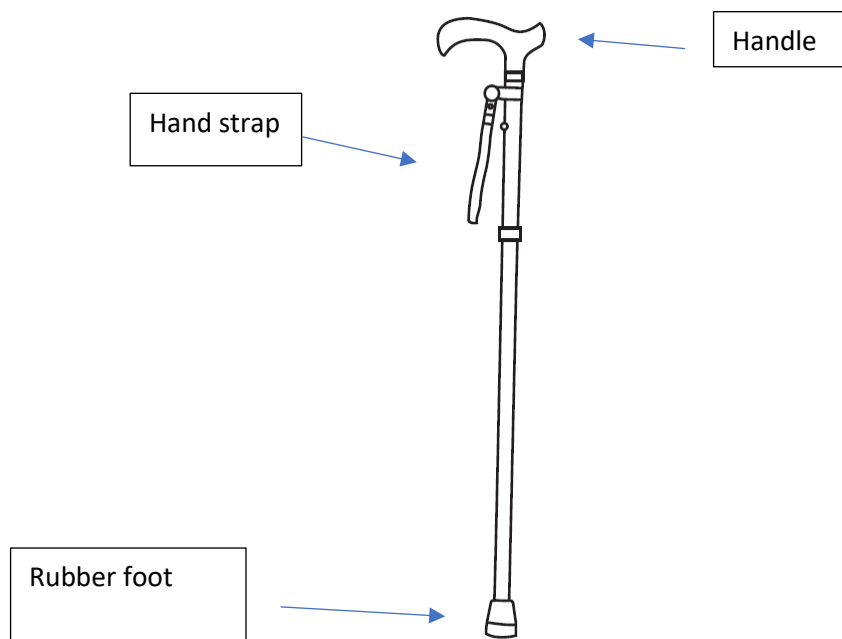
Consult your physician or health care professional before using any assistive device. He/she will advise and inform you on the most appropriate use in your case.

Distributor: this instruction manual must be given to the users of this product.

## PRODUCT DESTINATION

Folding walking sticks are walking aids for people with reduced mobility. They are foldable, made of aluminum and easy to transport.

## PRODUCT DESCRIPTION

## SAFETY PRECAUTIONS

Failure to follow the safety precautions may result in the risk of serious injury.

- Do not exceed the maximum weight limit of 110 kg.
- Only use the accessories and parts supplied by the manufacturer.
- Visually check the overall condition of the product (tube, handle, strap, stand).
- Make sure that the height adjustment clips are fully inserted before each use and the anti-noise screw tightened.
- Make sure that all parts are securely fastened and in good working order.
- Do not make any adjustments to the product while it is in use.
- Always use the handle for support.
- If any parts are damaged, broken or worn, contact your dealer.

## ASSEMBLY AND USE

### ASSEMBLY

1. To assemble the cane, allow the 3 lower catches to engage in the holes and fit the 4 parts together.
2. To fold the cane, press down on the 3 lower catches and pull the 4 parts apart to fold them. The catches may be a little stiff to prevent the cane from folding unintentionally.
3. To adjust the height, press the highest latch, pull or push the handle to one of the 5 desired heights and let the latch snap back into the corresponding hole.

### USE

To use the product, support yourself on the handle. Adjust the height if necessary. Move forward slowly.

## CARE AND MAINTENANCE

Failure to care for and maintain your product may result in the risk of serious injury.

- Clean the product regularly with lukewarm water and a soft cloth. Do not use abrasive products, detergents or solvent-based products.
- Dry with a soft, clean, dry cloth to prevent oxidation.
- Make sure there is no build-up of dirt or debris around the end caps.
- Do not use cleaning products containing phenol or chlorine, as this could damage aluminum and polyamide materials.
- Periodically check all rubber fixings, handles and foot for wear and strength.
- Before each use, check that the rubber end cap is in good condition.
- Before each use, check that the height adjustment clips are fully extended and the anti-noise screw tightened.
- In the event of a malfunction, do not repair the product yourself. Consult an authorized dealer.

## CHANGE OF OWNERSHIP

The product can be reused. It must first be cleaned and disinfected in accordance with hygiene instructions. When transferring your product, remember to hand over all the necessary technical documents to the new user. Beforehand, the product must be inspected by an authorised specialist.

## RECYCLING

When the product has become unusable and you have to throw it away: please contact your specialist dealer. If you wish to carry out recycling for yourself, ask your local waste management company for disposal guidelines.

## STORAGE

Failure to observe the storage conditions can lead to deterioration of the product and thus to the risk of serious injury.

- Do not store your product for an extended period of time near a heat source or in the sun (e.g. behind a window or near a radiator) or near a cold source.
- Keep away from all flames and sources of sparks.
- Respect the conditions for storing your product :
- In a dry and temperate place
- Protect your product from dust, corrosion (e.g. abrasive elements, sand, sea water, salty air).
- Store all removed parts together in the same place (or locate them if necessary) to avoid mixing them with other products when reassembling.
- All components must be stored without any load (do not place any objects that are too heavy on the product elements, do not jam anything...).

## FEATURES TECHNICAL

Overall dimensions	Length 83.5 to 95.5 cm in 5 positions
Weight	0,37 kg
Maximum supported weight	110 kg

## PERIOD OF USE

The life expectancy of this product is 5 years under normal use, safety and maintenance conditions for a user. Beyond this period, the product can be used for as long as it is still in good condition.

If the device is used in a multi user environment (such as a nursing home or hospital) then service life may be reduced.

## GUARANTEE

This product is guaranteed for two years from the date of purchase by the user. IDENTITIES will provide replacement parts for any part that has broken due to a manufacturing, design or manufacturing deficiencies. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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