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IMPORTANT

Read the instructions carefully before using this product and keep them if necessary.

User: If you are unable to read or understand any warnings, precautions or instructions, please contact your health care professional or distributor before using this product or risk and damage may result.

Consult your doctor or health care professional before using a walking aid. He/she will advise and inform you on the most appropriate walking technique for you.

Distributor: this leaflet must be given to the users of this product.

SYMBOL APPEARING ON THE LABEL OR IN THE INSTRUCTION MANUAL


	Manufacturer's address		Invitation to consult the user manual
	Date of production		Follow the safety instructions
	Batch Number		CE Conformity
	Product Reference		Does not tolerate dampness
	Max Weight supported		Medical Device

PRODUCT DESTINATION

The Neo Classic Rollators are walking aids for people with reduced mobility. These products are intended for use indoors and outdoors and can be folded up for easy storage or transport.

PRODUCT DESCRIPTION

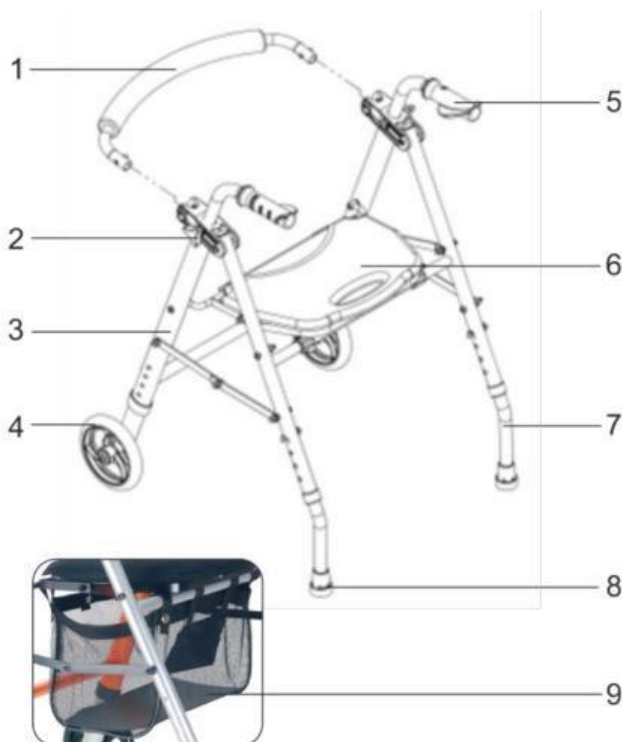1. Backrest
2. Button for adjustment, on the handles
3. Chassis
4. Front wheels fork
5. Handle
6. Seat
7. Rear foot
8. Anti-slip sole
9. Basket

SAFETY PRECAUTIONS

Failure to follow the safety precautions may result in the risk of serious injury.

Do not exceed the maximum weight limit of 136 Kg.

Do not exceed the maximum load limit in the basket (5kg). Loads must only be carried in the basket. Do not hang heavy objects on the rollator as this could affect its stability.

Make sure that all parts are securely fastened and in good working order.

Regularly check the screws and fasteners of the rollator and at the same height.

Make sure that the handle locking latch is securely engaged.

Do not use the rollator in stairs or escalators.

Use only accessories and parts supplied by the manufacturer.

The rollator is a walking aid **only. It is not a wheel chair.** It is not designed to carry a person. **Such usage can damage the wheels and the rollator structure.** The rollator could then be weakened and no longer meet the mechanical features.

Only use the rollator as a walking aid.

Do not use the rollator for walking backwards.

Before seating, make sure that the bars locking latch are fully engaged. Serious injuries to the user and/or chassis damages and /or wheels can result from the incorrect usage.

Do not make any adjustments to the rollator while in use.

All wheels must always be in contact with the ground.

Make sure to put both hands on the handles so that the rollator is in an even position of strength.

Make sure that the seat is fully locked in a flat position before use.

Do not try to push the rollator when someone is seated on it.

Do not use the seat to carry objects.

Do not use the seat without its backrest.

Do not seat on the seat when the rollator is on a slope.

While seated, do not try to reach objects by leaning forward. Do not lean backwards, the rollator could then tilt.

Use caution when climbing pavements or walking on inclined, uneven, or slippery surfaces. Failure to do so may result in a significant risk of falling or injury.

Do not use the walker on a wet, icy, or snowy slope.

If any parts are damaged, broken or worn, contact the distributor.

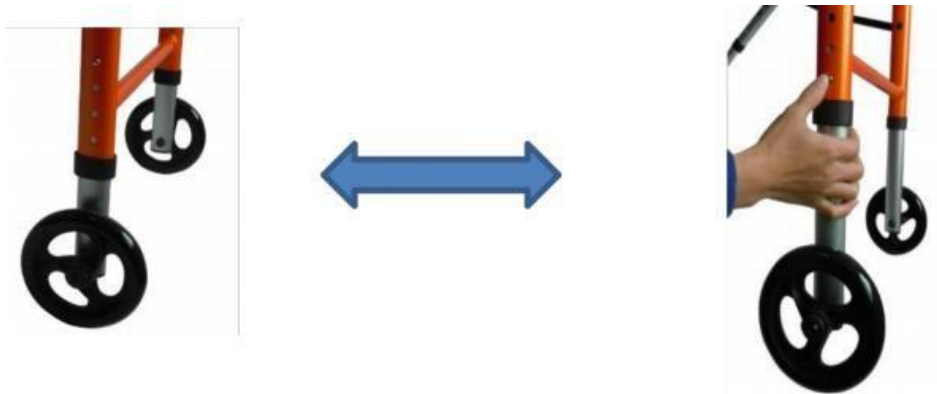
ASSEMBLY

1. Remove all packaging from rollator.
2. Remove the chassis and put it in front of you.
3. Unfold the structure and lower the seat.
4. Join the (oval) tubes of the front wheels and the (round) tubes of the rear feet into the structure, and adjust to the right height, making sure of the perfect locking.

ATTENTION : The seat height can be adjusted independently from the height of the front wheel tubes and the rear feet tubes.

It is essential that both front wheels are adjusted on the same height to avoid imbalance. The same applies to the rear feet.

5. Install the handles to the desired height and lock with the adjuster as shown below.

ATTENTION : To adjust the handle heights, unscrew the adjuster, adjust the height, then screw the adjuster. **Always adjust the two handles to the same height to avoid any imbalance.**

We recommend to place the handles at the user's wrist height

6. Install the backrest onto the chassis by pressing the button as shown below

SEAT

The seat is adapted for short rests and is not suitable for normal seating.

Check that the rollator is on an even surface, the front wheels and rear feet are perfectly in contact with the ground before seating.

FOLDING

The rollator can be folded for storage or transportation.

Pull seat upwards. It will fold completely the rollator. Mind your fingers during folding.

CARE AND MAINTENANCE

Failure to care for and monitor your product may result in the risk of serious injury.

- Clean the rollator regularly with warm water and a soft cloth. Do not use abrasive products, detergents or solvents.
- Dry with a clean, dry soft cloth.
- Make sure there is no accumulation of dirt or debris around the axle, brakes or wheels.
- Do not use cleaning products containing phenol or chlorine, or any solvent based products, as it could damage the aluminium and polyamide materials.
- Lubricate the wheels with a little oil if they squeak.
- Periodically check the wear and strength of all screws and bolts.
- Check that the brakes are working before each use.
- Check the parking brakes by pushing the handles into the locked position, then trying to push the rollator.
- If there is a malfunction, do not repair the rollator yourself. Consult an authorized dealer.
- **Checklist :**
To keep the rollator in good condition, check before each use, and check regularly as follows

Check the item	Anytime	Weekly	Monthly	Yearly
Clip and knob for height adjustment	X			
Structural weld at wheels		X		
Cleanliness				X
Screws and fasteners		X		
Stability of frame and backrest			X	
Condition of seat			X	
Condition of the wheels and pads		X		

CHANGE OF OWNERSHIP

The product can be reused. It must first be cleaned and disinfected in accordance with hygiene instructions. When transferring the rollator, remember to hand over all the necessary technical documents to the new user. Beforehand, the product must be inspected by an authorised specialist.

RECYCLING

When the product has become unusable and you have to throw it away: please contact your specialist dealer. If you wish to carry out recycling for yourself, ask your local waste management company for disposal guidelines.

STORAGE

Failure to observe the storage conditions may lead to deterioration of the product and thus to the risk of serious injury.

Do not store your rollator for an extended period of time near a heat source or in the sun (e.g. behind a window or near a radiator) or near a cold source.

Keep away from all flames and sources of sparks.

Observe the conditions for storing and stowing your rollators.

- In a dry and temperate place
- Protect your rollator by packaging it from dust, corrosion (e.g. abrasive elements, sand, sea water, salty air).
- Store all removed parts together in the same place (or locate them if necessary) to avoid mixing them with other products when reassembling.

- All components must be stored without any load (do not place any objects that are too heavy on the rollator elements, do not jam anything...).

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Seat dimensions	Width 36 x depth 24 x height 47 to 54 cm
Overall dimensions	Width 52 to 57 x depth 59 to 65 x height 80 to 97 cm
Wheel diameter	15cm
Weight	5.9 kg
Maximum load	136 kg
Maximum load for the basket	5 kg

PERIOD OF USE

The life expectancy of this product is 5 years under normal use, safety and maintenance conditions. Beyond this period, the product can be used for as long as it is still in good condition.

WARRANTY

This product is guaranteed for two years from the date of purchase by the user. IDENTITIES will provide replacement parts for any part that has broken due to a manufacturing, design or material defect. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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