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EN

IMPORTANT

Read this leaflet carefully before using this product and keep it for future reference.

User: If you are unable to read or understand any of the warnings, precautions or instructions, please contact your healthcare professional or distributor before using this product, as this may cause hazards and damage.

Consult a healthcare professional before using any medical device. He or she will advise and inform you on the most appropriate use of this product.

Distributor: This leaflet should be given to users of this product.

PRODUCT DESTINATION

GelPro compresses are medical devices for the treatment and relief of knuckle and muscle pain. They are applied directly to the painful area. Cold packs provide long-term relief from moderate to severe pain. Warm compresses provide short-term relief for mild and chronic pain.

SAFETY PRECAUTIONS

Failure to follow the safety precautions may result in the risk of serious injury.

- Each time you use it, check the condition of the compress, especially if the envelope is not pierced.
- Do not cut the product.
- People who are allergic to the external materials of this product (Chinlon, Spandex) should not use it on damaged skin.
- Do not use on sensitive areas.
- This product is for external use only; do not swallow.
- Children should use it under adult supervision.
- Avoid use for those with terminal circulatory dysfunction or those lacking autonomy.
- Consult a doctor before use if you are diabetic, have poor circulation, heart disease, rheumatoid arthritis, or if you are pregnant.
- Be careful when removing from the microwave; the pack may be very hot. If it is too hot, set it aside until it cools sufficiently for comfortable application. Always test the pack temperature before application.
- To avoid frostbite, which can occur if the pack is too cold for your skin, set it aside until it reaches a tolerable temperature. If the pack is not cold enough, place it in the freezer for an additional 20 minutes.
- Apply heat/cold to the affected area for no more than 20 minutes at a time, with 20-minute breaks between applications.
- Caution: Heating longer than recommended may result in physical injury or damage to your microwave.
- Do not wear while sleeping. Check the skin frequently during use. If you notice any irritation or burns, remove the product immediately.
- DO NOT use if the hot/cold pack contents leak and/or if the cover is damaged or torn. If the hot/cold pack contents come into contact with your eyes, remove the cover, rinse the affected area with water, and seek professional assistance immediately.
- DO NOT apply with pain-relieving creams, medicated lotions, or ointments.
- DO NOT use on unhealthy, damaged, or broken skin, or use hot on bruised or swollen areas that occurred within the last 48 hours, or on areas of the body where you cannot feel heat or cold, or with other forms of heat or cold.
- DO NOT use on individuals who cannot remove the product independently, such as children, infants, some elderly people, or those unable to follow all usage instructions.
- DO NOT apply additional pressure on the product.
- DO NOT overheat.
- DO NOT swallow the products or their contents. Keep out of reach of children and pets.
- Stop use and consult a doctor if pain worsens or persists for more than 7 days, or if you experience any discomfort, burning, swelling, rash, or other changes in your skin.

USE

Cooling Instructions

1. Store the pack in the freezer with storage bag so that it is always ready for cold application. Refrigerate the pack for at least 2 hours before use.
2. Check to make sure the temperature of the pack is suitable before application.
3. If it is too cold, better wrap it in towel or any soft cloth before applying on skin.

Microwave Method

1. Start with a room temperature pack.
2. Place the pack in the center of the clean microwave turntable (without touching the sides). Ensure that all additional microwave features, such as "browning," are turned off.
3. Heat the pack in 20-second intervals on medium power until it reaches the desired temperature. After removing it from the microwave, knead and flex the gel pack to evenly distribute the heat.
4. Check that the pack's temperature is suitable before application.
5. If more heat is needed, reheat in 10-second increments until the desired comfortable temperature is reached. Before reheating, check if any parts of the pack are too hot (this usually occurs at the edges of the pack), knead the pack to even out the temperature, rotate the pack 180 degrees, and reheat in the microwave. NEVER OVERHEAT.

Treatment

1. Choose the appropriate product.
2. Check if the compress is damaged. If it is damaged, do not use this product.
3. According to the treatment purpose and the cooling/microwave heating instructions mentioned above, cool/heat the product.
4. Place the product on the affected area for a maximum of 20 minutes, with 20-minute breaks between applications.

To ensure a long service life, check the condition of the product, including the presence of holes, before each use.

CARE

Failure to care for your product may result in the risk of serious injury.

- Cleaning: After use, wipe with a damp cloth and allow the packs to air dry completely before reusing. The pack is not machine washable.
- Do not use alcohol or other solvents to clean the compress.
- In case of wear or tear, do not attempt to repair the product: discard it or contact an authorized dealer to check if it can be handled by IDENTITES' customer service.

CHANGE OF PATIENT

The product can be reused. It must first be cleaned and disinfected in accordance with the hygiene and decontamination instructions. When changing the bed, remember to give the instructions to the new user. Beforehand, the product should be inspected to check its good condition.

RECYCLING

When the product has become unusable and you have to throw it away: please contact your specialist dealer. If you wish to carry out recycling by yourself, ask your local waste management company for disposal guidelines.

STORAGE

Failure to observe the storage conditions may result in deterioration of the product and therefore risk of serious injury

Keep away from flames and sparks.

Seal the packaging in a bag and store it at room temperature.

Observe the conditions for storing your products:

- In a dry and temperate place
- Protect your product with a dust cover
- Do not place heavy loads on it.

FEATURES TECHNICAL

The product consists of an outer packaging and an inner gel. The outer shell is made of 82% Chinlon and 18% Spandex. The gel is composed of 89% Polyethylene glycol, 10% Propoxylated Bis(2-ethylhexyl) terephthalate, and less than 1% Bismuth citrate.

Overall dimensions	835187 – 17 x 10.5 cm 835188 – 24 x 14 cm. Velcro strap length: 33 cm 835189 – 25.5 x 15 cm 835190 – 28 x 17.5 cm 835191 – 34 x 24 cm. Circumference: 103.5 cm, but very stretchy 835192 – 29 x 35 cm. Strap: 76 cm, very stretchy
Weight	835187 – 0,150 kg 835188 – 0,520 kg 835189 – 0,590 kg 835190 – 0,850 kg 835191 – 0,650 kg 835192 – 0,790 kg

PERIOD OF USE

The expected lifetime of this product is 5 years under normal use and maintenance. After this period, the product can be used for as long as it is still in good condition.

WARRANTY

This product is guaranteed for 1 year from the date of purchase by the user. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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