 **COMPRESSE THERMO-CRYO 12 x 29 CM, 20 x 29 CM**
Notice d'utilisation - DISPOSITIF MEDICAL

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EN

IMPORTANT

Read this leaflet carefully before using this product and keep it for future reference.

User: If you are unable to read or understand any of the warnings, precautions or instructions, please contact your healthcare professional or distributor before using this product, as this may cause hazards and damage.

Consult a healthcare professional before using any medical device. He or she will advise and inform you on the most appropriate use of this product.

Distributor: This leaflet should be given to users of this product.

PRODUCT DESTINATION

GelPro compresses are medical devices for the treatment and relief of knuckle and muscle pain. They are applied directly to the painful area. Cold use provides long-term relief from moderate to severe pain. Hot use provides short-term relief for mild and chronic pain.

SAFETY PRECAUTIONS

Failure to follow the safety precautions may result in the risk of serious injury.

- Each time you use it, check the condition of the compress, especially if the envelope is not pierced.
- Do not cut the product.
- People who are allergic to the external materials of this product (polyethylene nylon) should not use it on damaged skin.
- Do not use on sensitive areas.
- This product is for external use only; do not swallow.
- Children should use it under adult supervision.
- Avoid use for those with terminal circulatory dysfunction or those lacking autonomy.
- Consult a doctor before use if you are diabetic, have poor circulation, heart disease, rheumatoid arthritis, or if you are pregnant.
- Be careful when removing from the microwave; the pack may be very hot. If it is too hot, set it aside until it cools sufficiently for comfortable application. Always test the pack temperature before application.
- To avoid frostbite, which can occur if the pack is too cold for your skin, set it aside until it reaches a tolerable temperature. If the pack is not cold enough, place it in the freezer for an additional 20 minutes.
- Apply heat/cold to the affected area for no more than 20 minutes at a time, with 20-minute breaks between applications.
- Caution: Heating longer than recommended may result in physical injury or damage to your microwave.
- Do not wear while sleeping. Check the skin frequently during use. If you notice any irritation or burns, remove the product immediately.
- DO NOT use if the hot/cold pack contents leak and/or if the cover is damaged or torn. If the hot/cold pack contents come into contact with your eyes, remove the cover, rinse the affected area with water, and seek professional assistance immediately.
- DO NOT apply with pain-relieving creams, medicated lotions, or ointments.
- DO NOT use on unhealthy, damaged, or broken skin, or use hot on bruised or swollen areas that occurred within the last 48 hours, or on areas of the body where you cannot feel heat or cold, or with other forms of heat or cold.
- DO NOT use on individuals who cannot remove the product independently, such as children, infants, some elderly people, or those unable to follow all usage instructions.
- DO NOT apply additional pressure on the product.
- DO NOT overheat.
- DO NOT swallow the products or their contents. Keep out of reach of children and pets.
- Stop use and consult a doctor if pain worsens or persists for more than 7 days, or if you experience any discomfort, burning, swelling, rash, or other changes in your skin.

USE

Cooling Instructions

1. Keep the pack in the freezer with a storage bag so it is always ready for cold application. Refrigerate the pack for at least 2 hours before use.
2. Check that the pack's temperature is adequate before application.
3. If the pack is too cold, it is best to wrap it in a towel or soft cloth before applying it to the skin.

Microwave Method

1. Start with a pack at room temperature.
2. Make sure the gel is evenly distributed.
3. Heat in microwave for 15 seconds. Microwaves heat gel unevenly and you may find some points of the gel pack hotter than others. If more heat required, reheat for 5 seconds intervals until the desired temperature is reached.
4. Insert the gel pack into the wrap before using.

Treatment:

1. Select the appropriate product.
2. Check if the pack is damaged. If it is damaged, do not use the product.
3. According to the treatment purpose and the cooling/heating instructions above, cool or heat the product.
4. Place the product on the affected area for up to 20 minutes, with 20-minute breaks between applications.
5. If it is too hot or too cold, it is best to wrap it in a towel or soft cloth before applying it to the skin.

To ensure a long service life, check the condition of the product, including the presence of holes, before each use.

CARE

Failure to care for your product may result in the risk of serious injury.

- Cleaning: After use, wipe with a damp cloth and allow the packs to air dry completely before reusing. The pack is not machine washable.
- Do not use alcohol or other solvents to clean the compress.
- In case of wear or tear, do not attempt to repair the product: discard it or contact an authorized dealer to check if it can be handled by IDENTITES' customer service.

CHANGE OF PATIENT

The product can be reused. It must first be cleaned and disinfected in accordance with the hygiene and decontamination instructions. When changing the bed, remember to give the instructions to the new user. Beforehand, the product should be inspected to check its good condition.

RECYCLING

When the product has become unusable and you have to throw it away: please contact your specialist dealer. If you wish to carry out recycling by yourself, ask your local waste management company for disposal guidelines.

STORAGE

Failure to observe the storage conditions may result in deterioration of the product and therefore risk of serious injury

Keep away from flames and sparks.

Seal the packaging in a bag and store it at room temperature.

Observe the conditions for storing your products:

- In a dry and temperate place
- Protect your product with a dust cover
- Do not place heavy loads on it.

FEATURES TECHNICAL

The product consists of an outer packaging and an inner gel. The outer shell is made of nylon polyethylene and fleece fabric. The gel consists of 78.5% purified water, 20% glycerine and 1.5% sodium carboxymethylcellulose.

Overall dimensions	835200 – 12 x 29 cm 835201 – 20 x 29 cm
Weight	835200 – 0,33 kg 835201 – 0,55 kg

PERIOD OF USE

The expected lifetime of this product is 5 years under normal use and maintenance. After this period, the product can be used for as long as it is still in good condition.

WARRANTY

This product is guaranteed for 1 year from the date of purchase by the user. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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