**FAUTEUIL DE TOILETTES ENFANT ULYSSE***Notice d'utilisation* - DISPOSITIF MEDICAL**CHILDREN'S TOILET COMMODE CHAIR ULYSSE***Instructions manual* - MEDICAL DEVICE**ASIENTO DE INODORO INFANTIL ULYSSE***Manual de instrucciones* - DISPOSITIVO MEDICO**REF. 811193**

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INSTRUCTIONS MANUAL EN

IMPORTANT

Read the instructions carefully before using this product and keep them if necessary.

User: If you are unable to read or understand any warnings, precautions or instructions, please contact your health care professional or distributor before using this product or risk and damage may result.

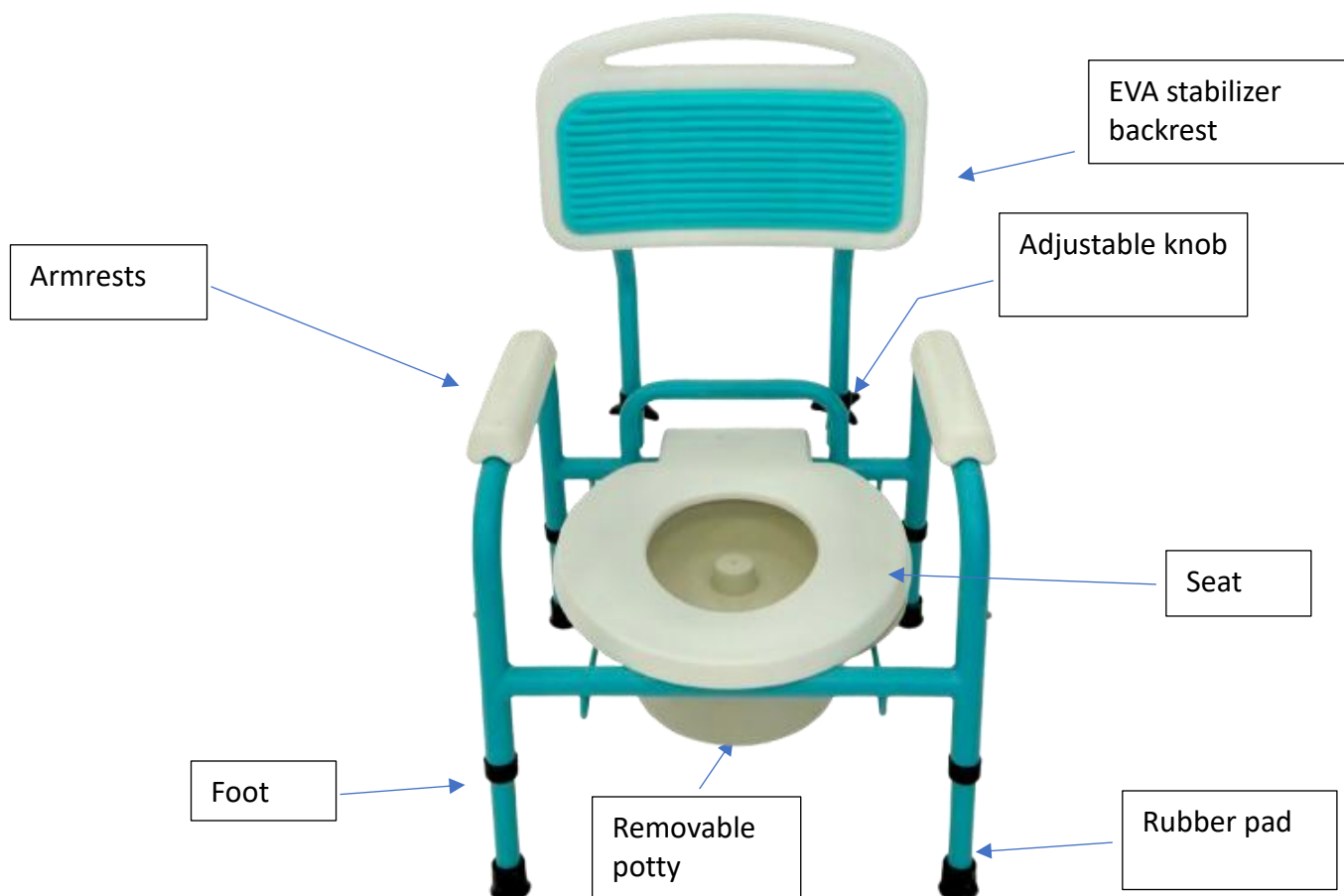
Consult your physician or health care professional before using any assistive device. He/she will advise and inform you on the most appropriate use in your case.

Distributor: this instruction manual must be given to the users of this product.

PRODUCT DESTINATION

The children's toilet commode chair Ulysse is a medical device. It is an assistive toileting aid designed for children under 1.4 meters in height. It provides comfortable positioning and facilitates toilet use for children with disabilities or those who may easily lose balance. The chair is equipped with a removable potty.

PRODUCT DESCRIPTION

SAFETY PRECAUTIONS

Failure to follow the safety precautions may result in the risk of serious injury.

- Never exceed the maximum supported weight of 100 kg.
- Ensure all components are secured before use.
- Make sure the product is in good working condition.
- Visually inspect the product.
- Check the stability of the chair before each use.
- Before each use, make sure to secure the backrest by tightening the knobs.
- Use only on a flat surface.
- Before each use, check that all four legs are at the same height and that the clips are fully engaged.
- Never lean while seated on the chair. The risk of tipping increases with heavy, unilateral loads.
- Do not stand on the commode chair—slipping hazard!
- Be cautious of pinching hazards during use.
- In case of damaged, broken, worn, or missing parts, contact the retailer.
- Any serious incidents related to the product must be reported to the manufacturer and to the competent authority in the member state where the user and/or patient is located.
- Ensure that the height-adjustment buttons fully protrude through the corresponding hole on each leg extension. This will ensure the leg extensions are securely locked in position and that the height is even.
- Check that the rubber tips on the leg extensions are neither torn, worn, nor missing. Immediately replace any tips showing these defects.
- All four leg extensions with rubber tips MUST touch the floor simultaneously at all times.

ASSEMBLY & USE

- Carefully unpack and examine the product to ensure that it is complete.
- Before use, check for any signs of damage, such as cracked welding, peeling paint, cracked or damaged seat or bent tube. If you notice any of these problems, please contact us and refrain from using the product.

ASSEMBLY :

- Before use, ensure that the chair is stable on a flat, horizontal surface.
- Before assembling the seat and lid, allow them to reach room temperature if they have been exposed to cold. This will prevent the seat and lid clamps from cracking when installing onto the commode's back cross brace.
- Align the seat clamps with the back rail of the frame. (Both clamps should be positioned outside of the pail brackets). Press down on the seat until the clamps snap firmly into place.
- Install the commode bucket by placing it into the designated frame. To remove, reverse the process.
- Install the backrest by sliding it into the frame and tightening the knob. Before using the chair, ensure the knob is securely tightened.
- Install the legs by pressing the push buttons and sliding them into the frame. Once locked into position, the push buttons will "CLICK" as they pass through the frame.

Adjusting the Leg Height

- Before use, adjust the height of each leg according to your needs, ensuring that all legs are at the same height. Once the desired height is reached, lock the legs by aligning the buttons with the appropriate holes.

USE :

- Do not allow children or unauthorized persons to play with the equipment or use it without adequate supervision.
- Do not sway back and forth or sideways while using the product, as this could cause excessive stress and lead to damage and failures that may result in injury.
- Do not use the product outside the home or on uneven surfaces (such as in the garden), as this could put stress on the legs and potentially damage them.
- Do not place fingers or foreign objects in the exposed tubular sections, as this may cause serious injury.

CARE AND MAINTENANCE

Failure to care for and maintain your product may result in the risk of serious injury.

- Regularly clean the product with a damp cloth and disinfectant. Do not use abrasive products, detergents, or solvent-based cleaners.
- Dry with a clean, soft, dry cloth.
- The chair should be completely serviced every month.
- It is recommended to perform maintenance on the product at least once a month. If the product is subjected to heavy use and/or used in a multi-user environment, we recommend having it serviced more frequently.
- Before each use, check the condition and operation of the following items: seat, backrest, legs, knobs, pads, and bucket.
- In case of malfunction, do not attempt to repair the product yourself. Consult an authorized dealer.

CHANGE OF OWNERSHIP

The product can be reused. It must first be cleaned and disinfected in accordance with hygiene instructions. When transferring your product, remember to hand over all the necessary technical documents to the new user. Beforehand, the product must be inspected by an authorised specialist.

RECYCLING

When the product has become unusable and you have to throw it away: please contact your specialist dealer. If you wish to carry out recycling for yourself, ask your local waste management company for disposal guidelines.

STORAGE

Failure to observe the storage conditions can lead to deterioration of the product and thus to the risk of serious injury.

- Do not store your product for an extended period near a heat source or in the sun (e.g. behind a window or near a radiator) or near a cold source.
- Keep away from all flames and sources of sparks.
- Respect the conditions for storing your product:
 - In a dry and temperate place
 - Protect your product from dust, corrosion (e.g. abrasive elements, sand, sea water, salty air).
- Store all removed parts together in the same place (or locate them if necessary) to avoid mixing them with other products when reassembling.
- All components must be stored without any load (do not place any objects that are too heavy on the product elements, do not jam anything...).

PERIOD OF USE

The life expectancy of this product is 5 years under normal use, safety and maintenance conditions for a user. Beyond this period, the product can be used for as long as it is still in good condition.

If the device is used in a multi user environment (such as a nursing home or hospital) then service life may be reduced.

FEATURES TECHNICAL

Overall dimensions	Width: 47 cm x Depth: 45 cm x Height: 71 to 81 cm
Seat dimensions	Width: 34 cm x Depth: 36 cm x Height: 26 to 36.5 cm in 3 positions
Backrest dimensions	37 x 122 cm
Height from the bottom of the backrest to the seat	22 cm
Width between armrests	34 cm
Cutout	20 x 16 cm
Weight	5,2kg
Maximum allowed weight	100kg

GUARANTEE

This product is guaranteed for two years from the date of purchase by the user. IDENTITIES will provide replacement parts for any part that has broken due to a manufacturing, design or manufacturing deficiencies. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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