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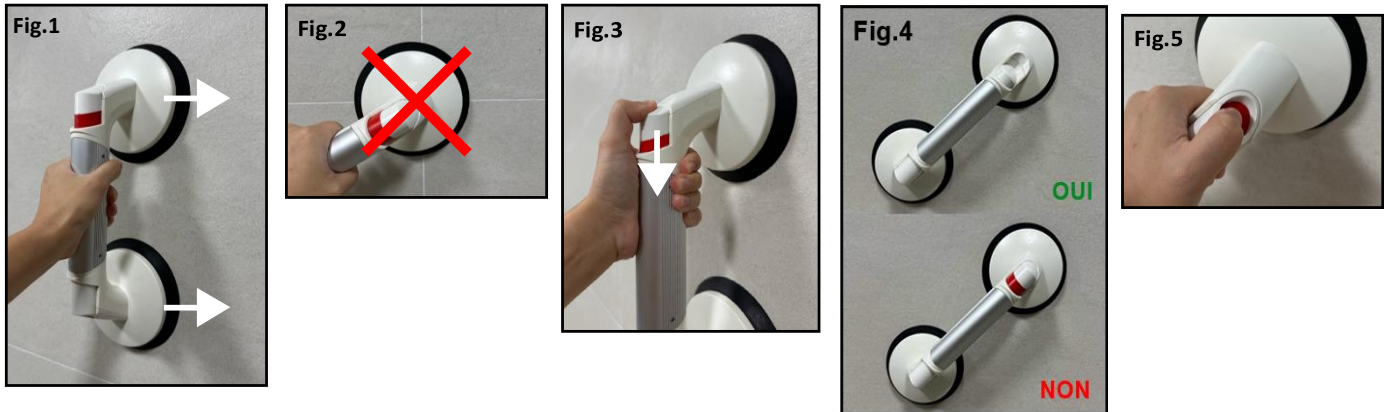
EN

## IMPORTANT

Please Read this manual carefully before using this product and keep it for future reference.

User: If you are unable to read or understand the warnings, precautions, or instructions, please contact your healthcare professional or distributor before using this product to avoid risks and damages.

Consult your doctor or healthcare specialist before using an assistive device. They will advise and inform you on the most appropriate use for your situation.

Distributor: This manual must be provided to users of this product.

THIS ITEM MAY BE USED TO SUPPORT HEAVY LOADS. IMPROPER USE MAY IMPAIR ITS PERFORMANCE AND PERMANENTLY DAMAGE IT.

The maximum weight supported by the bar is 60 kg.

WARNING: THE SUCTION GRAB BAR MUST BE USED AS A SUPPORT AID AND MUST NEVER SUPPORT THE FULL BODY WEIGHT.

- Since suction cups can lose adhesion over time, it is recommended to reattach the support bar to ensure it adheres perfectly to the surface.
- The locking indicators are designed to ensure successful installation. They are not intended to guarantee good adhesion to the surface during subsequent uses. Reinstalling and regularly testing the support bar is the best guarantee of a perfect installation.
- If you leave the grab bar mounted for a long period without reattaching it from time to time, the suction cups may lose their adhesion. In this case, the support bar may be less adherent when reassembled. It will then be necessary to set it aside for 3 or 4 days to allow the suction cups to regain their original shape. You can then reuse the grab bar under optimal efficiency conditions. In any case, check the assembly before each use.

## INSTALLATION

- Place the grab bar at the desired location near the bathtub, in the shower, near the toilet... The suction cups must be placed on a flat, uniform, smooth, and non-porous surface. Placing the suction cups on a crack or tile joint will prevent proper adhesion and make the grab bar unstable.
- Thoroughly clean the mounting surface. Ensure it is dry and free of dust.
- Place the grab bar, ensuring that the suction cups are not placed on a crack or tile joint. Slightly moisten the suction cups for better adhesion.
- Press firmly on the grab bar to expel air from the suction cups (Figure 1) and lock by pressing firmly on the locking button about 3 to 4 times after the red part inside the handle is fully depressed (Figure 3).
- IMPORTANT: Ensure that the RED PART WARNINGS are fully depressed (Figure 4). If RED is still visible, release the lock with the unlock button (Figure 5) and lock again. The LOCKING PADS (near the suction cups) must not be placed

on water droplets or on a surface different from that on which the suction cups rest.

- Retest the adhesion of the grab bar a few minutes after installation.

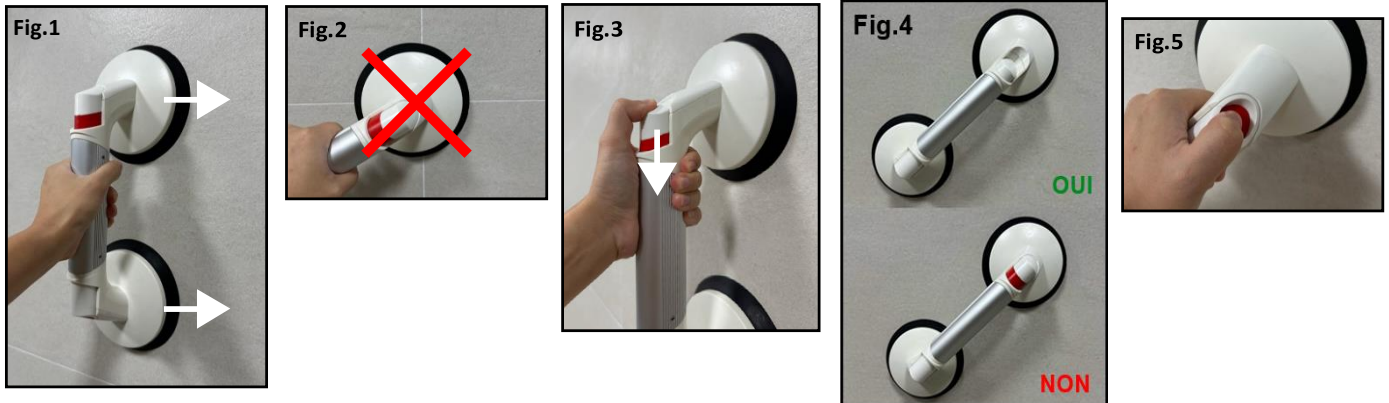
## REMOVAL

Release the lock by pressing the unlock button once (Figure 5).

Slowly slide the grab bar towards the edge of its support surface. You can also slide your finger under the grab bar to slightly lift the suction cup. **DO NOT PULL ON THE GRAB BAR, AS THIS COULD CAUSE PERMANENT DAMAGE**

## NOTE

THE LOCKING INDICATORS ARE ONLY AIDS TO INDICATE THE QUALITY OF THE INSTALLATION. YOU MUST ALWAYS MANUALLY TEST THE GRAB BAR BEFORE EACH USE.

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