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CAUTION

Read the instructions carefully before using this product and keep them if necessary.

User: If you are unable to read or understand any warnings, precautions or instructions, please contact your health care professional or distributor before using this product or risk and physical damage may result.


Consult your doctor or healthcare professional before using a medical device. He will recommend you the most appropriate use.

Distributor: this manual must be given to the users of this product.

PRODUCT DESTINATION

The Pelican shower/commode chair is a medical assistive device designed for children with reduced mobility, including those with multiple disabilities or spasticity. Suitable for both home and medical use, it provides secure support in a seated position. Its rolling and tilting design makes bathing and showering easier.

PRODUCT DESCRIPTION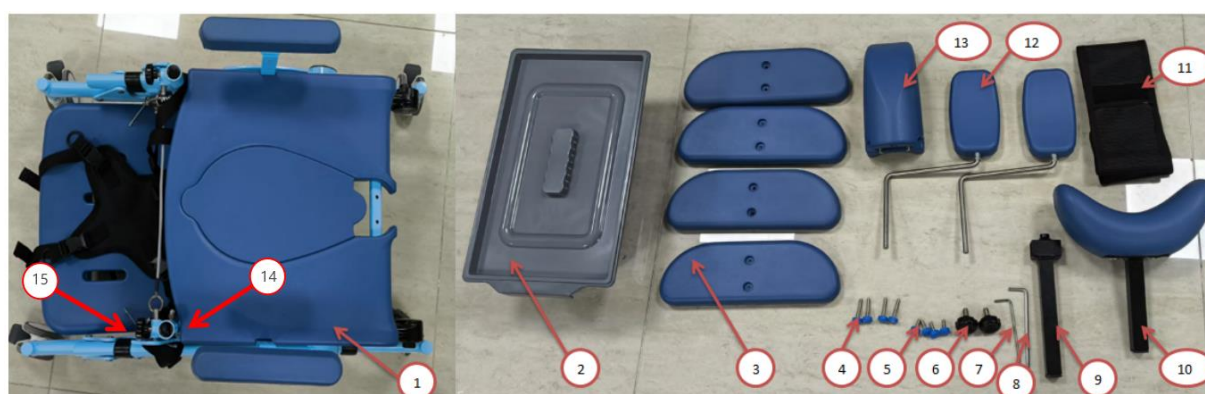

List of Parts :

No.	Parts Name	Quantity	Comments
1	Body (Frame + Backrest)	1	
2	Commode Pan	1	
3	Side Panel	4	
4	Side Panel Screw M5*38	4	For the installation of 2 side panels on each side
5	Side Panel Screw M5*17	4	For the installation of 1 side panel on each side
6	Thumb Screw M8*15	2	To secure the headrest
7	Allen Key 3#	1	To adjust the chest cushion
8	Allen Key 4#	1	To install the chest cushion
9	Headrest Height Adjustment Lever	1	
10	Headrest Forward/Backward Adjustment Lever	1	
11	Calf Straps	1	
12	Chest Cushion	2	
13	Leg Separator	1	
14	Hex Head Cap Screw M6x50	2	For installing backrest
15	Knob Nut M6	2	For installing backrest

SAFETY PRECAUTIONS


Failure to follow the safety precautions may result in the risk of serious injury.

- Do not exceed the maximum weight limit of 100 kg.
- Do not use the shower chair on stairs or escalators.
- Use only accessories and parts provided by the manufacturer.
- Ensure that the adjustment knobs are properly tightened at the same level and that the handle is securely fixed.
- Always sit on the seat when the shower chair is stationary and the brakes are locked. Improper use may cause serious injuries to the user and/or damage to the chair frame and/or wheels.
- Before using the shower chair, make sure:
 - The chair is properly assembled.
 - All components of the chair are securely fastened.
 - The chair has no visible defects.
- Ensure that the shower chair is properly locked before use.
- Make sure all parts (wheels, cylinders, handles, brakes, headrest, guardrails) are securely attached and in good working condition.
- Do not make any adjustments to the product while it is in use.
- All wheels must always remain in contact with the ground.
- Check the proper functioning of the brakes before use.
- When the shower chair is stationary, the brakes must be locked. Do not attempt to move the product while the brakes are locked.
- Ensure that the seat is properly clipped in a flat position before use.
- Do not use the chair without the backrest.
- When seated, do not attempt to reach for objects by leaning forward.
- If any parts are damaged, broken, or worn out, contact the retailer.
- Beware of the risks of pinching and falling associated with the cylinders and folding mechanisms. Handle these elements carefully to avoid accidents.
- Ensure the child's seat is secured to prevent the risk of falling. Use appropriate restraining devices to guarantee their safety (harness, footrest hinge).
- Follow the recommended dimensions and size guidelines for the user to ensure safe and appropriate use.
- Before each use, check the backrest inclination adjustment and ensure that the adjustment pins are properly engaged.

ASSEMBLY

Step 1 : Backrest Installation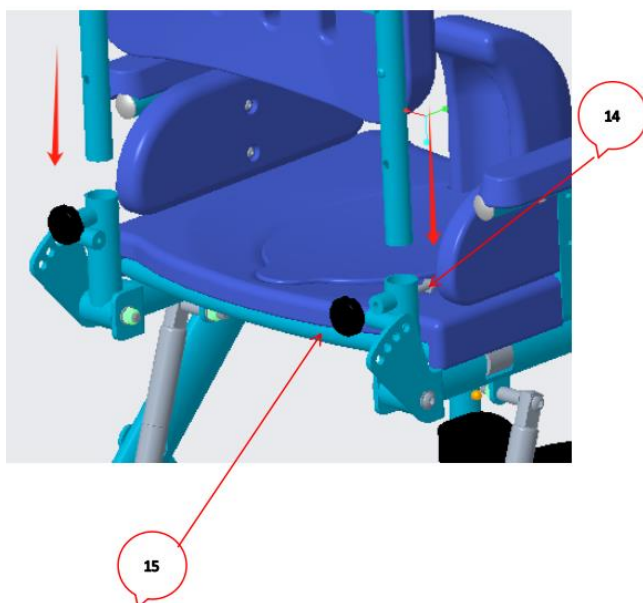


Take the backrest out of the box and install it on the frame, then secure it with the thumb screw in the frame.
Note: Use both hands and insert both sides into the tube simultaneously when installing the backrest.

Step 2 : Unfold the Footrest

Press the joint button on both sides and open the footrest to the desired angle. Use the cord located at the bottom to deploy the pedals.

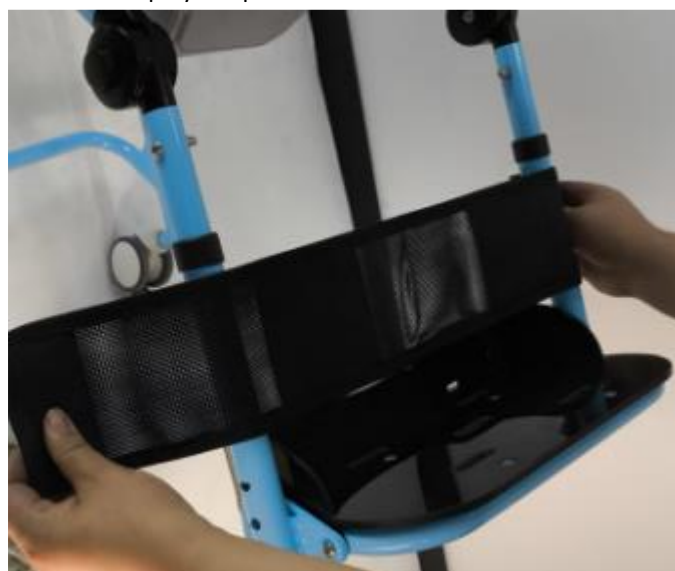Secure the calf strap in the appropriate position and smooth out the Velcro.

Step 4 : Resetting the Seat Frame Angle

Operate the gas spring switch handle with both hands to reset the seat angle. Once each part is fully deployed, the result should match the photo above.

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Step 6: Install the leg separator

Align the holes of the leg separator with those on the seat, then insert it into the seat to complete the installation.

Step 7: Install the commode pan

For size L, insert the commode pan from the back of the seat as shown above.

For size S, insert the commode pan from the back of the seat as shown above.

Step 8 : Chest Cushion Installation

Insert the stainless steel rod of the chest cushion into the body, then tighten the two screws using a 4# Allen key. Install the other side in the same way, ensuring symmetry on both sides.

Step 9 : Side Panel Installation

Seat width range		
Quantity of side panels	Size L	Size S
0	43.5 cm	38.5 cm
1	38.5 cm	33.5 cm
2	33.5 cm	28.5 cm

The user can determine the quantity of side panels based on their needs.

To install one side panel on each side, use the 17 mm short screw.

To install two side panels on each side, use the 38 mm short screw.

Install the side panels onto the armrest tubes. Once the screws are tightened, place the plastic cover to complete the installation.

USE

Step 1 : Seat height adjustment: 565 mm, 585 mm, 610 mm (three available heights).


Step 2 : Leg rest angle adjustment: adjustable to multiple angles.

Adjust the footrest as shown in the photo below.


Press the joint button on both sides simultaneously, then rotate the leg rest to the desired angle. Release the button and wait for a click to complete the leg rest angle adjustment.

Step 3: Height adjustment of leg rest: adjustable on 3 levels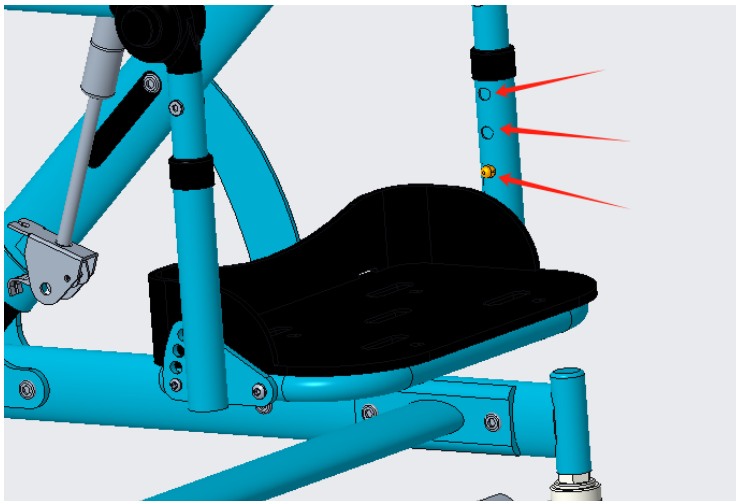

Press the push pins on both sides of the leg tubes simultaneously. Move the leg rest up or down to the desired height. Make sure the push pins are fully ejected before use.

Step 4 : Backrest angle adjustment: adjustable to 4 levels, from 90° to 120°.

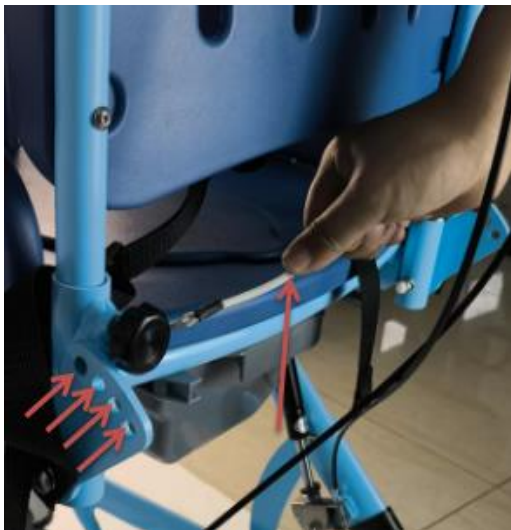Use the cord located behind the backrest to adjust the angle to the desired position. Once the adjustment is complete, you will hear a click. Ensure that the locking pins are fully engaged before use.

Step 5 : Seat tilt adjustment: adjustable from 90° to 135°.

The tilt angle adjustment is performed by simultaneously operating the gas spring handle with both hands. When the seat reaches the desired angle, release the switch handle to finalize the adjustment.

Note: Ensure that the user's safety belt is properly secured before making the tilt adjustment.

Step 6 : Chest Cushion Height Adjustment: adjustable to 4 levels.

Use a 4# Allen key to loosen the screws on the adjustment base, then move the adjustment base up or down to the desired height. Tighten the screws to finalize the height adjustment of the chest cushion. Repeat the same operation on the other side, ensuring symmetry on both sides.

Step 7 : Chest Cushion Width Adjustment: adjustment range of 70 mm on each side.

Use a 4# Allen key to loosen the screws on the adjustment base, then move the adjustment base left or right until the desired width is achieved. Tighten the screws to finalize the width adjustment of the chest cushion. Repeat the same operation on the other side, ensuring symmetry on both sides.

Step 8 : Chest pad depth adjustment

Loosen the 2 screws shown in the photo above using the #3 wrench, adjust the stainless steel rod to the desired position, then tighten the screws to complete the chest pad depth adjustment. Perform the same steps on the other side, ensuring symmetry on both sides.

Step 9 : Chest Cushion Depth Adjustment

Loosen the screws indicated by the arrows in the photo above using a 3# Allen key. Adjust the chest cushion to the desired angle, then tighten the screws to finalize the angle adjustment. Repeat the same operation on the other side, ensuring symmetry on both sides.

Step 10 : Opening and Closing the Chest Cushion

Opening: Pull up the plastic handle of the locking pins to slide the chest cushion outward, making it easier for the user to exit the seat.

Closing: Pull up the plastic handle of the locking pin, then rotate the chest cushion to the desired position. When you hear a click, the closure is complete.

Step 10 : Locking and Unlocking the Casters

When the chair is in use, the casters must be locked.

To lock the caster, press the brake plate with the tip of your foot until the braking is fully engaged.

To move the product, the brake must be released.

To release the brake, lift the brake plate with the tip of your foot until the brake is fully disengaged.

CARE AND MAINTENANCE

Failure to care for and monitor your product may result in the risk of serious injury.

- Regularly clean the product with warm water and a soft cloth. Do not use abrasive products or detergents.
- Dry with a clean, dry soft cloth.
- Ensure that no dirt or debris accumulates around the brakes or wheels.
- Do not use cleaning products containing phenol, chlorine, or any solvent-based products, as they may damage the aluminum and polyamide materials.
- Lubricate the wheels with a little oil if they squeak.
- Regularly check that all screws are properly fixed and tightened.
- Ensure that the brakes function correctly before each use.
- In case of malfunction, do not attempt to repair the chair yourself. Contact an authorized dealer.

Checklist :

To keep the product in good condition, check before each use and regularly as follows:

Check the condition and functionality	Anytime	Weekly
Gas springs	X	
Frame stability	X	
Fixing supports (screws, knobs)	X	
Brakes	X	
Guardrails	X	
Headrest	X	
Backrest, seat, leg separator		X
Adjustment pins	X	
Tilt/reclining handle	X	
Footrest hinge		X
Safety harness	X	

CHANGE OF OWNERSHIP

The product can be reused. It must first be cleaned and disinfected in accordance with hygiene instructions. When transferring the product, remember to hand over all the necessary technical documents to the new user. Beforehand, the product must be inspected by an authorised specialist.

RECYCLING

When the product has become unusable and you have to throw it away: please contact your specialist dealer. If you wish to carry out recycling by yourself, ask your local waste management company for disposal guidelines.

STORAGE

Failure to observe the storage conditions may lead to deterioration of the product and thus to the risk of serious injury.

Do not store your product for an extended period of time near a heat source or in the sun (e.g. behind a window or near a radiator) or near a cold source.

Keep away from all flames and sources of sparks.

Observe the conditions for storing and stowing your products.

- In a dry and temperate place
- Protect your product by packaging it from dust, corrosion (e.g. abrasive elements, sand, sea water, salty air).
- Store all removed parts together in the same place (or locate them if necessary) to avoid mixing them with other products when reassembling.

- All components must be stored without any load (do not place any objects that are too heavy on the product elements, do not jam anything...).

TECHNICAL SPECIFICATIONS

822158S :

Overall dimensions	Length 72 cm x Width 49.5 cm
Backrest dimensions	Height 34.5 cm x Width 35 cm
Seat dimensions	Depth 32 cm x Width 28/33/38 cm x Height 55/57/59 cm
Frame	Aluminium
Weight	15 kg
Maximum weight capacity	100 kg

822158L :

Overall dimensions	Length 77 cm x Width 50 cm x Height 100/107 cm
Backrest dimensions	Height 48 cm x Width 41 cm
Seat dimensions	Depth 36 cm x Width 33/38/42 cm x Height 56/58/60 cm
Frame	Aluminium
Weight	16 kg
Maximum weight capacity	100 kg

PERIOD OF USE

The life expectancy of this product is 5 years under normal use, safety and maintenance conditions. Beyond this period, the product can be used for as long as it is still in good condition.

WARRANTY

This product is guaranteed for two years from the date of purchase by the user. IDENTITIES will provide replacement parts for any part that has broken due to a manufacturing, design or material defect. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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