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IMPORTANT

Read the instructions carefully before using this product and keep them if necessary.

User: If you are unable to read or understand any warnings, precautions or instructions, please contact your health care professional or distributor before using this product or risk and damage may result.

Consult your physician or health care professional before using any assistive device. He/she will advise and inform you on the most appropriate use in your case.

Distributor: this instruction manual must be given to the users of this product.

PRODUCT DESTINATION

The retractable bed rail on stand is a medical device (MD) designed to secure a bed and prevent falls, particularly for elderly individuals who are agitated or disoriented. It is installed between the mattress and the bed base, while being easily retractable to allow easy entry and exit from the bed. This MD is ideal for home or institutional use, providing a practical and secure solution.

PRODUCT DESCRIPTION

A. Side frame

B. Handrail pipe

C. Left/Right lower frame pipe

D. Lower frame pipe

E. Leg tube

F. C-shaped buckle

G. Tool wrench

H. Safety belt

SAFETY PRECAUTIONS

Failure to follow the safety precautions may result in the risk of serious injury.

- Do not exceed the maximum weight limit of 100 Kg.
- Ensure that all parts are securely fastened and in good working condition. Only use accessories and parts provided by the manufacturer. Do not make any adjustments to the bedrail while it is in use.
- In case of damaged, broken, or worn parts, contact the retailer.
- Check the bedrail before each use: Make sure it is not damaged or worn out.
- Proper fixation: The bedrail must be securely attached to the bed frame before use. Use the provided safety belt to secure the entire assembly and position the 2 feet on the ground before use.
- Avoid pinch hazards: Be cautious when retracting the bedrail to prevent pinching fingers. Handle the bedrail carefully and ensure your fingers are not in the moving parts.
- Intended use: This product is designed to secure a bed and should only be used for this purpose. Do not use it for other applications.
- Supervision of children: Children must be supervised when using the bed equipped with this bedrail.
- Regular inspection: Regularly inspect the bedrail to ensure it is functioning properly and there are no signs of failure.
- Avoid incorrect assembly: Make sure to follow the assembly instructions to avoid improper installation, which could compromise safety.
- Do not force: Do not apply excessive pressure to the bedrail when it is in the retracted or deployed position.

ASSEMBLY & USE


OPTION A ASSEMBLY:

1. Remove the side frame A and rotate it in the direction of the arrow.


2. Complete the installation as shown in the figure.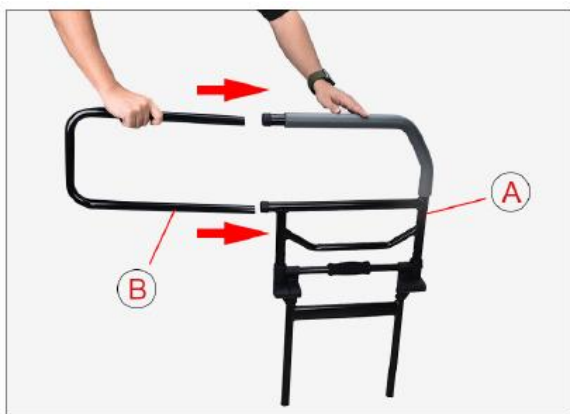

3. Remove the handrail pipe B and insert it following the direction of the arrow.

4. Insert the C-shaped buckle (part F) into the corresponding hole.

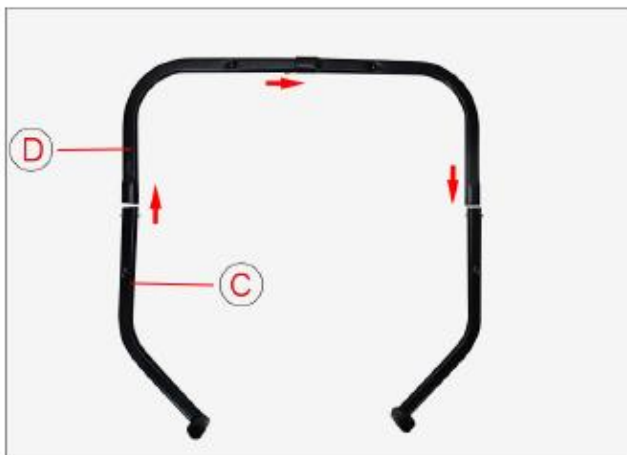5. Remove the lower frame tube C and lower frame tube D, and insert them in the order following the direction of the arrows (make sure the letters match one by one).

6. Press the ball before insertion, then continue until the ball pops out.

7. Remove the assembled side frame A from step 4 and the assembled lower frame tube C from step 6 and insert them following the direction of the arrows.

8. Adjust to the appropriate position and insert the C-type clamp F into the hole.


9. Adjust the leg tube E to the relaxed position.

10. Insert the leg tube E into the side frame A as shown in the illustration.

11. After assembly.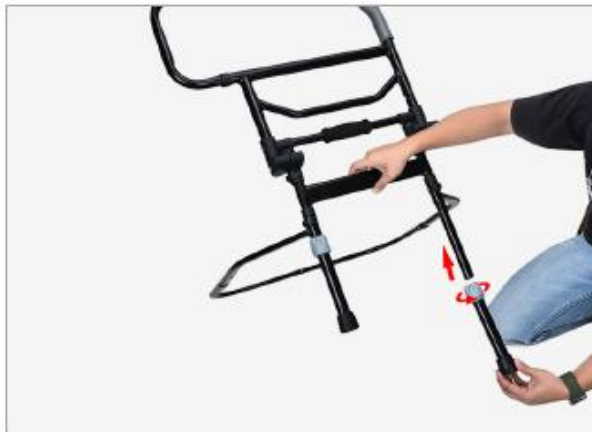

12. The installation of the other leg tube is identical to the previous step.

13. Installation is complete, as shown in the picture.

ASSEMBLY OPTION B: (Using the other side of the bed)

1. Remove the handrail whose direction needs to be changed.

2. Remove the plastic fixing base on both sides in the direction of the arrow.

3. Use a wrench to remove the locked screws in the direction of the arrows on the left and right sides.

4. Then, remove the fixing piece in the direction of the arrow.

5. Follow the instructions in the image above and rotate the left and right sides, as shown in image 1 above before rotating, and as shown below in image 2 once the rotation is complete.

6. Use a wrench to tighten the screws in the direction of the arrows on the left and right sides.

7. Install as shown in the picture.

8. Then, use a slight force to pull upward in the direction of the arrow.


9. Insert the fixed base components that you removed in step 2 into the direction of the arrow.


10. The rotation and installation are completed as shown in the image above.


USE :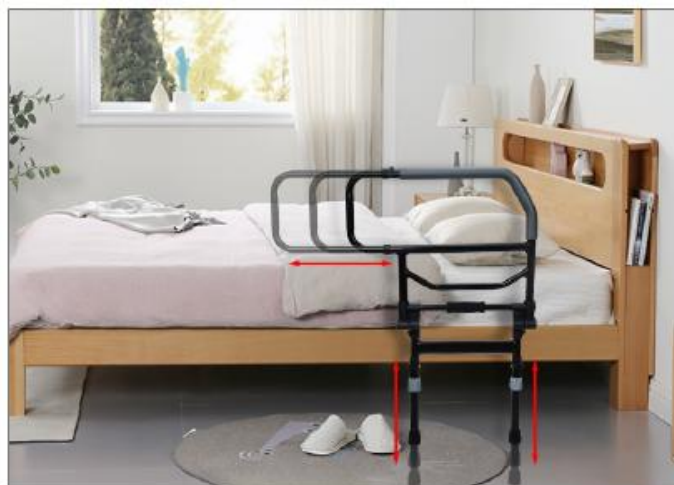

1. Adjustable armrest length, adjustable footrest height.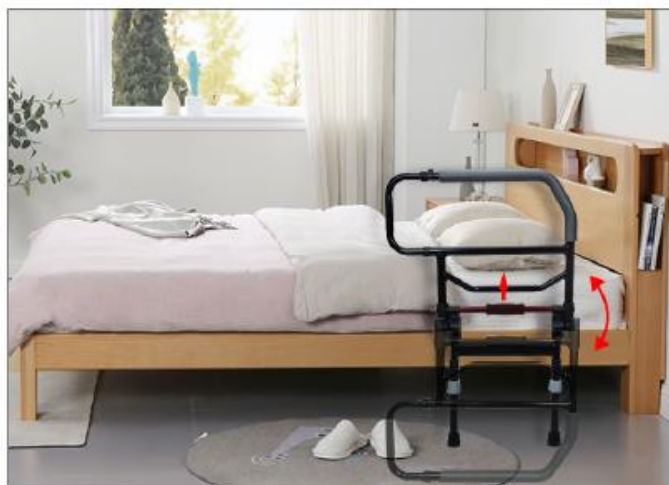

2. The bed armrests can be pressed, flipped, and folded.

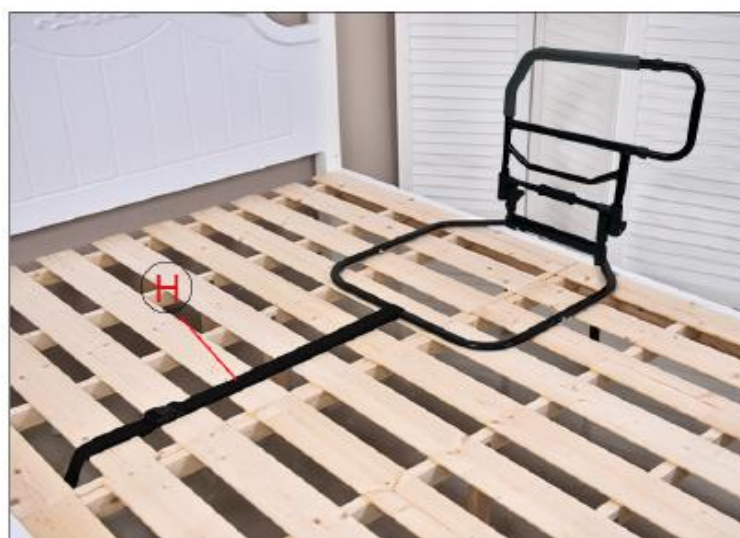3. Installation of the H safety belt as shown.

CARE AND MAINTENANCE

Failure to care for and monitor your product may result in the risk of serious injury.

- Regular Cleaning: Clean the product regularly with warm water and a soft cloth. Avoid abrasive products, detergents, or solvents.
- Drying: Dry the product with a clean, dry cloth.
- Cleaning Products: Do not use cleaning products containing phenol or chlorine, as they may damage the product.
- Inspection: Periodically check the wear and stability of all screws. Immediately replace any broken, damaged, or worn parts.
- Repair: In case of malfunction, do not attempt to repair the product yourself. Consult an authorized dealer.

- **Checklist :**

To keep the product in good condition, check before each use, and check regularly as follows:

Check the item	Anytime	Weekly	Monthly
Height Adjustment Pins	X		
Condition and Functionality of the Safety Belt	X		
Cleanliness			X
Screws and Fastening Elements		X	
Frame Stability	X		

CHANGE OF OWNERSHIP

The product can be reused. It must first be cleaned and disinfected in accordance with hygiene instructions. When transferring your product, remember to hand over all the necessary technical documents to the new user. Beforehand, the product must be inspected by an authorised specialist.

RECYCLING

When the product has become unusable and you have to throw it away: please contact your specialist dealer. If you wish to carry out recycling for yourself, ask your local waste management company for disposal guidelines.

STORAGE

Failure to observe the storage conditions can lead to deterioration of the product and thus to the risk of serious injury.

- Do not store your product for an extended period of time near a heat source or in the sun (e.g. behind a window or near a radiator) or near a cold source.
- Keep away from all flames and sources of sparks.
- Respect the conditions for storing your product: in a dry and temperate place, protect your product from dust, corrosion (e.g. abrasive elements, sand, sea water, salty air).
- Store all removed parts together in the same place (or locate them if necessary) to avoid mixing them with other products when reassembling.
- All components must be stored without any load (do not place any objects that are too heavy on the product elements, do not jam anything...).

PERIOD OF USE

The life expectancy of this product is 5 years under normal use, safety and maintenance conditions for a user. Beyond this period, the product can be used for as long as it is still in good condition.

If the device is used in a multi user environment (such as a nursing home or hospital) then service life may be reduced.

FEATURES TECHNICAL

Height of the Support Relative to the Bottom of the Mattress	60 cm
Length of the Support (3 Adjustments)	54.5/64.5/74.5 cm
Under-Mattress Dimensions	width 59.5 x depth 57 cm
Height Relative to Under-Mattress	21 à 35 cm with continuous adjustment
Weight	3,9 kg
Maximum User Weight	100 kg

GUARANTEE

This product is guaranteed for two years from the date of purchase by the user. IDENTITIES will provide replacement parts for any part that has broken due to a manufacturing, design or manufacturing deficiencies. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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