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## INSTRUCTIONS MANUAL EN

## IMPORTANT

Read the instructions carefully before using this product and keep them if necessary.

User: If you are unable to read or understand any warnings, precautions or instructions, please contact your health care professional or distributor before using this product or risk and damage may result.

Consult your physician or health care professional before using any assistive device. He/she will advise and inform you on the most appropriate use in your case.

Distributor: this instruction manual must be given to the users of this product.

## PRODUCT DESTINATION

The child walker frame is a walking aid for kids with reduced mobility. This walker is designed for indoor use and folds up for easy storage or transport. It can be used outdoors on flat surfaces. It is also available in a version with wheels.

## SAFETY PRECAUTIONS

Failure to follow the safety precautions may result in the risk of serious injury.

- Do not exceed the maximum weight limit of 100 Kg.
- Make sure that all parts are secure and in good working order.
- Do not use the walker on stairs or escalators.
- Only use accessories and parts supplied by the manufacturer.
- The walking frame is a walking aid. It is not a wheelchair. It is not designed to carry a person sitting on it. Such use may cause damage to the feet/wheels and the structure of the walking frame itself
- Do not use the walker to walk backwards.
- Do not make any adjustments to the walker while it is in use.
- All tips must always be in contact with the ground.
- Be sure to put your hands on each handle so that the walker can be in the parallel force position.
- Use caution when climbing pavements or walking on sloping, uneven or slippery surfaces. Failure to do so may result in a significant risk of falling or injury.
- Do not use the walker on a wet, icy, or snowy slope.

- If any parts are damaged, broken or worn, contact the dealer.

## USE

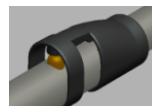
1. Handle
2. Folding mechanism
3. Front cross tube
4. Rear tube
5. Adjustment pin
6. Front leg tube
7. Tips (or 2 front wheels depending on the model)

826293: The product is equipped with 2 front wheels

Carefully remove all components from the carton. Before use, check the product for defects. A healthcare professional has assessed that the product provides the necessary support and that its settings are appropriate for the user.


### UNFOLDING :

To unfold the product, pull the frame outward via the handgrips completely and a “click” to be heard. To ensure the walker is lock into place, check if this button here is fully popped out.

### FOLDING :

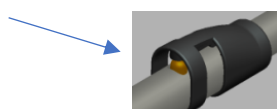Depress push-button on horizontal tube, then pull the frame inner ward direction in order to fold the walker.

### HEIGHT ADJUSTMENT :

Depress push-button completely, then adjust the leg to your desired hole. Repeat this step for all 4 legs. Make sure the button is totally popped out and ALL 4 four buttons are on the same height.

WHEELS shall be always assembled in the front legs of the walker. UNFOLDING, HEIGHT ADJUSTMENT & FOLD is exactly the same on 826293.

## CARE AND MAINTENANCE

Failure to care for and monitor your product may result in the risk of serious injury.

- Clean the walker regularly with warm water and a soft cloth. Do not use abrasive products, detergents or solvents.
- Dry with a clean, dry soft cloth.
- Make sure there is no accumulation of dirt or debris under the tips.
- Do not use cleaning products containing phenol or chlorine, as this could damage the product.
- Periodically check the wear and strength of all screws and bolts. Replace all broken, damaged or worn parts immediately.
- If there is a malfunction, do not repair the rollator yourself. Consult an authorized dealer.
- Checklist :
- To keep the product in good condition, check before each use, and check regularly as follows:

| Check the product                           | Anytime | Weekly |
|---------------------------------------------|---------|--------|
| Check the condition of the adjustment pins  | X       |        |
| Check the condition of the feet and handles |         | X      |
| Check the wheels                            | X       |        |
| Check the folding mechanism                 | X       |        |

## CHANGE OF OWNERSHIP

The product can be reused. It must first be cleaned and disinfected in accordance with hygiene instructions. When transferring your product, remember to hand over all the necessary technical documents to the new user. Beforehand, the product must be inspected by an authorised specialist.

## RECYCLING

When the product has become unusable and you have to throw it away: please contact your specialist dealer. If you wish to carry out recycling for yourself, ask your local waste management company for disposal guidelines.

## STORAGE

Failure to observe the storage conditions can lead to deterioration of the product and thus to the risk of serious injury.

- Do not store your product for an extended period of time near a heat source or in the sun (e.g. behind a window or near a radiator) or near a cold source.
- Keep away from all flames and sources of sparks.
- Respect the conditions for storing your product.
- In a dry and temperate place
- Protect your product from dust, corrosion (e.g. abrasive elements, sand, sea water, salty air).
- Store all removed parts together in the same place (or locate them if necessary) to avoid mixing them with other products when reassembling.
- All components must be stored without any load (do not place any objects that are too heavy on the product elements, do not jam anything...).

## PERIOD OF USE

The life expectancy of this product is 5 years under normal use, safety and maintenance conditions for a user. Beyond this period, the product can be used for as long as it is still in good condition.

If the device is used in a multi user environment (such as a nursing home or hospital) then service life may be reduced.

## FEATURES TECHNICAL

826292 :

|                          |                                              |
|--------------------------|----------------------------------------------|
| Overall dimensions       | Width. 42,5 x Depth. 32 x Height. 61 / 72 cm |
| Width between handles    | 40 cm                                        |
| Weight                   | 2,2 kg                                       |
| Maximum supported weight | 100 kg                                       |

826293 :

|                          |                                              |
|--------------------------|----------------------------------------------|
| Overall dimensions       | Width. 42,5 x Depth. 34 x Height. 61 / 72 cm |
| Width between handles    | 37 cm                                        |
| Weight                   | 2,2 kg                                       |
| Maximum supported weight | 100 kg                                       |

## GUARANTEE

This product is guaranteed for two years from the date of purchase by the user. IDENTITIES will provide replacement parts for any part that has broken due to a manufacturing, design or manufacturing deficiencies. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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