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INSTRUCTIONS MANUAL EN

IMPORTANT

Read the instructions carefully before using this product and keep them if necessary.

User: If you are unable to read or understand any warnings, precautions or instructions, please contact your health care professional or distributor before using this product or risk and damage may result.

Consult your physician or health care professional before using any assistive device. He/she will advise and inform you on the most appropriate use in your case.

Distributor: this instruction manual must be given to the users of this product.

PRODUCT DESTINATION

The Neo Dandy indoor rollator is a walking aid for people with reduced mobility. This rollator is designed for indoor use and folds up for easy storage or transport. It can be used outdoors on flat surfaces.

SAFETY PRECAUTIONS

Failure to follow the safety precautions may result in the risk of serious injury.

- Do not exceed the maximum weight limit of 100 kg.
- Do not exceed the maximum load limit in the basket (5 kg). Loads should only be carried in the basket. Do not hang heavy objects on the rollator as this could affect its stability.
- Ensure that all parts (wheels, cables, and handles) are securely attached and in good working condition.
- Do not use the rollator on stairs or escalators.
- Only use accessories and parts provided by the manufacturer.
- Before each use, check that the indoor rollator is fully unfolded and that the handles are at the same height.
- The rollator is a walking aid. It is not a wheelchair. It is not designed to carry a person sitting on it. Such use may damage the wheels and the structure of the rollator, compromising its integrity and mechanical features for future use. Only use the rollator as a walking support.
- Do not use the rollator to walk backward. Do not use the rollator to sit down.
- Do not make any adjustments to the rollator while it is in use.
- Check that the brakes are functioning properly before use. All wheels must always be in contact with the ground.
- When the rollator is stationary, the brakes must be locked. Do not attempt to move the rollator if the brakes are locked.
- Make sure to place your hands on each handle so that the rollator can be in a parallel strength position.
- Be careful when going up curbs or when using inclined, uneven, or slippery surfaces. Failing to follow this guideline can pose a significant risk of falling or injury.
- Do not use the rollator on a wet, icy, or snowy slope.
- In case of damaged, broken, or worn parts, contact the retailer.

PRODUCT DESCRIPTION

No.	Part description	No.	Part description
1	Handle	7	Brake cable
2	Brake handle	8	Front fork
3	Height adjustment part	9	Front wheel
4	Tray	10	Rear wheel
5	Basket with cloth bag	11	Brake set
6	Folding mechanism button	12	Knob

MOUNTING & USE

1. Remove the item from the box and check the contents and packaging for any damage or to ensure it is complete. The indoor rollator is delivered in a kit and must be assembled.

Main frame*1	Tray*1
Handrail tube*1	User manual*1
Cloth bag*1	Knobs*4

2. Insert the front legs into the front frame on both sides until you hear a click. Also, ensure that the button fits into the hole, then secure everything with the knobs. (See images)

Repeat this operation at the back by inserting the rear legs into the back frame on both sides, then secure with the knobs.

3. Place the indoor rollator in an upright position and attach the basket to unfold it.

4. Open the gray height adjustment elements on each side of the frame. (See image below)

Pull the
basket back
to secure it.

5. Attach the handles to the frame and close the gray height adjustment elements so that both sides are at the same height.

6. Place the tray on the frame.

Height Adjustment

- To adjust the height, open the gray height adjustment elements and adjust the handles to the appropriate height.
- Close the gray adjustment elements on each side and ensure they are securely locked and set to the same height.

Brake Function

- Pull the brake handle upward to brake the indoor rollator.
- Press the brake handle downward for parking.

Folding and Storage

- Remove the tray and press the yellow folding mechanism button to fold and store the indoor rollator.

CARE AND MAINTENANCE

Failure to care for and monitor your product may result in the risk of serious injury.

- Clean the rollator regularly with warm water and a soft cloth. Do not use abrasive products, detergents or solvents.
- Dry with a clean, dry soft cloth.
- Make sure there is no accumulation of dirt or debris around the axle, brakes or wheels.
- Do not use cleaning products containing phenol or chlorine, as this could damage aluminium and polyamide materials.
- Lubricate the wheels with a little oil if they squeak.
- Periodically check the wear and strength of all screws and bolts. Replace all broken, damaged or worn parts immediately.
- Check that the brakes are working before each use.
- Check the parking brakes by pushing the handles into the locked position, then trying to push the rollator.
- If there is a malfunction, do not repair the rollator yourself. Consult an authorized dealer.
- Checklist :**

To keep the rollator in good condition, check before each use, and check regularly as follows :

Check the item	Anytime	Weekly	Monthly
Clip and knob for height adjustment		X	
Structural weld at wheels		X	
Cleanliness	X		
Screws and fasteners		X	
Stability of frame and backrest			X
Brakes	X		

CHANGE OF OWNERSHIP

The product can be reused. It must first be cleaned and disinfected in accordance with hygiene instructions. When transferring your product, remember to hand over all the necessary technical documents to the new user. Beforehand, the product must be inspected by an authorised specialist.

RECYCLING

When the product has become unusable and you have to throw it away: please contact your specialist dealer. If you wish to carry out recycling for yourself, ask your local waste management company for disposal guidelines.

STORAGE

Failure to observe the storage conditions can lead to deterioration of the product and thus to the risk of serious injury.

- Do not store your product for an extended period of time near a heat source or in the sun (e.g. behind a window or near a radiator) or near a cold source.
- Keep away from all flames and sources of sparks.
- Respect the conditions for storing your product :

- In a dry and temperate place
- Protect your product from dust, corrosion (e.g. abrasive elements, sand, sea water, salty air).
- Store all removed parts together in the same place (or locate them if necessary) to avoid mixing them with other products when reassembling.
- All components must be stored without any load (do not place any objects that are too heavy on the product elements, do not jam anything...).

PERIOD OF USE

The life expectancy of this product is 5 years under normal use, safety and maintenance conditions for a user. Beyond this period, the product can be used for as long as it is still in good condition.

If the device is used in a multi user environment (such as a nursing home or hospital) then service life may be reduced.

FEATURES TECHNICAL

Total Depth (open)	65,5 cm
Total Width (open)	60 cm
Minimum Total Height	89 cm
Maximum Total Height	93,5 cm
Maximum Load of the Tray	5 kg
Maximum Load of the Bag	5 kg
Maximum Weight Supported	100 kg

GUARANTEE

This product is guaranteed for two years from the date of purchase by the user. IDENTITES will provide replacement parts for any part that has broken due to a manufacturing, design or manufacturing deficiencies. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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