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INSTRUCTIONS MANUAL EN

IMPORTANT

Read the instructions carefully before using this product and keep them if necessary.

User: If you are unable to read or understand any warnings, precautions or instructions, please contact your health care professional or distributor before using this product or risk and damage may result.

Consult your physician or health care professional before using any assistive device. He/she will advise and inform you on the most appropriate use in your case.

Distributor: this instruction manual must be given to the users of this product.

PRODUCT DESTINATION

The lightweight NEO EASY rollator is a walking aid device designed for people with reduced mobility. This product can be used both indoors and outdoors. It is equipped with a seat to allow for sitting and taking breaks.

SAFETY PRECAUTIONS

Failure to follow the safety precautions may result in the risk of serious injury.

1. Do not exceed the maximum weight limit of 136kg.
2. Ensure that all parts (wheels, cables, and handles) are securely fastened and in good working condition.
3. Do not use the rollator on stairs or escalators.
4. Check that the handle installation is not loose or chipped, to avoid compromising the safety of use.
5. Use only the accessories and parts provided by the manufacturer.
6. The rollator is a walking aid. It is not a wheelchair. It is not designed to carry a person sitting on it. Such use may damage the wheels and the rollator's structure. This could weaken it and compromise its mechanical characteristics during future use. Only use the rollator as a walking support.
7. Do not use the rollator to walk backward.
8. Do not make any adjustments to the rollator while it is in use.
9. Check the proper functioning of the brakes before use.
10. All wheels must always be in contact with the ground.
11. Make sure to place your hands on each handle so that the rollator can be in a parallel force position.
12. Be careful when going up curbs or when using inclined, uneven, or slippery surfaces. Failure to follow this instruction may result in a significant risk of falling or injury.
13. Do not use the rollator on wet, icy, or snowy slopes.
14. In case of damaged, broken, or worn parts, contact the retailer.

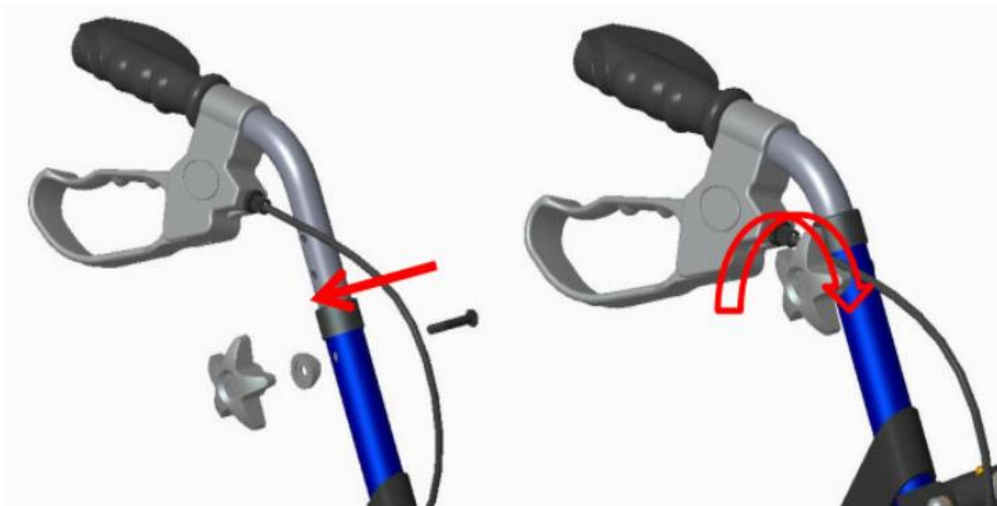
DESCRIPTION OF THE PRODUCT

1. Brake system
2. Frame
3. Folding system
4. Brake
5. Wheels
6. Handle
7. Backrest
8. Seat
9. Bag

MOUNTING & USE

1. The handle must be installed before using the rollator: unscrew the knob of the fixed handle, insert the handle into the inner tube of the bracket from left to right, and adjust it to the appropriate height for the user. Then use the star-shaped knob to lock it. The frame must be fully opened until all the wheels are stable and can be used.

2. Temporary braking during use: Lift the brake lever firmly to brake. For long-term parking: Hold the brake lever with your hand and press it down firmly. When you hear a "click," the parking brake is engaged.

3. Fold up the seat board, then remove the shopping basket and lift the round steel under the basket to fold it.

CARE AND MAINTENANCE

Failure to care for and monitor your product may result in the risk of serious injury.

- Clean the walker regularly with lukewarm water and a soft cloth. Do not use abrasive products, detergents or solvent-based products.
- Dry with a soft, clean, dry cloth.
- Make sure there is no build-up of dirt or debris around the axle, brakes or wheels.
- Do not use cleaning products containing phenol or chlorine, as this could damage aluminium and polyamide materials.
- Lubricate the wheels with a little oil if they squeak.
- Periodically check all screws and bolts for wear and strength. Replace all broken, damaged or worn parts immediately.
- In the event of a malfunction, do not repair the rollator yourself. Consult an authorised dealer.

Checklist :

To keep your walker in good condition, check before each use and regularly as follows.

Check the item	Anytime	Weekly	Monthly
Condition and Operation of the Brakes	X		
Condition and Operation of the Wheels		X	
Overall Condition of the Product		X	
Cleanliness	X		
Screws and Fastening Elements		X	
Stability of the Frame			X

CHANGE OF OWNERSHIP

The product can be reused. It must first be cleaned and disinfected in accordance with hygiene instructions. When transferring your product, remember to hand over all the necessary technical documents to the new user. Beforehand, the product must be inspected by an authorised specialist.

RECYCLING

When the product has become unusable, and you have to throw it away: please contact your specialist dealer. If you wish to carry out recycling for yourself, ask your local waste management company for disposal guidelines.

STORAGE

Failure to observe the storage conditions can lead to deterioration of the product and thus to the risk of serious injury.

- Do not store your product for an extended period of time near a heat source or in the sun (e.g. behind a window or near a radiator) or near a cold source.
- Keep away from all flames and sources of sparks.
- Respect the conditions for storing your product :
- In a dry and temperate place
- Protect your product from dust, corrosion (e.g. abrasive elements, sand, sea water, salty air).
- Store all removed parts together in the same place (or locate them if necessary) to avoid mixing them with other products when reassembling.
- All components must be stored without any load (do not place any objects that are too heavy on the product elements, do not jam anything...).

PERIOD OF USE

The life expectancy of this product is 5 years under normal use, safety and maintenance conditions for a user. Beyond this period, the product can be used for as long as it is still in good condition.

If the device is used in a multi user environment (such as a nursing home or hospital) then service life may be reduced.

FEATURES TECHNICAL

Overall Dimensions	62 cm (width) x 75 cm (depth) x 77/92.5 cm (height)
Seat Dimensions	45 cm (width) x 25.5 cm (depth) x 57 cm (height)
Width Between Armrests	49 cm
Folded Width	21 cm
Maximum Weight Capacity	136 kg
Weight	5,6 kg

GUARANTEE

This product is guaranteed for two years from the date of purchase by the user. IDENTITIES will provide replacement parts for any part that has broken due to a manufacturing, design or manufacturing deficiencies. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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