BANDEAU CRYO GELPRO

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EN

IMPORTANT

Read this leaflet carefully before using this product and keep it for future reference.

User: If you are unable to read or understand any of the warnings, precautions or instructions, please contact your healthcare professional or distributor before using this product, as this may cause hazards and damage.

Consult a healthcare professional before using any medical device. He or she will advise and inform you on the most appropriate use of this product.

Distributor: This leaflet should be given to users of this product.

PRODUCT DESTINATION

The Cryo GelPro headband is a medical device designed to relieve headache-related pain. It is applied directly to the forehead and eyes. This product is intended for the general public.

SAFETY PRECAUTIONS

Failure to follow the safety precautions may result in the risk of serious injury.

- Each time you use it, check the condition of the compress, especially if the envelope is not pierced.
- Do not cut the product.
- People who are allergic to the external materials of this product (Chinlon, Spandex) should not use it on damaged skin.
- Do not use on sensitive areas.
- This product is for external use only. Do not ingest.
- Children must use under adult supervision.
- Avoid use in individuals with end-stage circulatory dysfunction or those who are not autonomous.
- Consult a physician before use if you are diabetic, have poor circulation, heart disease, rheumatoid arthritis, or if you are pregnant.
- To avoid frostbite, which may occur if the pack is too cold for your skin, set it aside until it reaches a tolerable temperature. If the pack is not cold enough, place it back in the freezer for an additional 20 minutes.
- Apply cold to the affected area for a maximum of 20 minutes, with 20-minute breaks between applications.
- Do not wear while sleeping. Check the skin frequently during use. If irritation or burning occurs, remove the product immediately.
- DO NOT use if the contents are leaking and/or the pouch is damaged or torn. If the contents come into contact with your eyes, remove the pouch, rinse the affected area with water, and seek medical attention immediately.

- DO NOT apply with analgesic creams or lotions, medicated ointments, or other topical creams.
- DO NOT use on diseased, damaged, or broken skin.
- DO NOT use on individuals unable to remove the product on their own, such as children, infants, some elderly people, or individuals unable to follow all usage instructions.
- DO NOT apply additional pressure to the product.
- DO NOT heat.
- DO NOT swallow the product or its contents. Keep out of reach of children and pets.
- Discontinue use and consult a doctor if pain worsens or persists for more than 7 days, or if you experience any discomfort, burning, swelling, rash, or other skin changes.

USE

Cooling Instructions

1. Store the pack in the freezer with storage bag so that it is always ready for cold application. Refrigerate the pack for at least 2 hours before use.
2. Check to make sure the temperature of the pack is suitable before application.
3. If it is too cold, better wrap it in towel or any soft cloth before applying on skin.

Treatment

1. Check if the compress is damaged. If it is damaged, do not use this product.
2. Place the product on the affected area for a maximum of 20 minutes, with 20-minute breaks between applications.

To ensure a long service life, check the condition of the product, including the presence of holes, before each use.

CARE

Failure to care for your product may result in the risk of serious injury.

- Cleaning: After use, wipe with a damp cloth and allow the packs to air dry completely before reusing. The pack is not machine washable.
- Do not use alcohol or other solvents to clean the compress.
- In case of wear or tear, do not attempt to repair the product: discard it or contact an authorized dealer to check if it can be handled by IDENTITES' customer service.

CHANGE OF PATIENT

The product can be reused. It must first be cleaned and disinfected in accordance with the hygiene and decontamination instructions. When changing the bed, remember to give the instructions to the new user. Beforehand, the product should be inspected to check its good condition.

RECYCLING

When the product has become unusable and you have to throw it away: please contact your specialist dealer. If you wish to carry out recycling by yourself, ask your local waste management company for disposal guidelines.

STORAGE

Failure to observe the storage conditions may result in deterioration of the product and therefore risk of serious injury
Keep away from flames and sparks.

Seal the packaging in a bag and store it at room temperature.

Observe the conditions for storing your products:

- In a dry and temperate place
- Protect your product with a dust cover
- Do not place heavy loads on it.

FEATURES TECHNICAL

The product consists of an outer packaging and an inner gel. The outer shell is made of 82% Chinlon and 18% Spandex. The gel consists of Crosslinked acrylic resin: 15%-25%; Polyhydric alcohol: 40%-60%; Electrolytic salt: 2%-10%; Additives (stabilizer, etc.): <0.5%; De-ionized water: 20%-40%.

Overall dimensions	835202 – Width 23,5 x Height 15,5 cm
Weight	835202 – 0,470 kg

PERIOD OF USE

The expected lifetime of this product is 5 years under normal use and maintenance. After this period, the product can be used for as long as it is still in good condition.

WARRANTY

This product is guaranteed for 1 year from the date of purchase by the user. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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