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IMPORTANT

Read the instructions carefully before using this product and keep them if necessary.


User: If you are unable to read or understand any warnings, precautions or instructions, please contact your health care professional or distributor before using this product or risk and damage may result.

Consult your physician or health care professional before using any assistive device. He/she will advise and inform you on the most appropriate use in your case. Distributor: this instruction manual must be given to the users of this product.

PRODUCT PURPOSE

The Rest N Roll shopping cart is a walking aid designed for people with reduced mobility. This product is intended for outdoor use. It is a foldable cart for carrying groceries, equipped with a seat and parking brakes.

PRODUCT DESCRIPTION

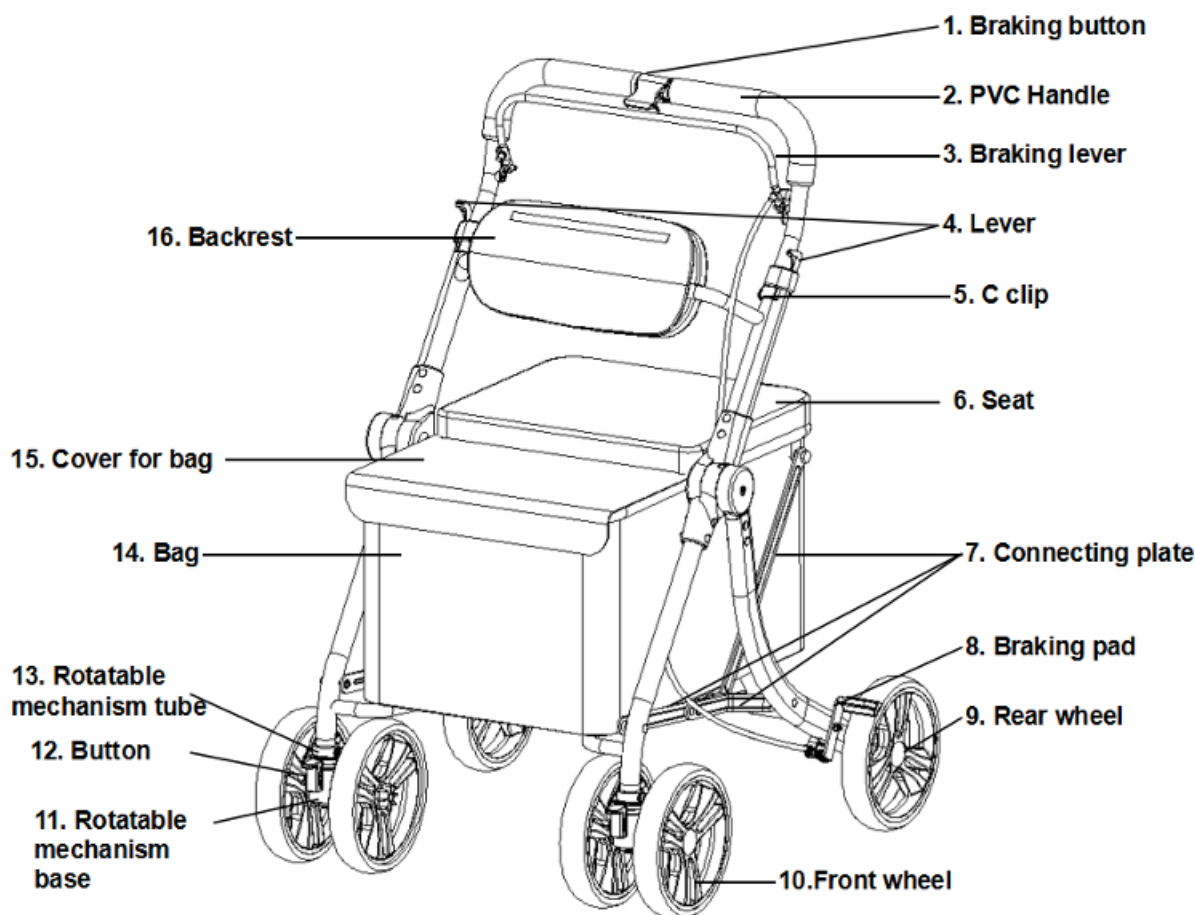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Failure to follow the safety precautions may result in serious injury.

- Do not attempt to assemble or use this shopping cart until you have read and fully understood the entire manual. If you have any doubts or questions about warnings or instructions, consult your retailer or a qualified technician before assembly. Failure to do so may lead to dangerous situations and potential injury.
- Users must perform regular inspections to ensure that all components are securely connected and locked into their designated positions.
- Retailers or technicians should assist with height adjustments, bolt tightening, optimal positioning of supports, and checking the flexibility of the braking system.
- All bolts must be properly tightened to securely fix connection parts and prevent structural play under load. The braking system must completely immobilize the wheels when the handle is engaged. Ignoring these warnings may lead to injury.
- Do not exceed the maximum user weight limit of 110 kg.

- Do not exceed the maximum basket load limit of 5 kg. Loads should be carried only in the basket. Do not hang heavy items on the rollator, as this may compromise its stability.
- Ensure that all parts (wheels, cables, and handles) are securely attached and in good working condition.
- Make sure that the adjustment knobs are tightened and set to the same height.
- When using this cart or any other mobility aid, always ensure that the handle and height settings are in proper condition, the wheels and moving parts function correctly, and the braking system is operating effectively.
- All wheels must remain in contact with the ground during use to ensure stability. The parking brake must be fully engaged and locked before using the seat function.
- After each height adjustment, verify that the handle is securely fixed and remains stable under pressure. During use, ensure all frame parts are tightly secured and that moving parts operate freely without obstruction.
- Do **not** use the product on stairs or escalators.
- Use only accessories and parts provided by the manufacturer.
- This product is a **walking aid**, not a **wheelchair**. It is **not designed to carry a person while seated**. Doing so may damage the wheels and the frame, weakening the product and compromising its future performance. Use this product **exclusively** as a walking support.
- Do not walk backwards while using the rollator.
- Do not make any adjustments while the rollator is in use. Always check the brakes before use.
- All wheels must remain in contact with the ground at all times.
- When stationary, the brakes must be engaged. Do not attempt to move the rollator with the brakes locked.
- Beware of pinching hazards during folding.
- Place your hands on each handle to ensure the rollator is in a parallel, balanced position.
- Use caution when stepping onto curbs or navigating sloped, uneven, or slippery surfaces. Failure to do so may result in serious falls or injuries. Do not use the rollator on wet, icy, or snowy slopes.
- If any parts are damaged, broken, or worn out, contact your retailer.
- The product must always be placed in a stable position, avoiding sudden shocks to prevent structural damage.

ASSEMBLY AND USE

- Ensure that the C-clip is fully inserted into the handlebar socket to prevent any looseness or detachment during use.
- Frequently inspect the rollator to ensure all connections and components are properly secured and locked into their designated positions. This check should be performed before each use.

ASSEMBLY :

1. Installing the front wheel: Push the base of the swivel mechanism upward. When you hear a click, this indicates that the base and the tube of the swivel mechanism are fully aligned, confirming that the front wheel is correctly installed (as shown in Figure 10).
2. Removing the front wheel: Use your thumb and index finger to press the notch located beneath the swivel mechanism tube, compressing it inward until it becomes narrower than the inner diameter of the swivel base tube. With your other hand, pull the base of the swivel mechanism downward. Once the base is completely detached from the swivel tube, the front wheel is removed (as shown in Figure 11).

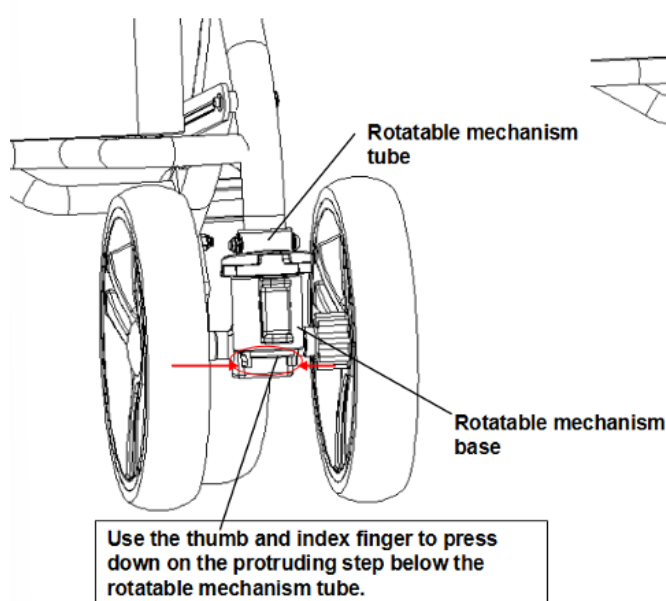
Figure 10

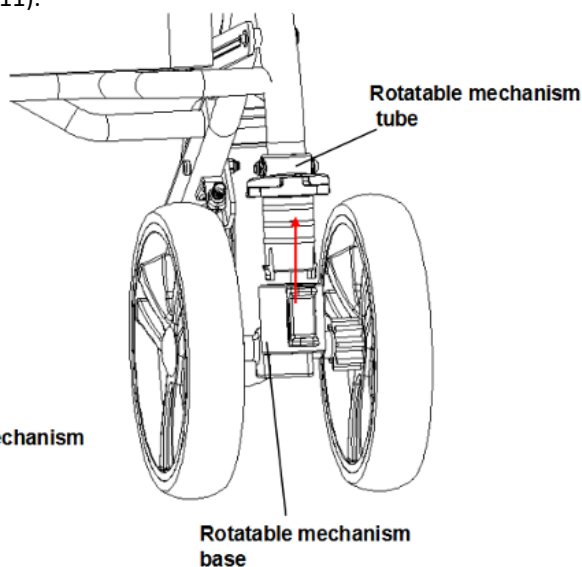
Figure 11

Installing the handle tubes :

Remove the C-clips, then insert the handle tubes into the outer handle tubes (as shown in Figure 13). Secure the handle tubes using the C-clips on both sides (see Figure 14).

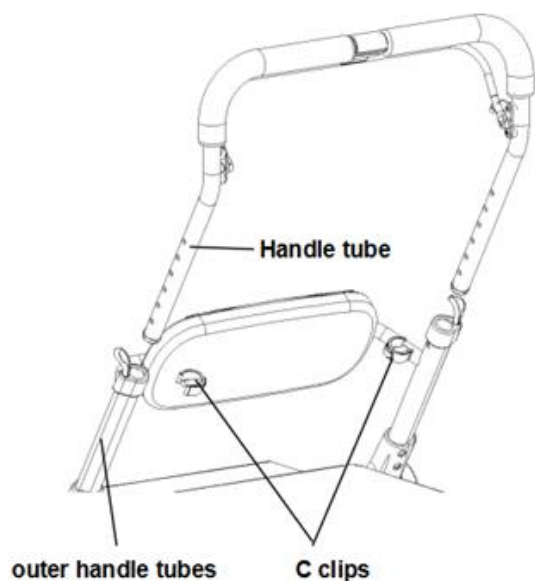
Figure 13

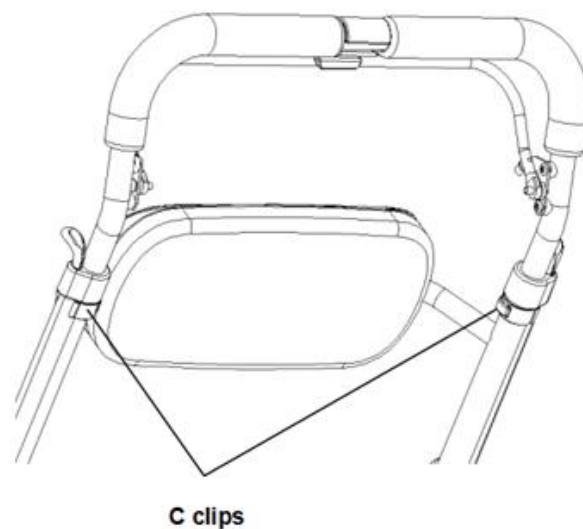
Figure 14

USE :

Unfolding :

- With one hand, hold the seat plate and push it downward, while the other hand pulls the adjustable handle tube outward, as shown in Figure 1, until the cart is fully opened. When you hear a "click", it indicates that the cart is properly unfolded.
- At this point, the two folding plates underneath should be aligned (see Figure 2), confirming that the cart is fully opened. *(For clarity, the image does not show the shopping bag.)*
- Note: Before use, test the cart by holding the side tube and applying pressure to ensure that it does not move and remains securely in the open position.

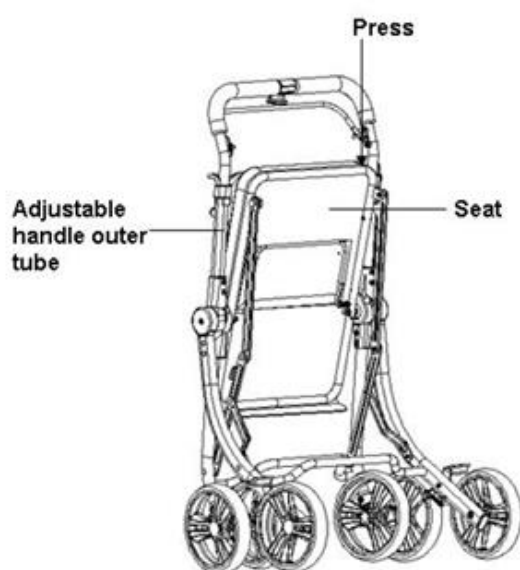
Figure 1

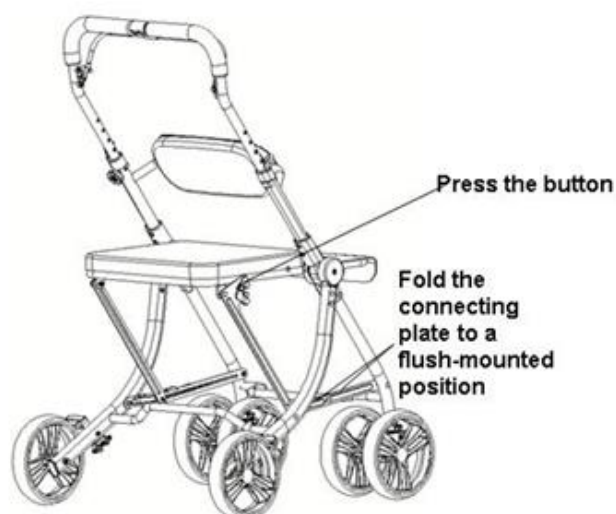
Figure 2

Folding :

While folding, hold the handle with one hand and press the release latch inward with the other hand. Then firmly pull the seat upward to complete the folding process, as shown in Figure 2.

Adjusting the handle tube :

Note: Refer to the safety instructions in the safety summary of this manual before proceeding.

Start by removing the C-clip, then adjust the adjustable handle tube up or down to the desired height (as shown in Figure 3). Secure it in place with the C-clip.

Braking :

1. Engaging the Brake: Hold the handle and brake lever with both hands, then press the brake lever toward the handle, as illustrated in Figure 3. Next, lock the brake lever using the locking button (see Figure 3). Make sure the brake is engaged when you hear a “click.”
2. Releasing the Brake: Release the brake by unfastening the locking loop in the opposite direction until you hear a “click.” The brake is then disengaged.

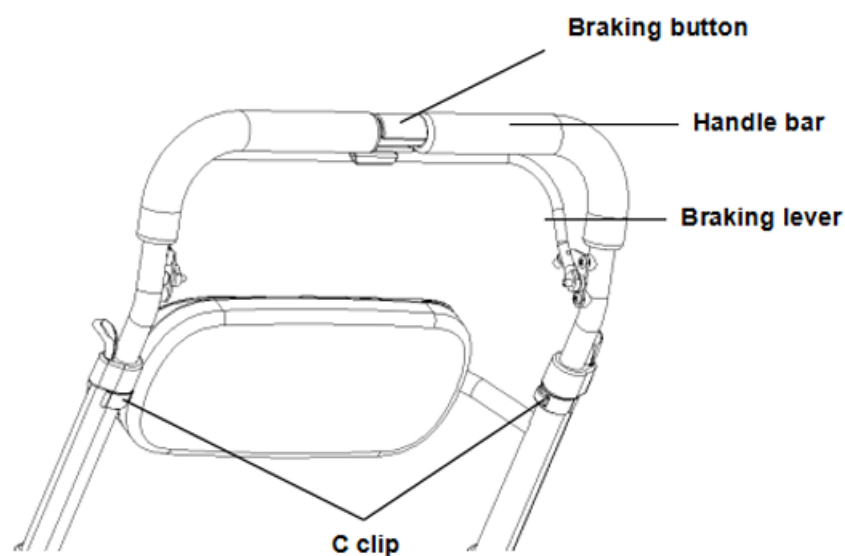
Figure 3

Swivel front wheel mechanism :

1. When the directional lock button is lifted upward and secured into the fixed tube, the front wheels are locked, as shown in Figure 4.
2. When the directional lock button is pushed downward to disengage from the fixed tube, distinct tactile feedback is felt during the downward motion. The front wheels can then rotate 60° to the left and 60° to the right, as shown in Figures 5 and 6.
3. When the directional lock button is pushed even further down and second tactile feedback is felt, the front wheels are unlocked and can rotate 360°, as illustrated in Figure 5.

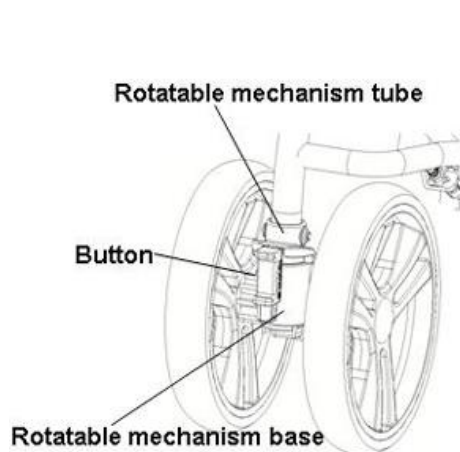
Figure 4

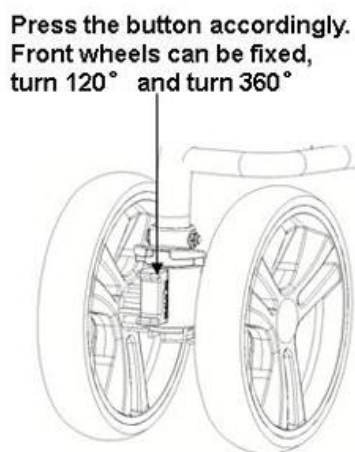
Figure 5

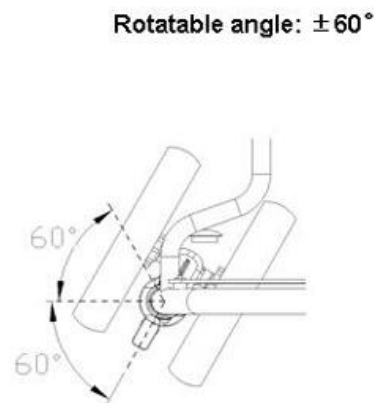
Figure 6

Rest mode and rolling mode :

By pulling the levers on each side with both hands and then pushing the handle forward (toward the bag side), the cart switches to rest mode and can brake automatically when seated, as shown in Figures 7 and 8.

Similarly, by pulling the levers on each side with both hands and then pushing the handle backward (toward the seat side), the cart switches to rolling mode.

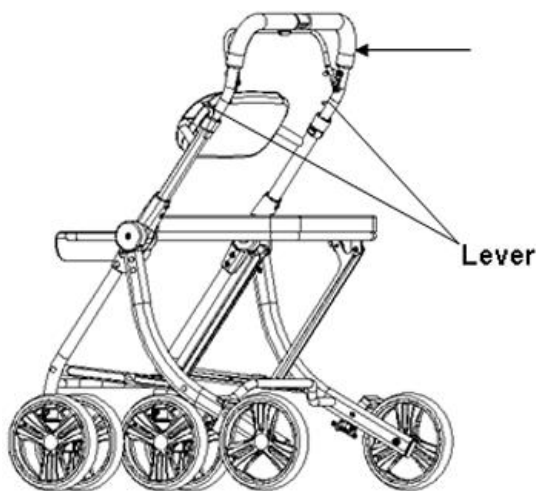
Figure 7

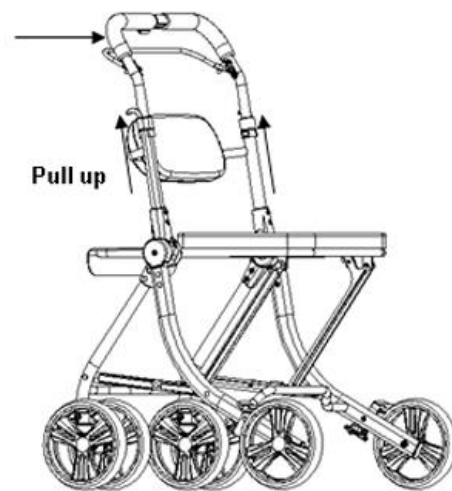
Figure 8

Upright position after folding :

As shown in Figure 9, there is an L-shaped plate. By pressing the plate button to fold, you will hear a "click" indicating that the folding is complete, and the cart can then stand upright on its own.

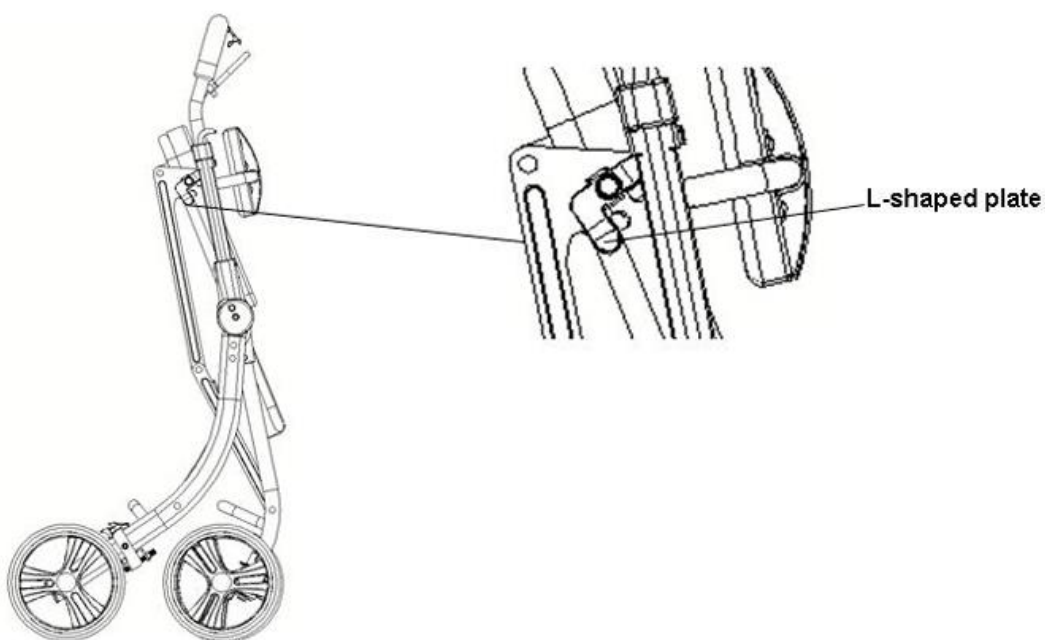
Figure 9

CARE - MAINTENANCE

! Failure to properly maintain your product may result in serious injury.

- Clean the rollator regularly using warm water and a soft cloth. Do not use abrasive products, detergents, or solvent-based cleaners.
- Dry with a clean, soft, and dry cloth.
- For stains on the fabric bag, clean with water and a mild detergent.
- Check the screws to ensure the folding mechanism is locked. If any screws are loose, tighten them.
- Before each use, verify that the C-lock clip on the handle is fully engaged in the adjustment hole.
- Replace any damaged components immediately.
- Ensure there is no buildup of dirt or debris around the brakes or wheels.
- Do not use cleaning products that contain phenol or chlorine, as these may damage aluminum and polyamide parts.
- Lubricate the wheels with a small amount of oil if they begin to squeak.
- Regularly inspect all screws and bolts for wear and tightness. Replace any broken, damaged, or worn parts immediately.
- Check that the brakes are functioning properly before each use.
- Test the parking brakes by pushing the handles into the locked position, then try to push the rollator.
- If the rollator malfunctions, do not attempt to repair it yourself. Contact an authorized retailer for assistance.

- **Checklist :**

To keep the rollator in good condition, check before each use and perform regular inspections as follows :

Check the following items	Anytime	Weekly	Monthly
Height adjustment clips		X	
Frame welds near the wheels		X	
Cleanliness	X		
Screws and fasteners		X	
Frame and backrest stability			X
Condition of the seat		X	
Brakes	X		

CHANGE OF OWNERSHIP

The product may be reused. It must first be cleaned and disinfected in accordance with hygiene guidelines. When transferring the product, make sure to provide all technical documentation to the new user. Beforehand, the product should be inspected by an authorized specialist.

RECYCLING

When the product is no longer usable and needs to be disposed of, please contact your specialized retailer. If you wish to recycle it yourself, consult your local waste management authority for proper disposal instructions.

STORAGE

! Failure to follow storage conditions may lead to product deterioration and potential serious injury.

- Do not store the product for long periods near a heat source or in direct sunlight (e.g., behind a window or near a radiator), or near a source of cold.
- Keep away from open flames and spark sources.
- Follow the recommended conditions for storing your product:
 - In a dry, temperature-controlled environment
 - Protected from dust, corrosion (e.g., abrasive elements, sand, seawater, salty air)
- Store all removed parts together in the same location (or label them if necessary) to avoid mixing with other products during reassembly.
- All components must be stored without any load placed on them (do not place heavy objects on parts or wedge them under pressure).

EXPECTED LIFESPAN

The expected lifespan of this product is **5 years** under normal conditions of use, safety, and maintenance for a single user. After this period, the product may continue to be used as long as it remains in good condition. If the product is used in multi-user environments (such as nursing homes or hospitals), its lifespan may be reduced.

TECHNICAL SPECIFICATIONS

Overall dimensions	Width 57.5 x depth 62 x handle height 87 to 97.5 cm in 6 positions
Seat dimensions	Width 34.5 x depth 30 x height 50 cm
Weight	7,4 kg
Maximum user weight	110 kg

GUARANTEE

This product is guaranteed for two years from the date of purchase by the user. IDENTITIES will provide replacement parts for any part that has broken due to a manufacturing, design or manufacturing deficiencies. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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