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## INSTRUCTIONS MANUAL EN

## IMPORTANT

Read the instructions carefully before using this product and keep them if necessary.

User: If you are unable to read or understand any warnings, precautions or instructions, please contact your health care professional or distributor before using this product or risk and damage may result.

Consult your physician or health care professional before using any assistive device. He/she will advise and inform you on the most appropriate use in your case.

Distributor: this instruction manual must be given to the users of this product.

## PRODUCT DESTINATION

The 4-wheel steel rollator is a walking aid designed for people with reduced mobility. This product can be used both indoors and outdoors. It is equipped with a seat to allow the user to sit and take breaks.

## SAFETY PRECAUTIONS

Failure to follow safety precautions may result in serious injury.

Please read these instructions carefully.

- Do not exceed the maximum weight limit of 150 kg.
- Ensure that all parts (wheels, cables, and handles) are securely fastened and in proper working condition.
- Do not use the rollator on stairs or escalators.
- Check that the handle installation is not loose or chipped to avoid compromising user safety.
- Use only accessories and parts provided by the manufacturer.

- The rollator is a walking aid. It is not a wheelchair and is not designed to transport a seated person. Using it in this way may damage the wheels and the frame, weakening the device and compromising its mechanical performance in future use. Use the rollator strictly as a walking support.
- Do not walk backwards while using the rollator.
- Do not make any adjustments to the rollator while it is in use.
- Always check that the brakes are functioning properly before use.
- All four wheels must always remain in contact with the ground.
- All components must be checked to ensure they are undamaged and securely attached before use.
- Make sure to place your hands on both handles so that the rollator is positioned with even force on both sides.
- Be cautious when stepping up curbs or navigating sloped, uneven, or slippery surfaces. Ignoring this warning may lead to a serious risk of falls or injuries.
- Do not use the rollator on wet, icy, or snowy inclines.
- In case of damaged, broken, or worn-out parts, contact the retailer.
- Be cautious of pinch points when folding the rollator.

## DESCRIPTION OF THE PRODUCT

1. Handgrip
2. Brake housing
3. Adjustment button / Knob
4. 6" rear wheel
5. 6" front wheel with fork
6. Shopping bag
7. Seat
8. Backrest
9. Brake cable

## MOUNTING & USE

### ASSEMBLY

#### Step 1:

Check that all components are included in the package. Lay them out flat on the floor.

**Step 2:**

To unfold the rollator, lift the strap located under the seat. Place the rollator on its wheels, then press down on the center bar until it is fully opened.

**Important:**

Always ensure that the side supports are fully extended before use.

**Step 3:**

Insert the handlebar into the tube, making sure the brake cable is positioned outside the rollator frame. Select a comfortable height, then secure it using the provided tightening knob. Turn clockwise to tighten and counterclockwise to loosen. Repeat the process for the other handlebar.

**Step 4:**

Remove all packaging from the backrest. Insert the ends of the backrest into the two brackets, press the two buttons on the backrest, then push it fully into the brackets.

Make sure the locking pins are properly aligned with the holes (see image below).

To remove the backrest, press both buttons and pull the backrest out of the brackets.

**Step 5:**

Attach the front and rear legs with the wheels.

**Step 6:**

Assembly is complete.

## USE

To fold the rollator for transport:

While holding the rollator by the handle, lift the seat and then use the lifting strap to raise the rollator. As you lift it off the ground, the rollator will automatically fold.

Note: It is recommended to lock the brakes when folding and storing the rollator.

## Brake Operation :

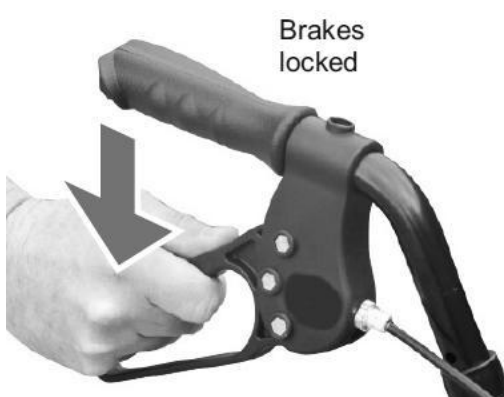
To engage the brakes:

Squeeze the brake levers upward and hold them to activate the brakes (as indicated by the arrow). Release the levers to disengage the brakes.


To lock the brakes:

Push the brake lever downward until it locks and remains in the down position (as indicated by the arrow).

To unlock the brakes:

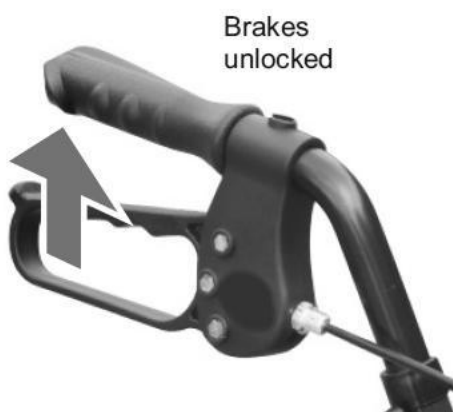Pull the brake lever upward until the lock is released (as indicated by the arrow).

## CARE AND MAINTENANCE

Failure to care for and monitor your product may result in the risk of serious injury.

- Clean the rollator regularly with warm water and a soft cloth. Do not use abrasive products, detergents, or solvent-based cleaners.

- Dry with a clean, soft, dry cloth.
- Ensure there is no build-up of dirt or debris around the brakes or wheels.
- Do not use cleaning products containing phenol or chlorine, as they may damage the rollator's materials.
- Lubricate the wheels with a small amount of oil if they begin to squeak.
- Periodically check for wear and the tightness of all screws and bolts. Immediately replace any broken, damaged, or worn parts.
- In case of malfunction, do not attempt to repair the rollator yourself. Contact an authorized dealer.

#### Checklist :

To keep your walker in good condition, check before each use and regularly as follows.

Check the item	Anytime	Weekly	Monthly
Condition and operation of the brakes	X		
Condition and operation of the wheels		X	
General condition of the product		X	
Cleanliness	X		
Screws and fasteners		X	
Frame stability			X
Condition and operation of the folding mechanism		X	
Condition of the handles		X	

### CHANGE OF OWNERSHIP

The product can be reused. It must first be cleaned and disinfected in accordance with hygiene instructions. When transferring your product, remember to hand over all the necessary technical documents to the new user. Beforehand, the product must be inspected by an authorised specialist.

### RECYCLING

When the product has become unusable, and you have to throw it away: please contact your specialist dealer. If you wish to carry out recycling for yourself, ask your local waste management company for disposal guidelines.

### STORAGE

Failure to observe the storage conditions can lead to deterioration of the product and thus to the risk of serious injury.

- Do not store your product for an extended period of time near a heat source or in the sun (e.g. behind a window or near a radiator) or near a cold source.
- Keep away from all flames and sources of sparks.
- Respect the conditions for storing your product :
- In a dry and temperate place
- Protect your product from dust, corrosion (e.g. abrasive elements, sand, sea water, salty air).
- Store all removed parts together in the same place (or locate them if necessary) to avoid mixing them with other products when reassembling.
- All components must be stored without any load (do not place any objects that are too heavy on the product elements, do not jam anything...).

### PERIOD OF USE

The life expectancy of this product is 5 years under normal use, safety and maintenance conditions for a user. Beyond this period, the product can be used for as long as it is still in good condition.

If the device is used in a multi user environment (such as a nursing home or hospital) then service life may be reduced.

### FEATURES TECHNICAL

Overall Dimensions	Width 61.7 x Depth 60.5 x Height 76.1 to 87.3 cm
Seat Dimensions	Width 33 x Depth 29 x Height 57 cm
Maximum Weight Capacity	150 kg
Weight	7,9 kg

## GUARANTEE

This product is guaranteed for two years from the date of purchase by the user. IDENTITIES will provide replacement parts for any part that has broken due to a manufacturing, design or manufacturing deficiencies. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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