**ACCOUDOIRS DE TOILETTES SMART***Notice d'utilisation***DISPOSITIF MEDICAL****SMART TOILET ARMRESTS***Instructions manual***MEDICAL DEVICE****REPOSABRAZOS DE INODORO SMART***Manual de instrucciones***DISPOSITIVO MEDICO****REF. 811197**

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EN

IMPORTANT

Read the instructions carefully before using this product and keep them if necessary.

User: If you are unable to read or understand any warnings, precautions, or instructions, please contact your health care professional or distributor before using this product or risk and damage may result.


Consult your physician or health care professional before using any assistive device. He/she will advise and inform you on the most appropriate use in your case.

Distributor: this instruction manual must be given to the users of this product.

PRODUCT DESTINATION

Smart toilet armrests are toileting aids designed for people with reduced mobility. These products are intended for indoor use.

PRODUCT DESCRIPTION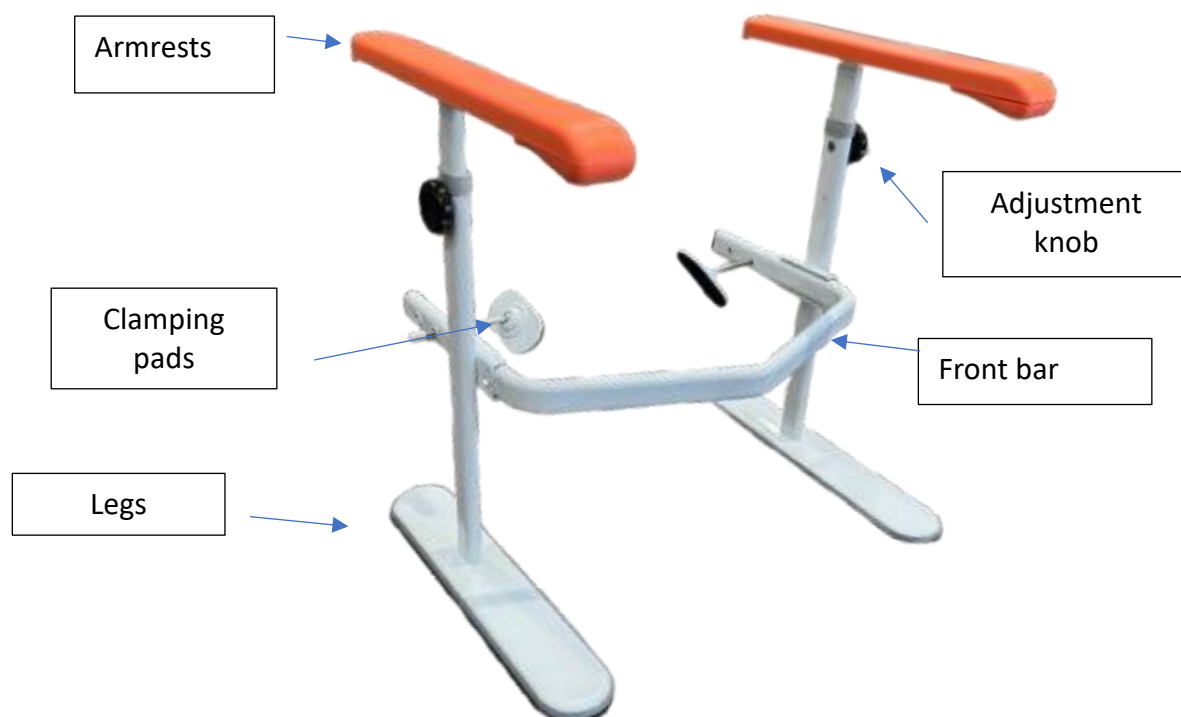

The product is supplied with 2 Allen keys and 16 M8 screws.

SAFETY PRECAUTIONS



Failure to follow safety precautions may result in serious injury.



- Do not exceed the maximum weight limit of 100 kg.
- Use only accessories and parts supplied by the manufacturer.
- Visually inspect the general condition of the frame.
- Make sure all legs are adjusted to the same height.
- Ensure that all parts are properly secured and in good working condition.
- Do not make any adjustments to the toilet armrest while it is in use.
- Do not use the toilet armrest on stairs or on uneven flooring.
- Always place both hands on the handles so that the toilet armrest remains in a stable, parallel position.
- Do not use the toilet armrest as a step or climbing aid.
- Do not use the toilet armrest for walking backward.
- Always lean on the toilet armrest when using it.
- It is recommended to install the product on tiled flooring.
- Install the product in a location where surrounding walls and other equipment do not pose a risk of pinching your hands.
- Ensure that the knobs are securely tightened for height adjustment before each use.
- Test the product after installation.
- In case of damaged, broken, or worn parts, contact your dealer.

ASSEMBLY AND USE

ASSEMBLY :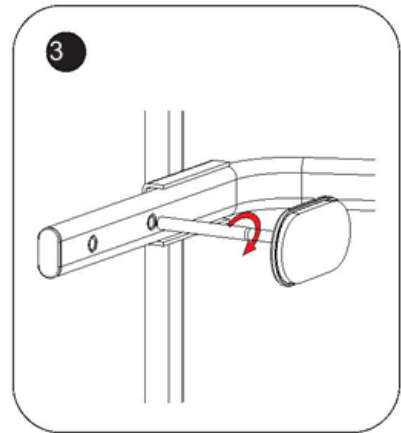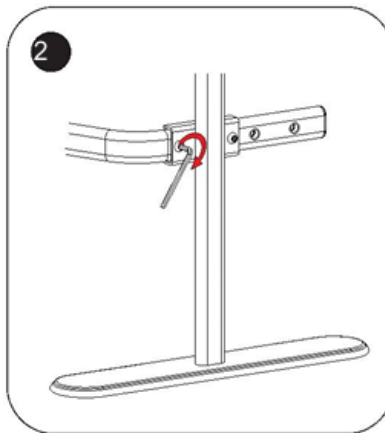

1. Use the screws to assemble the base and the frame of the product.
2. Align the frame of the product with the hole in the front bar and tighten it with screws.
3. Screw the clamping pad into the hole of the front bar, tightening it to the maximum position.

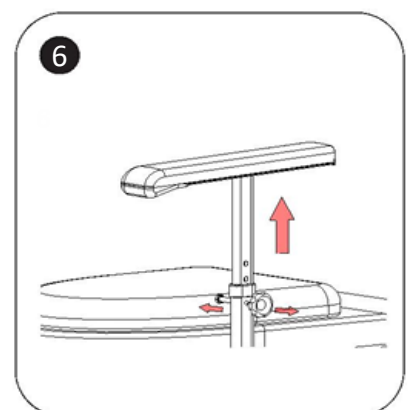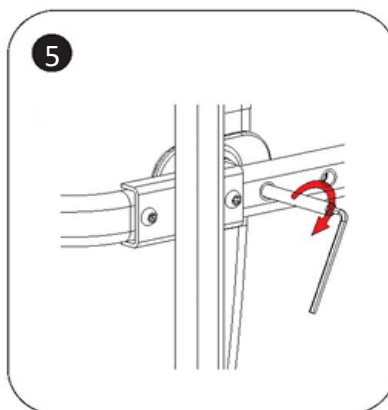4. Push the assembled handrail onto the toilet.
5. Use an Allen key to tighten the clamping pad against the toilet.
6. Armrest height adjustment: loosen the handle screw by turning it counterclockwise and adjust the armrest to the appropriate height. Then align the hex screw with the hole in the armrest and tighten the handle and screw.

After installation, check and confirm the following points :

1. Check if the product is loose. If it is, please reinstall it by following the installation steps.
2. Check if the armrest is securely fixed. If not, inspect the fastening components one by one and retighten them.

USE :

To use the product, place your forearms on the armrests. Adjust the height if necessary. Sit down slowly.

CARE - MAINTENANCE

Failure to maintain and service your product may result in serious injury.

- Regularly clean the product (all fastenings) with damp cloth and a disinfectant. Do not use abrasive cleaners, detergents, or solvent-based products.
- Dry with a clean, soft, dry cloth.
- It is recommended to service the product at least once a year. If the product is subject to intensive use and/or use in a multi-user environment, we recommend having it serviced more frequently.
- In case of malfunction, do not attempt to repair the product yourself. Contact an authorized dealer.
- **Checklist :**

To keep the product in good condition, check before each use, and carry out regular inspections as follows :

Check the item	Anytime	Weekly
Condition and functioning of the armrests	X	
Cleanliness		X
Screws and fastening components		X
Clamping pads and adjustment knobs	X	
Stability of the frame	X	

CHANGE OF OWNERSHIP

The product can be reused. It must first be cleaned and disinfected in accordance with hygiene instructions. When transferring your product, remember to hand over all the necessary technical documents to the new user. Beforehand, the product must be inspected by an authorized specialist.

RECYCLING

When the product has become unusable, and you must throw it away: please contact your specialist dealer. If you wish to carry out recycling for yourself, ask your local waste management company for disposal guidelines.

STORAGE

Failure to observe the storage conditions can lead to deterioration of the product and thus to the risk of serious injury.

- Do not store your product for an extended period of time near a heat source or in the sun (e.g. behind a window or near a radiator) or near a cold source.
- Keep away from all flames and sources of sparks.
- Respect the conditions for storing your product.
- In a dry and temperate place
- Protect your product from dust, corrosion (e.g. abrasive elements, sand, sea water, salty air).
- Store all removed parts together in the same place (or locate them if necessary) to avoid mixing them with other products when reassembling.

- All components must be stored without any load (do not place any objects that are too heavy on the product elements, do not jam anything...).

PERIOD OF USE

The life expectancy of this product is 5 years under normal use, safety and maintenance conditions for a user. Beyond this period, the product can be used for as long as it is still in good condition.

If the device is used in a multi user environment (such as a nursing home or hospital) then service life may be reduced.

FEATURES TECHNICAL

Overall dimensions	Width 64.5 × Depth 48 × Height 62–74.5 cm, adjustable in 6 positions
Width between armrests	51 cm
Armrest length	46 cm
Maximum supported weight	100 kg

GUARANTEE

This product is guaranteed for two years from the date of purchase by the user. IDENTITIES will provide replacement parts for any part that has broken due to a manufacturing, design, or manufacturing deficiencies. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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