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USER MANUAL EN

IMPORTANT

This manual is provided solely as a reference for the operation of the product. Please read and understand the user manual before using the product, as the manufacturer cannot assume any consequences or liability for any damage incurred from using this manual for other purposes, or if the user fails to perform required maintenance activities or ignores warning messages.

Please familiarize yourself with the device by reading this manual before putting it into service. If you do not understand any part of this manual or require additional services, please contact the local dealer authorized by our company.

The following symbols are used to indicate warnings and precautions. These symbols appear throughout the user manual and are very important! It is essential that you follow the specified warnings and safety instructions.


- This lift transfer chair is specifically designed for short-distance indoor movement of disabled, elderly, or weak individuals weighing less than 150 kg, such as transfers between a bed, sofa, wheelchair, toilet, table, and bathroom. It is an ideal mobility aid for use in homes, hospitals, nursing homes, and other flat-surface environments. It is not recommended for use in outdoor environments such as lawns, gravel roads, slopes greater than 6°, or vehicle traffic areas.
- Caregivers can easily transfer patients by controlling the chair's button, which greatly improves care efficiency. It helps caregivers move immobile patients to the living room, toilet, or bathroom. In summary, this significantly reduces caregivers' physical effort and lowers the risk of secondary injuries during the transfer process.
- This product is detachable, easy to transport, and convenient to store.
- • This transfer chair may be fitted, as an option, with a MADAGASCAR suspension kit. This kit allows the addition of a suspension system with a strap to assist the user's transfer under suitable conditions. Installation of the kit is described at the end of the instructions for use (SECTION: INSTALLATION OF THE MADAGASCAR SUSPENSION KIT). Before use, the user or caregiver must ensure that the kit is correctly secured, that the locking elements are properly in place, and that the maximum authorised load of the device is not exceeded.

Important: Use of the suspension kit does not exempt the user from complying with the general safety instructions, weight limits, maintenance recommendations, and warnings stated in these instructions for use. Any improper use may result in a risk of falling, injury, or product damage.

PRODUCT COMPONENTS

Parts List :

1. Backrest x1
2. Main frame x1
3. Seat x1
4. Seat (with holes) x1
5. Power adapter x1
6. Mounting accessories x1
7. Feet with braked wheels x2

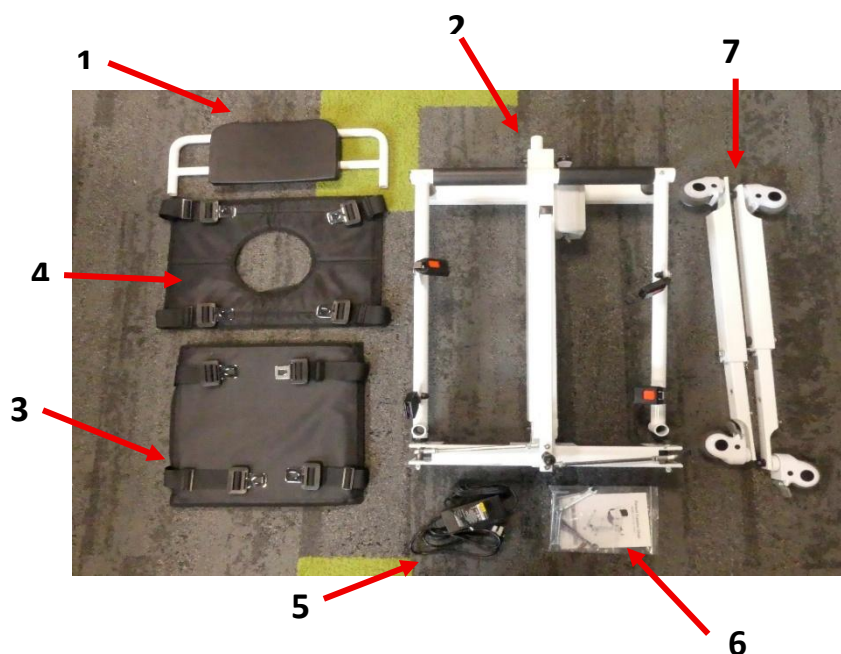SAFETY PRECAUTIONS

This transfer chair is an active, non-sterile, and reusable product. Please read the instruction manual for the correct use of this product. If you have specific needs, please contact the authorized local dealer or after-sales service.

Safety Warning

- Please keep this manual in an easily accessible place and store it properly. This instruction manual clearly explains how to use the equipment safely and correctly to prevent any damage or harm to yourself or others.
- It is essential to use the adapter compatible with this product when charging.
- Use in the shower only if necessary, after removing the battery, switching off the power supply, and protecting the charging port with its watertight cover. Never charge the chair when it is wet.
- The adapter light turns red during charging. Charging stops automatically when the battery is fully charged, and the indicator turns green.
- Do not open the battery case while charging.
- Do not use the product while charging.
- Do not use this product, including with the MADAGASCAR suspension kit and harness strap, if the user's weight exceeds 150 kg. The maximum authorised load of the chair, suspension kit, and strap must never be exceeded.
- Keep the product out of reach of children to prevent injury.
- It is strictly forbidden to use the chair if objects obstruct it, as this may damage the device.
- Our company assumes no responsibility for any loss or damage resulting from failure to comply with these warnings and safety instructions. Please pay special attention to them.
- Never use a transfer chair that does not work properly or is damaged.
- Check the safety buckles twice after securing the patient.
- Always spread the base wide when performing static transfers.
- If the transfer chair is not functioning correctly or if the wheel brakes are defective, stop using it and contact your local dealer or after-sales service.
- To reduce the risk of fire and electric shock, do not place the chair or battery in a humid environment.
- If the equipment emits smoke or is in any other abnormal condition that may pose a danger (such as fire), immediately disconnect the power supply and contact your local dealer or after-sales service.
- If the battery or electrical components get wet, which may cause dangerous situations (such as a short circuit), immediately remove the battery case and contact your local dealer or after-sales service.
- Do not disassemble or modify the equipment. Unauthorized modifications will void the warranty.
- Before charging the chair, please ensure that the power cord is not damaged. Do not use a damaged charger, and contact your local dealer or after-sales service.
- In case of thunder or lightning, disconnect the power supply and stop charging.
- Do not touch the charger power cable with wet hands to avoid electric shock.
- Do not use insecticides, alcohol, thinners, or chemical liquids to clean the equipment, as this may cause peeling or deterioration of the equipment surface.
- The recommended service life of the transfer chair is five years under normal use. Prolonged use may cause the mechanism to age, malfunction, or even lead to accidents. Please contact a qualified medical device recycling agency for proper recycling and replacement.
- Do not touch the electrical ports, as this may injure the patient.
- All maintenance, repair, or parts replacement must be performed by qualified and authorized personnel. Any unauthorized intervention will void the warranty.
- This transfer chair is equipped with four wheels with brakes. Before moving the chair, ensure that all four brakes are released. After reaching the target position, press the brake pedals to lock the wheels.
- When transferring the patient to the destination, the caregiver must lower the chair to an appropriate height while ensuring safety to prevent injury or property damage. When lowering, stop in advance as you approach the target object (sofa, bench, bed, toilet, etc.), then press the down button briefly to adjust the height gradually. If the target object is hard, avoid direct contact between the transfer chair and the object (leave a space of 1 to 2 cm).
- The service life of this transfer chair in clinical use is five years. It depends not only on the structural strength but also on usage conditions, user habits, and proper maintenance.
- Before sitting in the transfer chair, ensure that the front wheel brake is properly locked to prevent movement; otherwise, there is a risk of tipping or falling.
- Before moving, ensure that the backrest is properly locked and that the patient is securely seated.
- It is forbidden to charge the battery during or after a shower when the chair is still wet.
- Do not use the transfer chair without a caregiver, as this may cause injury.
- The patient is NOT the intended operator. When the patient is sitting alone in the transfer chair, they should not operate it by themselves (in case there are obstacles under the seat). Do not press the lift/lower button without assistance, as the patient may not notice obstacles under the seat.
- When the caregiver uses the transfer chair, do not move the chair if the lock is not engaged and if the patient's safety belt is not fastened, as this could result in injury.

- The transfer chair must not be serviced or repaired while a patient is seated.
- Keep your feet away from the column motor to avoid injury when the chair is lowering.


ELECTROMAGNETIC INTERFERENCE PREVENTION

- Do not use portable radio transmitters when the transfer chair is powered on, and avoid approaching radio transmission systems, such as radio stations.
- Under extreme conditions, electromagnetic interference may occur between this device and other equipment. Do not use this device near sensitive equipment or equipment that may produce strong magnetic interference.
- Caution : Mobile phones or anti-theft systems may affect the normal operation of the chair.

WHAT YOU NEED TO KNOW BEFORE FIRST USE

- Please do not use the transfer chair before fully reading and understanding this manual.
- The maximum load specified for this transfer chair is 150 kg, including when used with the MADAGASCAR suspension kit and the sling harness. The device is limited to use by one person only, and this maximum load must never be exceeded.
- The caregiver must have the necessary physical strength, intelligence, and endurance to operate the transfer chair safely. They must not have visual impairment and must be able to read and understand the manual without difficulty. Before first use, please practice several times without a passenger.
- Do not use the chair in low light conditions or if you have blurred vision.
- The protection rating of this product is IP54. Please do not expose it to moisture or shower water for prolonged periods. The product may be used in the shower after removing the battery, switching off the power supply, and protecting the charging port with its waterproof cover.
- Do not use the transfer chair if the temperature is below 10°C or above 40°C.
- Please charge the battery before using the chair for the first time.
- When the ambient temperature is below 0°C, do not charge the battery to avoid damaging it or causing injury.
- Please check if the packaging is damaged before unpacking, in case you need to request repair or replacement. Without the original packaging, returns will not be accepted.


Electrical Operating Principle :

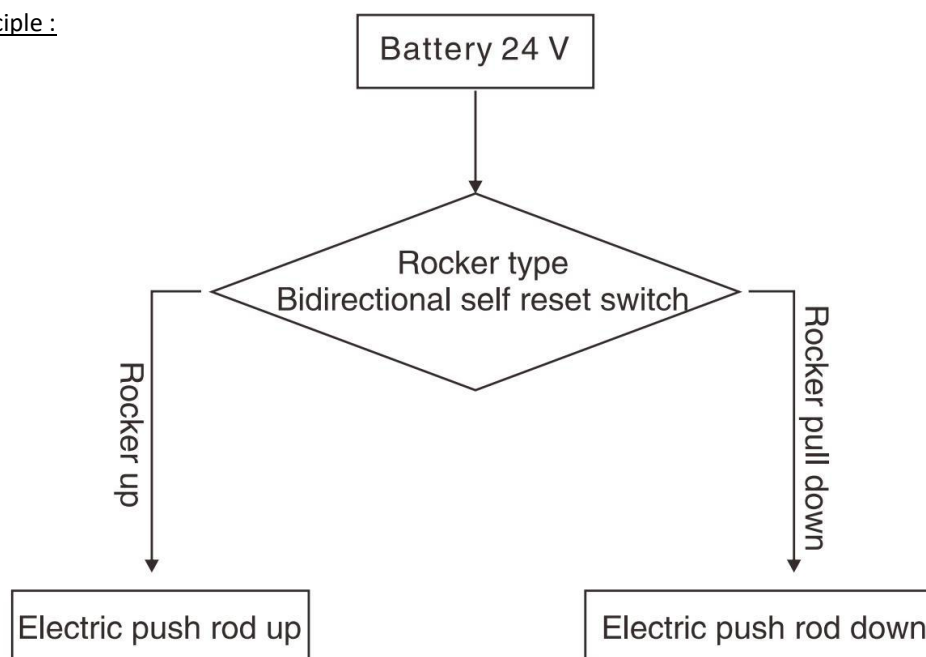INSTALLATION AND START-UP


1. First, remove the pre-installed screws on both sides of the frame, as well as the pre-installed screws on the left and right legs.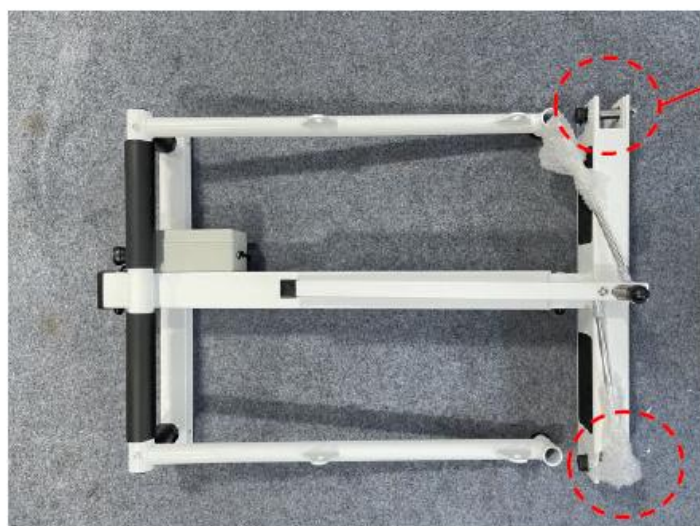

2. Next, install both legs onto the frame (make sure to correctly distinguish the left and right sides), then connect the two connecting rods to the base of the frame as shown in the diagram.

(Note: Please strictly follow the left and right installation order.)

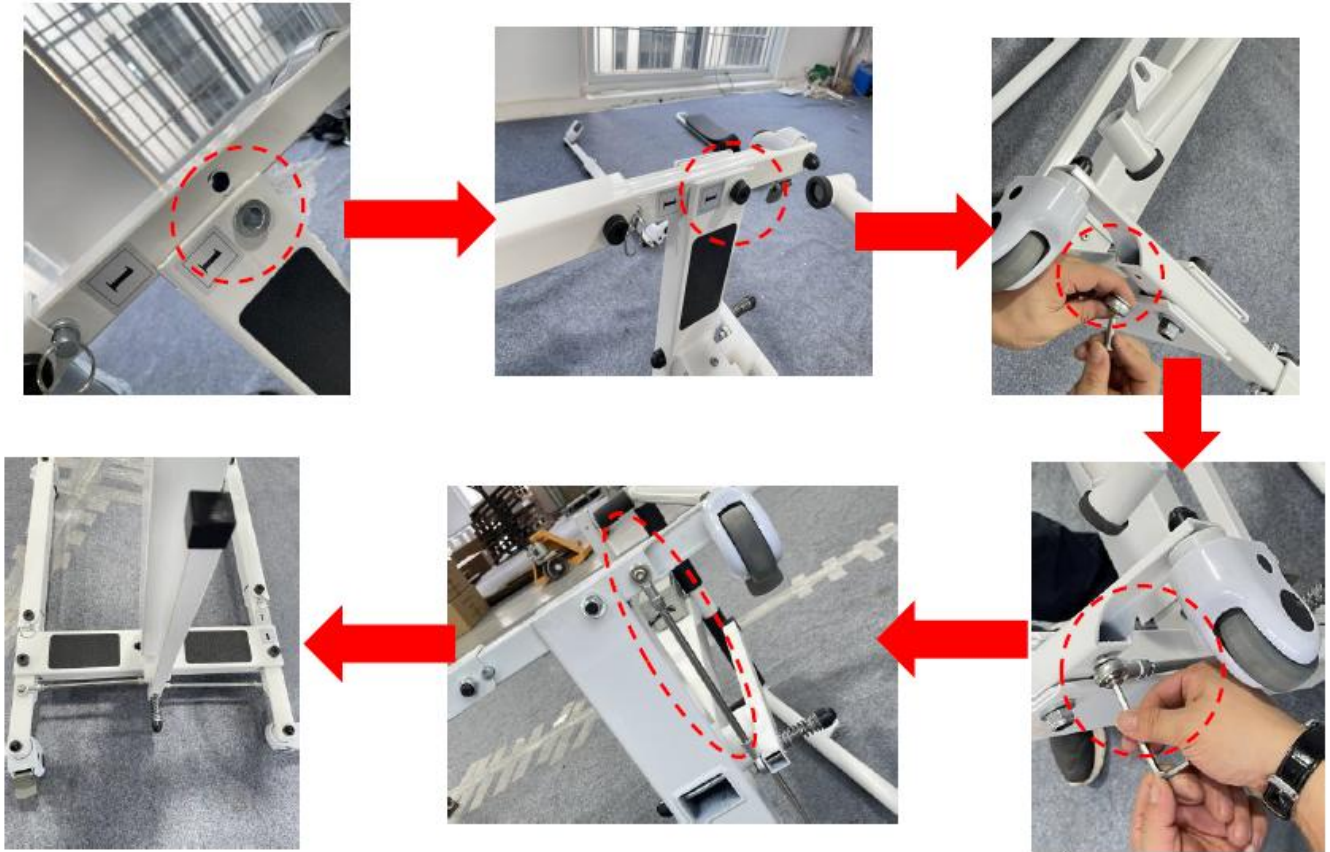
3. Lift the frame until the armrests are in the fixed position, install the seat cushion, fasten the safety buckle, and then install the backrest.

- After charging, the product can be used normally. The height can be adjusted up and down to the desired level.
- Before use, it is essential to check that the central axis of the frame is properly inserted to ensure safety.
- According to the desired height, press the lift button until the appropriate height is reached, then release to stop. If the button is not released, the product will automatically stop at its highest point. The power supply of this product is designed to operate at low voltage, so you can use it safely.

Product Controller Diagram :

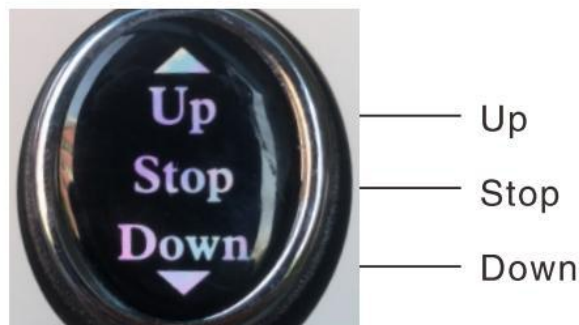INSTALLATION OF THE MADAGASCAR SUSPENSION KIT

1. Installation of the upper assembly with electric actuator

1. Present the upper assembly with electric actuator and identify marks I and J as shown in diagram 1.
2. Position the two parts of the upper assembly facing each other, then align the fixing holes as shown in diagram 2.
3. Insert fixing screws d into the designated locations, then fit the nuts and washers in accordance with diagram 3.
4. Tighten the fixings using a suitable tool, without applying excessive force, to avoid damaging the parts.
5. Install the protective caps on the fixings, as shown in diagrams 4 and 5.
6. Position the rod of the electric actuator on the designated fixing point, as shown in diagram 6.
7. Insert fixing screws d, then tighten the assembly securely using a suitable tool, as shown in diagram 7.
8. Fit the protective cap onto the actuator fixing, as shown in diagrams 8 and 9.
9. Check that the upper assembly is correctly assembled, that the fixings are securely tightened, and that no component has abnormal play.
10. Once the upper assembly has been assembled, install vertical mast F into its designated housing on the chair, as shown in diagrams 11 to 13.
11. Ensure that the vertical mast is correctly inserted, stable, and locked before any use.
12. After assembly, check the correct positioning of the upper assembly with electric actuator, the cable, and the moving parts, as shown in diagram 14.

2. Installation of the sling harness

1. Fully unfold the sling harness before installation.
2. Attach the ends of the sling to the designated attachment points.
3. Check that each carabiner or attachment system is properly closed and locked.
4. Ensure that the straps are not twisted, worn, cut, or damaged.
5. Check that the sling is correctly distributed and positioned before placing the user in it.
6. Before each transfer, check that the sling and suspension kit are securely attached.

Warning: Never use the sling harness if a carabiner is not properly closed, if a strap is twisted, or if any component shows signs of wear or damage.

TROUBLESHOOTING

If the transfer chair cannot move up or down, please use the charger to help identify the issue. The charger has an indicator light. Follow the guide below for troubleshooting :

Check if the battery is charged :

Step 1: Connect only the battery side to the charger.

- No light on the charger → The battery is empty and needs to be recharged.
- Green light, but the transfer chair still does not move → The battery may be damaged.

Step 2: Then connect both ends of the charger.

- If the charger had no light before and now shows red → Charging is in progress.
- If the light remains green or is flashing → The battery is likely damaged.
- If the charger had a green light before, and after connecting both ends it stays green or quickly changes from red to green → The battery is likely damaged.

Note: If the issue persists after these checks, please contact your service provider or the manufacturer to replace the battery.

MAINTENANCE AND DISINFECTION

- It is recommended to wipe the seat ring with a soft cotton cloth and alcohol before use, then disinfect it after cleaning. The transfer chair and its accessories should be cleaned with medical alcohol or other non-corrosive disinfectants.
- During cleaning and disinfection, do not charge or use the transfer chair.
- It is strictly forbidden to wash or disinfect the transfer chair and its accessories by immersion in liquid.
- It is strictly forbidden to use corrosive detergents to clean the transfer chair and its accessories.
- It is strictly forbidden to use corrosive disinfectants to disinfect the transfer chair and its accessories.
- The transfer chair and its accessories can only be used after the cleaning and disinfecting agents are completely dry.
- When the transfer chair is not used for a long period, it is recommended to take it out and charge it regularly to avoid battery failure.
- Do not leave the transfer chair continuously plugged in. It is recommended to use the battery at least once a week and to fully discharge it before recharging.
Although lithium batteries do not have a memory effect, this practice helps to extend battery life.
- The battery must be disposed of in accordance with local regulations when the transfer chair is discarded.
- It is not necessary to clean and maintain the transfer chair after each use. When the battery is empty, simply recharge it to continue using it normally.
If shaking is observed during lifting or lowering, apply a small amount of electric lubricating oil to the two long shafts.
- After a certain period of use, check that the screws in each part are not loose. If necessary, tighten them promptly.
- The internal power supply of this transfer chair operates on low direct current (DC) voltage.
If water is present on the surface of the chair, it can be wiped off with a towel. Never immerse the transfer chair in water.

• Checklist :

In order to keep the transfer chair in good condition, check before each use and inspect regularly as follows:

Check Item	Anytime	Weekly	Monthly
Four wheels	X		
Locking straps	X		
Battery level	X		
Control panel function	X		
Cleanliness	X		
Screws and fasteners		X	
Lifting motor column			X
Frame stability			X
Seat condition			X
Brakes	X		
Condition and operation of the suspension kit	X		

Operating Conditions :

- Ambient temperature range : +10°C to +40°C
- Relative humidity range : 30% to 75%
- Atmospheric pressure range: 700 hPa to 1060 hPa
- Required environment: Indoors, dry, low dust, free from corrosive or flammable materials.

Storage Conditions :

- Ambient temperature: 0°C to +50°C
- Relative humidity: ≤ 93%
- Atmospheric pressure range: 500 hPa to 1060 hPa
- Required environment: No corrosive gases, well-ventilated room.

TECHNICAL SPECIFICATIONS

- Type of mobility: Indoor
- Waterproof rating: IP54
- Overall dimensions: Width 60.5 cm x Depth 87 cm
Leg spacing: 60.5 to 91 cm. Equipped with 2 braked wheels. Usable width within the frame : 51 cm.
Handle height: 86.5 to 122 cm. Seat height adjustable thanks to the adjustable seat straps.

- Removable backrest: Width 33 cm x Height 20 cm.
- Net Weight : 33.5 kg
- Maximum user weight : 150 kg
- Li-on Battery : 2Ah, 44.4Wh
- Motor DC24V

WARRANTY

This product is guaranteed for two years from the date of purchase by the user. IDENTITÉS will provide replacement parts for any component that breaks due to a manufacturing, design, or material defect. Please contact the distributor from whom you purchased the product if you need to make a warranty claim.

Failures resulting from accidental damage, misuse, modifications, or normal wear and tear are not covered by this warranty. Contact the reseller from whom you purchased the product to apply the warranty.

The warranty covers only manufacturing and structural defects (excluding wear parts: wheels, brakes, handles, natural wear of the upholstery and foam, plastic components, etc.).

Electronic parts: 6-month warranty for the motor, charger, batteries, cables, fuses, controller, and joystick, provided the product is used correctly.

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