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IMPORTANT

Read the instructions carefully before using this product and keep them if necessary.

User: If you are unable to read or understand any warnings, precautions or instructions, please contact your health care professional or distributor before using this product or risk and damage may result.

Consult your physician or health care professional before using any assistive device. He/she will advise and inform you on the most appropriate use in your case.

Distributor: this instruction manual must be given to the users of this product.

PRODUCT DESTINATION

The Neo Fold rollator is a walking aid designed for people with reduced mobility. This rollator is suitable for both indoor and outdoor use and can be folded to make storage or transport easier.

PRODUCT DESCRIPTION

1. Push handles
2. Brake handles
3. Backrest
4. Handle adjustment
5. Seat
6. Storage bag
7. Rear wheels
8. Front wheels
9. Footrest

SAFETY PRECAUTIONS

Failure to follow the safety precautions may result in the risk of serious injury.

- Do not exceed the maximum weight limit of 136 Kg.
- Do not exceed the maximum load limit in the basket (5kg). Only transport loads in the basket. Do not hang heavy objects on the rollator as this could affect its stability.
- Make sure that all parts (wheels, cables, handles and knobs) are secure and in good working order.
- Do not use the rollator on stairs or escalators.
- Only use accessories and parts supplied by the manufacturer.
- The rollator is a walking aid. It is not a wheelchair. It is not designed to carry a person sitting on it. Such use may damage the wheels and the structure of the rollator itself. The rollator could be weakened and no longer meet its mechanical characteristics in future use. Only use the rollator as a walking support.
- Do not use the rollator to walk backwards.
- Do not make any adjustments to the rollator while it is in use.
- Check that the brakes work properly before use.
- All wheels must always be in contact with the ground.
- When the rollator is in the stationary position, the brakes must be locked. Do not attempt to move the rollator if the brakes

are locked.

- Be sure to put your hands on each handle so that the rollator can be in the parallel force position.
- Use caution when climbing pavements or walking on sloping, uneven or slippery surfaces. Failure to do so may result in a significant risk of falling or injury.
- Beware of the risk of pinching when folding.
- Do not use the walker on a wet, icy, or snowy slope.
- If any parts are damaged, broken or worn, contact the dealer.

MOUNTING & USE

Using the Rollator – Opening and Closing :


Opening the Rollator :

To open the rollator:

1. Pull up on the handle located on the seat. (See Image 1)
2. Continue pulling until the arms lock into position.
3. Then, push both side bars down until the seat is flat and you hear an audible "click" indicating the seat is securely locked. (See Images 2 and 3)
4. **DO NOT** use the seat or the rollator if you do not hear this "click."

Correct Use of the Rollator :

- It is essential not to leave a large gap between you and the rollator during use, to avoid losing your balance or allowing the rollator to move away from you. (See Images 4 and 5)
- The handles must be adjusted to the correct height. (See Image 6)
 - To adjust to the correct height: stand upright with your arms at your sides, and set the handles at wrist level.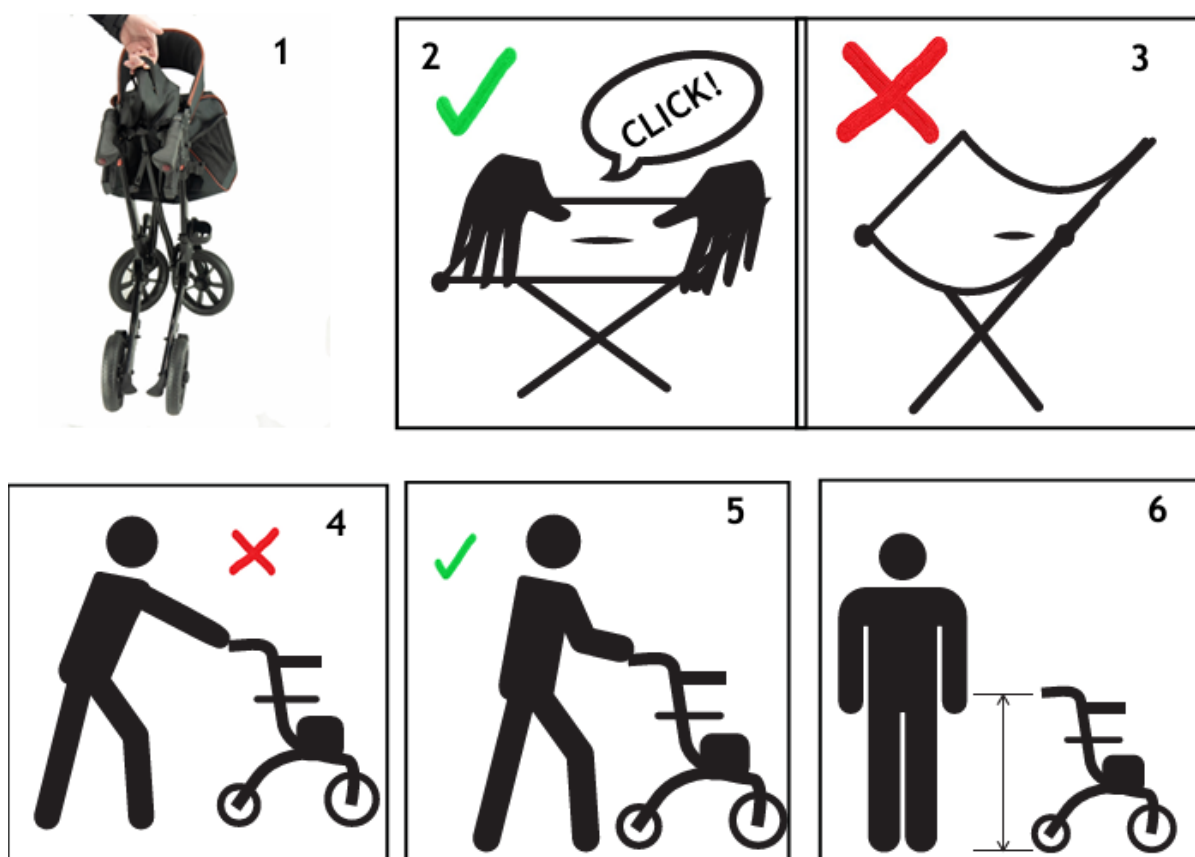

Adjusting your Rollator :


Several parts of your rollator can be adjusted to suit your needs. These adjustments can be made as follows :

Push handles :

The height of the handles can be adjusted by pressing the grey button located at the front of the handle section, then raising or lowering the handle to the desired height.

(See Image 7)

When the handle reaches the closest height to your desired position, you will hear an audible "click" indicating that the handle is locked in place.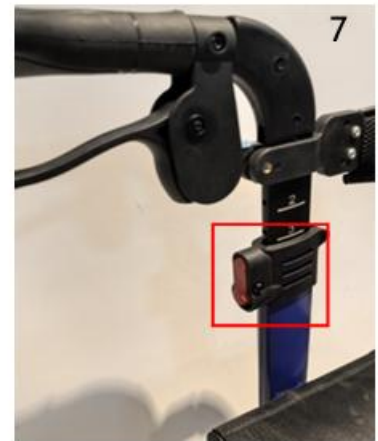

Adjusting the backrest height :

The backrest is height-adjustable to provide greater comfort when using the rollator seat.

To adjust the backrest height :


1. Locate the star-shaped grey knob on each side of the frame. (See Image 8)
2. Turn the knob counterclockwise to release the backrest.
3. Adjust the backrest to the desired height.
4. Once the desired height is reached, turn the knob clockwise until it is fully locked. (See Image 9)
5. Repeat the procedure on both sides.

Removing the storage bag :

When folding the rollator, you may remove the storage bag if desired, although it is not necessary to do so. To remove the bag, follow these quick and simple steps :

- The bag can be easily slid out from the frame tubes on both sides. (See Images 10 and 11)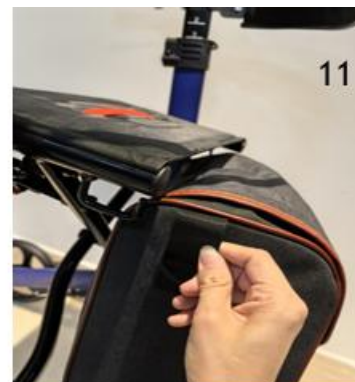


Replacing the bag :

To replace the bag, simply follow the reverse steps of the removal process described above.

Using the brakes :

If you want to stop the rollator while pushing it, gently pull the brake handles upward toward the grips until the rollator comes to a stop. Do not attempt to push the rollator while the brakes are engaged. (See Image 12)

If you want to park the rollator, push the brake handles downward until they lock into position. (See Image 12).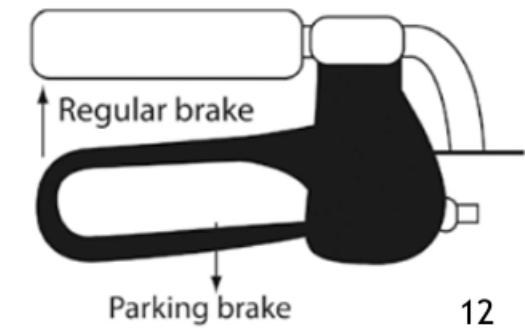

- If you need to rest by sitting on the rollator seat, ALWAYS make sure the parking brakes are locked.
- NEVER attempt to push the rollator while the seat is in use.
- NEVER release the parking brakes while using the seat.

Curbs :

If you need to cross a street and no lowered curb is available, follow these simple steps :

Going up a curb :



Push the rollator forward until the front wheels touch the curb.

Gently lift the front wheels using the footrests. (See Image 13)

Push forward until the front wheels are on the sidewalk. (See Image 14)

- Push forward until the front wheels are above the sidewalk. (See Image 15)
- Gently lower the front wheels, then lift the rear wheels. (See Image 16)
- Gently push the rollator forward until the rear wheels are on the sidewalk, then carefully lower them onto the surface. (See Image 17)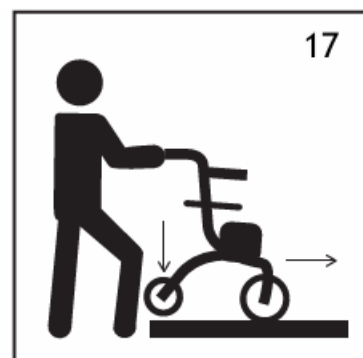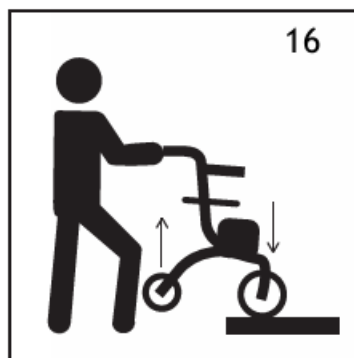

Going down a curb :

Follow these steps:


- Push the rollator forward until the front wheels go slightly over the curb.
- Gently lower the front wheels onto the road surface.
- Push the rollator forward until the rear wheels are above the road surface.
- Carefully lower the rear wheels until they touch the ground.
- Push the rollator forward until you reach the opposite curb.
- NEVER place any load on the rollator seat while going up or down a curb.

Folding the Rollator :

To fold the rollator, follow these simple steps:

1. Place one hand on the strap located on top of the seat.
2. Pull the strap upward; this will automatically begin to close the frame.
3. Bring the sides together until they meet in the center.
4. Push the handles downward until the locking pin clicks into place. You will hear a “click” when the pin has properly secured the rollator.

Images 18 to 21 illustrate the step-by-step process.

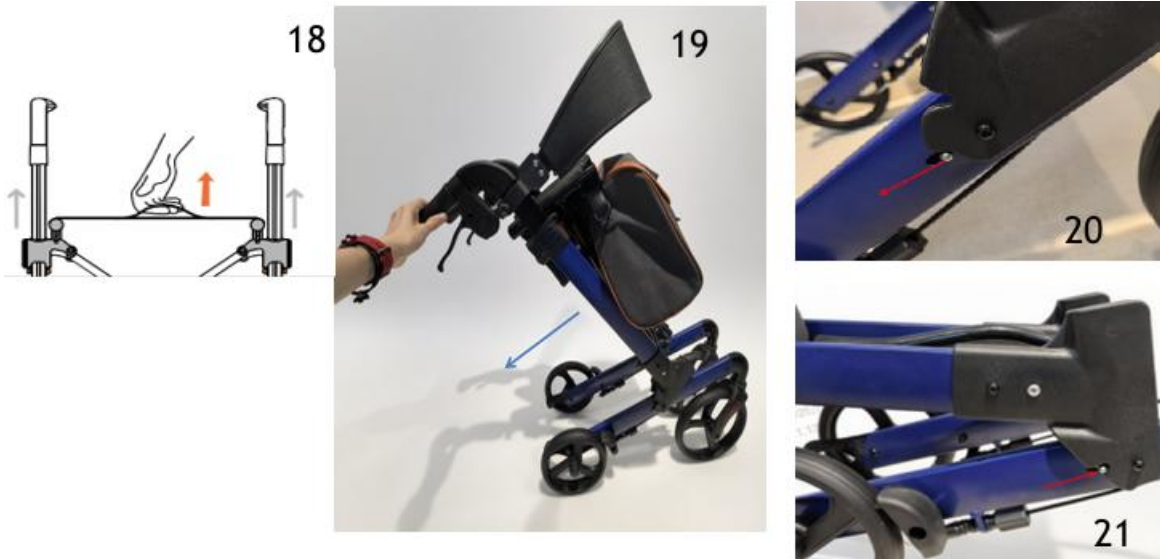CARE AND MAINTENANCE

Failure to care for and monitor your product may result in the risk of serious injury.

- **Tires :**
Tires should be checked regularly for signs of wear. If you notice excessive wear, replace the tires as soon as possible. The outdoor version is equipped with a tread, which should be kept clean and free from significant wear. If you notice that the tread is starting to wear down, we recommend replacing the tires promptly, as this may affect the grip they provide.
- **Brakes :**
When you receive your new rollator, the brakes will have been calibrated to work effectively and provide optimal grip. However, over time, the brakes may become loose or worn. If you notice any of these signs, it is time to have the brakes recalibrated. This can be done by your local dealer or any qualified technician.
- **Frame :**

The frame is the most important part of your rollator and must be kept in good condition. Before each use, inspect the frame by asking yourself the following questions:

1. Does the frame open and fold easily?
2. Does the rollator move straight, or does it pull to one side?
3. Are there any visible signs of damage to the frame?
4. Does the seat still make a "click" sound when fully opened?

If you notice any issue with one of these points, DO NOT use the rollator, Have it repaired by a certified dealer or technician.

- **Wheels :**

Always check the wheels before using the walker. Make sure all the wheels turn freely and smoothly. The front wheel forks should also swivel to allow steering. If they are stiff, adjustment is needed.

No wheel should have any play when pushing the walker. If you notice wobbling or unusual movement, this may indicate that one or more wheels are coming loose — they must be checked immediately.

- **Seat :**

The seat should show no signs of wear. If you notice holes, thinning fabric, or tears, DO NOT USE the seat. It may not support your weight and must be replaced.

- Clean the rollator regularly with warm water and a soft cloth. Do not use abrasive products, detergents or solvents.
- Dry with a clean, dry soft cloth.
- Make sure there is no accumulation of dirt or debris around the axle, brakes or wheels.
- Do not use cleaning products containing phenol or chlorine.
- Lubricate the wheels with a little oil if they squeak.
- Periodically check the wear and strength of all screws and bolts. Replace all broken, damaged or worn parts immediately.
- Check that the brakes are working before each use.
- Check the parking brakes by pushing the handles into the locked position, then trying to push the rollator.
- If there is a malfunction, do not repair the rollator yourself. Consult an authorized dealer.

- **Checklist :**

To keep the rollator in good condition, check before each use, and check regularly as follows :

Check the item	Anytime	Weekly
Clip and knob for height adjustment		X
Structural weld at wheels		X
Cleanliness	X	
Screws and fasteners	X	
Stability of frame and backrest	X	
Condition of seat		X
Brakes	X	

CHANGE OF OWNERSHIP

The product can be reused. It must first be cleaned and disinfected in accordance with hygiene instructions. When transferring your product, remember to hand over all the necessary technical documents to the new user. Beforehand, the product must be inspected by an authorised specialist.

RECYCLING

When the product has become unusable and you have to throw it away: please contact your specialist dealer. If you wish to carry out recycling for yourself, ask your local waste management company for disposal guidelines.

STORAGE

Failure to observe the storage conditions can lead to deterioration of the product and thus to the risk of serious injury.

- Do not store your product for an extended period of time near a heat source or in the sun (e.g. behind a window or near a radiator) or near a cold source.
- Keep away from all flames and sources of sparks.

- Respect the conditions for storing your product :
- In a dry and temperate place
- Protect your product from dust, corrosion (e.g. abrasive elements, sand, sea water, salty air).
- Store all removed parts together in the same place (or locate them if necessary) to avoid mixing them with other products when reassembling.
- All components must be stored without any load (do not place any objects that are too heavy on the product elements, do not jam anything...).

PERIOD OF USE

The life expectancy of this product is 5 years under normal use, safety and maintenance conditions for a user. Beyond this period, the product can be used for as long as it is still in good condition.

If the device is used in a multi user environment (such as a nursing home or hospital) then service life may be reduced.

FEATURES TECHNICAL

Overall dimensions	Width 60 x depth 65,5 cm
Handles height	74/87 cm (Size S) or 88/102 cm (Size L) adjustable in 6 positions
Seat dimensions	Width 43,5 x depth 21 x height 50 cm (Size S) or 62 cm (Size L)
Maximum bag load	5 kg
Maximum supported weight	136 kg

GUARANTEE

This product is guaranteed for two years from the date of purchase by the user. IDENTITIES will provide replacement parts for any part that has broken due to a manufacturing, design or manufacturing deficiencies. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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