**DEAMBULATEUR CAVALETTA***Notice d'utilisation***DISPOSITIF MEDICAL****CAVALETTA WALKING FRAME***Instructions manual***MEDICAL DEVICE****DEAMBULADOR CAVALETTA***Manual de instrucciones***DISPOSITIVO MEDICO****REF. 826318S – 826318M – 826318L**

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EN

IMPORTANT

Read the instructions carefully before using this product and keep them if necessary.

User: If you are unable to read or understand any warnings, precautions or instructions, please contact your health care professional or distributor before using this product or risk and damage may result.

Consult your physician or health care professional before using any assistive device. He/she will advise and inform you on the most appropriate use in your case.

Distributor: this instruction manual must be given to the users of this product.

PRODUCT DESTINATION

The Cavaletta walker is a walking aid device designed for people with reduced mobility. This walker is intended for indoor use and folds for easy storage or transport.

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Failure to follow the safety precautions may result in serious injury.

- Do not exceed the maximum weight limit of 85 kg, applicable to all sizes.
- Ensure that all parts (wheels and handles) are properly secured and in good working order.
- Do not stand up or sit down abruptly on the edge of the seat, as this may damage the product.
- Do not use the walker on stairs or escalators.
- Use only the accessories and parts supplied by the manufacturer.
- The walker is a walking aid. **It is not a wheelchair.** It is not designed to transport a person seated on it. **Such use may damage the wheels and the structure of the walker.** This could weaken it and compromise its mechanical properties for future use. Use the walker only as a walking support.
- Do not use the walker to walk backward.
- Do not make any adjustments to the walker while it is in use.
- Check the proper functioning of the brakes before use.
- All wheels must always be in contact with the ground.
- This walker is equipped with a directional locking system and an anti-reverse mechanism to ensure safe and stable use.
- Always place your hands on each handle so that the walker can maintain parallel force.
- Be cautious when using on sloped, uneven, or slippery surfaces. Failure to follow this instruction may result in a significant risk of falls or injuries.
- Do not use the walker on wet, icy, or snowy slopes.
- If any parts are damaged, broken, or worn, contact the retailer.
- It is essential that the products be thoroughly inspected at least once a year and periodically reviewed by a qualified professional to verify their suitability for the user and their needs. The reviewer is responsible for ensuring that any product is safe and stable for the user.
- This product contains small parts that could present a choking or strangulation hazard for young children.
- To avoid fire hazards, do not place the products near exposed heat sources such as cigarettes, electricity, or open flames.
- To prevent injury, ensure the user is kept away when folding and unfolding the product.
- Do not use the walker if any part is broken, torn, or missing. Damaged or worn parts must be repaired immediately.
- Never leave the user unattended while using the product.
- Avoid using the product on rough or uneven surfaces, near swimming pools, or close to stairways.
- Always use straps and supports correctly, ensuring they are adjusted for comfort and safety.
- Keep straps away from the user's neck to prevent the risk of strangulation.
- Tighten all adjustment knobs and buckles before use and after any adjustments.
- Position supports so that the user's weight is centered between the wheels.
- Always lock the brakes when parking, even on flat surfaces, and before making any adjustments.
- Ensure there is sufficient space for movement during transfers.
- Do not use this product for users whose height and weight exceed the specified limits.

- Regularly check for loose screws, cracks, and other signs of wear.
- Remove the product from use if any unsafe condition is detected.
- Do not use walker components for purposes other than those intended.
- Adequately supervise the user to avoid excessive movement or sudden stops.
- Ensure the user's feet remain within the frame boundaries.
- Exercise caution on uneven ground, ramps, or with users prone to involuntary movements or seizures.
- Do not add any accessories other than those approved by IDENTITES 49.
- Any modifications or alterations carried out by persons other than an IDENTITES 49 authorized technician will void the warranty, and IDENTITES 49 accepts no liability for any injury or damage caused.
- Any serious incident related to the products must be reported to the manufacturer and to the competent authority of the country where the user is established.

ASSEMBLY AND USE

1. Frame :

1.1 Unfold the base frame

- Remove the base frame from the carton and lock the rear casters.
- Lift the upper white tube (see Figure 1.1b).
- Stand the black support bar upright and insert the white tube into its slot (see Figure 1.1c).
- Align the holes on the white tube and the black support bar, then secure them with bolts (see Figure 1.1d). A 4 mm hex wrench is required.
- Ensure the base is securely assembled by gently shaking it from side to side.

1.2 Fold the base frame

- Lock the rear casters and remove all handles and accessories.
- Loosen the bolts connecting the white tube to the black support bar.
- Lift the white tube and lower the black support bar.
- Lower the white tube (see Figure 1.1a)

1.1 a

1.1 b

1.1 c

1.1 d

1.3 Height Adjustment

- Lock the rear casters and unlock the safety mechanism (see Figure 1.3a).
- Press the silver button using a tool and adjust the height one level at a time (see Figure 1.3b).
Avoid adjusting multiple levels at once to prevent damage.
- Make sure the silver button pops back out, then re-engage the safety mechanism (see Figure 1.3c).
- d. Gently press the lower frame up and down to check that it is securely fixed.

1.3 a

1.3 b

1.3 c

2. Casters :

- A: Swivel lock** – Prevents the caster from rotating.
B: Caster resistance – Adjusts the wheel resistance.
C: Brake pedal (lock) – Engages the brake.
D: Brake pedal (unlock) – Releases the brake.

2.1 Adjustment :

- a. Swivel lock (A) :
- To engage: Turn the knob 90° until it fully clicks into the lower slot.
 - To disengage: Lift the knob, turn it 90°, and make sure it does not fall back into the slot (see Figure 2.1c).

Lock all four swivel locks to help the user walk straight.

Locking only the rear casters provides stability while still allowing turning.

- b. Adjusting caster resistance (B) :
- Turn the knob to adjust resistance (see Figure 2.1d).
 - Please adjust according to the user's needs to avoid excessive or insufficient resistance.

Do not set the resistance to the maximum to the point where the rear casters can no longer rotate during use.

- c. Engaging the caster brake (C) :
 Press the lower brake pedal marked « **LOCK** » (see Figure 2.1e).
- d. Releasing the caster brake (D) :
 Press the upper brake pedal marked « **UNLOCK** » (see Figure 2.1f).

2.1 c

2.1 d

2.1 e

2.1 f

3. Handles :

Standard handles

Lowered handles


U-shaped handle

3.1 Attachment

All handles are attached in the same way:

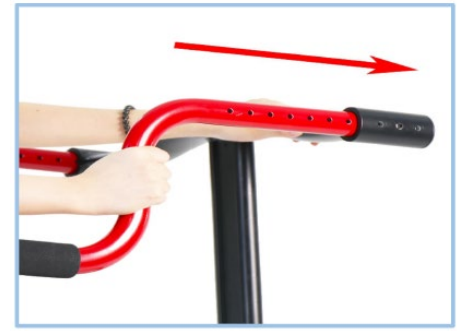- Insert the handle into the hole of the black support bar (see Figure 3.1a).
- Align the holes of the handle and the black support bar.
- Secure the handles to the black support bar with the knobs (see Figure 3.1b).
- Make sure the handles are properly secured by gently shaking them from side to side.

3.1 a

3.2 Adjustment

All handles are adjusted in the same way :

- Loosen the knobs securing the handles to the support bar.
- Adjust the handle length to match the user's arm size.
- Tighten the knobs to secure the handles.
- Make sure the handles are properly secured by gently shaking them from side to side.

3.1 b

U-shaped handle :

Press the buttons on both sides at the bottom of the U-shaped handle to remove the curved section, allowing easier access for the user.

Make sure both side buttons lock properly when reinstalling the curved section to prevent it from coming loose and causing injury.

4. Armrests :

PU armrests

Soft armrests

Swiveling armrests with handles

4.1 Attachment

4.1 a

4.1 b

4.1 c

All forearm support cushions are attached in the same way :

- Insert the handle into the joint of the forearm support cushion (see Figure 4.1a).
- Secure the handle with the forearm support cushion onto the black support bar using the same method as in Section 3.1 (see Figures 4.1b and 4.1c).
- Make sure the handles are firmly assembled by gently shaking them from side to side.

4.2 Adjustment

To adjust the position :

- Loosen the two joint bolts with a 4 mm hex key.
- Move the forearm support cushion forward or backward as needed.
- Tighten the bolts to secure the position.

To rotate :

- Loosen the two bolts under the cushion with a 4 mm hex key.
- Adjust the angle of the forearm support cushion as needed.
- Tighten the bolts to secure the position.

For the swiveling armrests with handles, the handles rotate together with the forearm support cushion


5. Backrest :

5.1 Attachment

- Insert the handles into the holes on each side of the backrest joints (see Figure 5.1a).
- Secure the handle with the backrest onto the black support bar using the same method as in Section 3.1.
- Tighten the two knobs located inside the backrest tube (see Figure 5.1b).
- Make sure the backrest and handles are firmly assembled by gently shaking them from side to side.

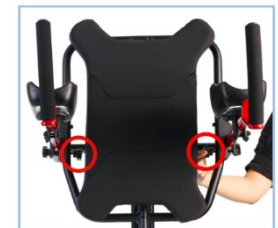5.1 a

5.1 b

5.2 Adjustment

- Loosen the two knobs located inside the backrest tube (see Figure 5.1b).
- Adjust the backrest forward or backward as needed.
- Tighten the knobs to secure the position.


6. Headrest :

Please note that the headrest can only be attached to the backrest.

6.1 Attachment

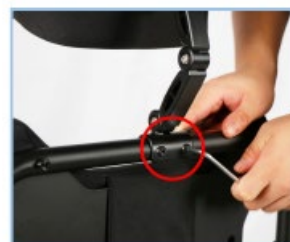6.1 a

6.1 b

- Align the holes of the headrest connectors with those of the backrest tube (see Figure 6.1a).
- Secure the headrest to the backrest with bolts and nuts (see Figure 6.1b). A 3 mm hex key and an 8 mm wrench are required.
- Check that the headrest is firmly assembled by gently shaking it.

6.2 Adjustment

To adjust the angle :

- Loosen the three bolts located behind the headrest cushion with a 4 mm hex key.
- Adjust the headrest angle as needed.
- Tighten the bolts to secure the position.

To adjust the position :

- Loosen the three bolts on the headrest joints with a 3 mm hex key.
- Adjust the headrest up or down, forward or backward as needed.
- Tighten the bolts to secure the position.

7. Saddle :

Please note that the seat can only be attached to the backrest.

7.1 Adjustment

To adjust the height :

- Loosen the bolts and nuts on the central tube behind the backrest using a 4 mm hex key and a 10 mm wrench (see Figure 7.1a).
- Adjust the seat height to a suitable position. If the seat is not needed, it can be removed directly.
- Align the holes and tighten the bolts and nuts.
- Shake the seat to ensure it is securely fixed.

7.1 a

To adjust the seat cushion position:

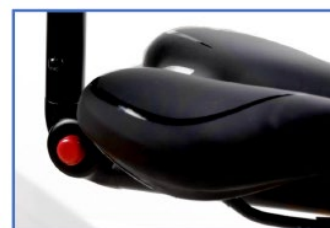
- Loosen the bolts on both sides with a 14 mm wrench (see Figure 7.1b).
- Adjust the seat cushion forward or backward as needed.
- Retighten the bolts to secure the position.

7.1 b

7.2 Use

- To lower the cushion when it is not needed, press the red button (see Figure 7.2a).
- To use the cushion, lift it until the red button pops out and a “click” is heard.

7.2 a

8. Leg separator :

8.1 Installation

- Align the holes of the leg separator with the lower tube of the frame.
- Attach the leg separator to the base frame using bolts (see Figure 8.1a). A 5 mm hex key is required.
- Make sure the leg separator is securely assembled by gently shaking it from side to side.

8.1 a

9. U-shaped harness & Hip positioner :

Please note that the U-shaped harness and the hip positioner can only be attached to the backrest.

U-shaped
harness

Hip positioner

9.1 Adjustment and Setting

We recommend putting on the hip positioner first.

To wear the hip positioner :

9.1 a

9.1 b

9.1 c

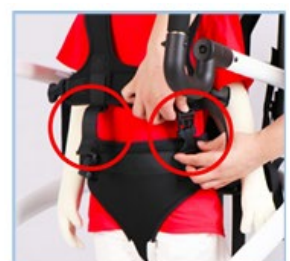
9.1 d

9.1 e

- Place the hip positioner between the user's legs, ensuring that the larger soft cushion is at the back and the smaller one at the front (see Figure 9.1a).
- Fasten the front cushion to the rear cushion. Adjust the cushions according to the hip width and the user's size (see Figure 9.1b).
- Thread the Velcro straps on both sides of the rear cushion through the loops of the front cushion and tighten them firmly (see Figures 9.1c & 9.1d).
- Place the user in the product and insert the metal hooks on both sides of the backrest into the insertion ends of the rear cushion (see Figure 9.1e).
- Adjust the length of the buckle straps on both sides according to the user's needs and comfort.

To wear the U-shaped harness :

- Check that all the buckles on the back of the U-shaped harness are fastened (see Figure 9.1f).
- Place the harness pads on the user's upper body.
- Insert the small hooks located on both sides of the backrest into the insertion ends on the sides of the harness pads (see Figure 9.1g).
- Fasten the harness pads together using the Velcro at point A, then secure the buckle at point B (see Figure 9.1h).
- If the user is also using the hip positioner, insert the front hooks of the harness pads into the insertion ends of the hip positioner (see Figure 9.1i).
- Adjust the length of all straps according to the user's needs and comfort.


Make sure the straps are not too loose, as this may fail to provide adequate support and could cause injury while walking.

Avoid overtightening, as this may cause discomfort or pressure on the user.

9.2 Attachment

To assemble the strap and the connector :

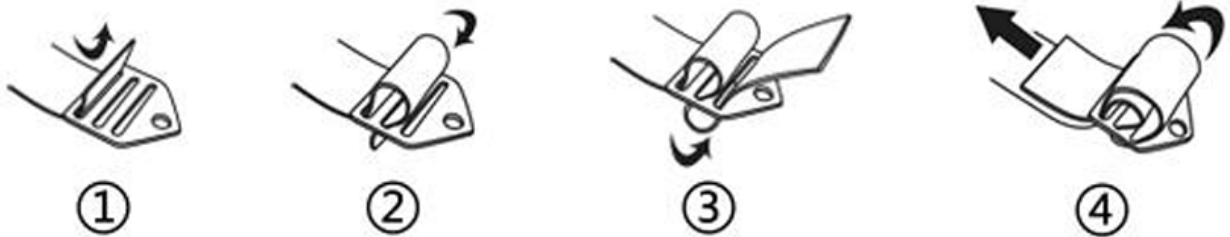If the strap and the connector are separated, assemble them as follows :

To fasten the strap with hooks :

- Insert the bolt into the hole of the connecting part between the backrest and the handle (see Figure 9.2a).
- Pass the bolt through the hole of the strap with hooks (see Figure 9.2b).
- Attach the nut to the bolt and tighten them using a 4 mm hex key and a 10 mm wrench (see Figure 9.2c).

9.2 a

9.2 b

9.2 c

9.2 d

To fasten Connector A :

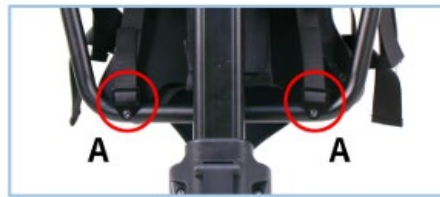
- Loosen the bolts and nuts located behind the lower tube of the backrest using a 4 mm hex key and a 10 mm wrench

(see Figures 9.2f and 9.2g).

- Wrap the strap with Connector A around the upper tube of the backrest, from front to back, then slide it down to the lower tube.
- Insert the bolts into the holes of Connector A and into the holes of the tube (see Figure 9.2h).
- Retighten the bolts and nuts to secure the position.

9.2 e

9.2 f

9.2 g

9.2 h

10. Ankle indicators

10.1 Attachment

10.1 a

10.1 b

10.1 c

- Wrap the strap without Velcro around the lower tube of the walker, as shown in Figure 10.1a.
- Pass the wrapped strap through the tightening buckle (see Figures 10.1b and 10.1c).
- Adjust the strap according to the user's needs and comfort.

10.2 Adjustment and Setting

Wrap the Velcro strap around the user's ankle, then pass it through the buckle on the other side and fasten it firmly.

For users with sensitive skin, we recommend wearing long socks or trousers to avoid friction from the strap while walking.

CARE - MAINTENANCE

Failure to maintain your product may result in a risk of serious injury.

CLEANING :

Frame

- Clean with a damp cloth and mild detergent.
- If disinfection is required, clean with a disinfectant solution using a damp cloth.

Covers, U-shaped harness, hip positioner, ankle indicators

The fabric covers of the headrest and backrest are removable and machine washable.

- Place the cover in a laundry bag to prevent damage during machine washing or drying.
- Wash in warm water (40 °C / 100 °F) with mild detergent in a washing machine.
- Tumble dry at low temperature or air dry.
- Do not bleach or iron.
- Do not dry clean.

Foams

- Clean with a damp cloth and mild detergent.
- Do not use bleach or biological cleaners.
- Lubricate the wheels with a small amount of oil if they squeak.
- Periodically check the wear and tightness of all screws and bolts. Immediately replace any broken, damaged, or worn parts.
- In case of malfunction, do not attempt to repair the rollator yourself. Contact an authorized dealer.

• Checklist :

To keep the walker in good condition, check it before each use, and carry out regular inspections as follows :

Check the item	Anytime	Weekly
Height adjustment clip		X
Frame welding at wheel level		X
Cleanliness	X	
Screws and fasteners		X
Condition and functioning of brakes	X	
Condition and functioning of support systems (straps, harnesses, supports)		X
Frame stability		X

CHANGE OF OWNERSHIP

The product may be reused. It must first be cleaned and disinfected in compliance with hygiene instructions. When transferring the product, be sure to provide all necessary technical documents to the new user. Beforehand, the product must be inspected by an authorized specialist.

RECYCLING

When the product becomes unusable and must be disposed of, please contact your specialized dealer. If you wish to handle recycling yourself, ask your local waste management company for disposal guidelines.

STORAGE

Failure to follow the storage conditions may lead to product deterioration and therefore a risk of serious injury.


- Do not store your product for a long period near a heat source or in direct sunlight (e.g., behind a window or near a radiator) or near a cold source.
- Keep away from flames and sparks.
- Follow the conditions for storing your product.
- Store in a dry, temperate place.
- Protect the product with packaging against dust and corrosion (e.g., abrasive elements, sand, seawater, salty air).
- Store all removed parts together in the same place (or label them if necessary) to avoid mixing with other products during reassembly.
- All components must be stored without any load (do not place heavy objects on the product parts or wedge anything against them)

SERVICE LIFE

The expected service life of this product is 5 years under normal conditions of use, safety, and maintenance for a single user. Beyond this period, the product may still be used as long as it remains in good condition. If the product is used in a multi-user environment (such as a nursing home or hospital), the service life may be reduced.

TECHNICAL SPECIFICATIONS

Model	Size S	Size M	Size L
A: Overall height including headrest	89 – 114 cm	104 – 139 cm	123 – 164 cm
B: Overall width	55.5 cm	60 cm	62.5 cm
C: Inside width	44 cm	49 cm	52 cm
D: Overall depth	86 cm	90.5 cm	92 cm
E: Inside depth	62 cm	66 cm	70 cm
F: Height from the bottom to the handle	53.5 – 67.5 cm	69 – 90 cm	86 – 113 cm
User height	100 – 125 cm	115 – 150 cm	134 – 175 cm
Maximum permitted weight	85 kg	85 kg	85 kg

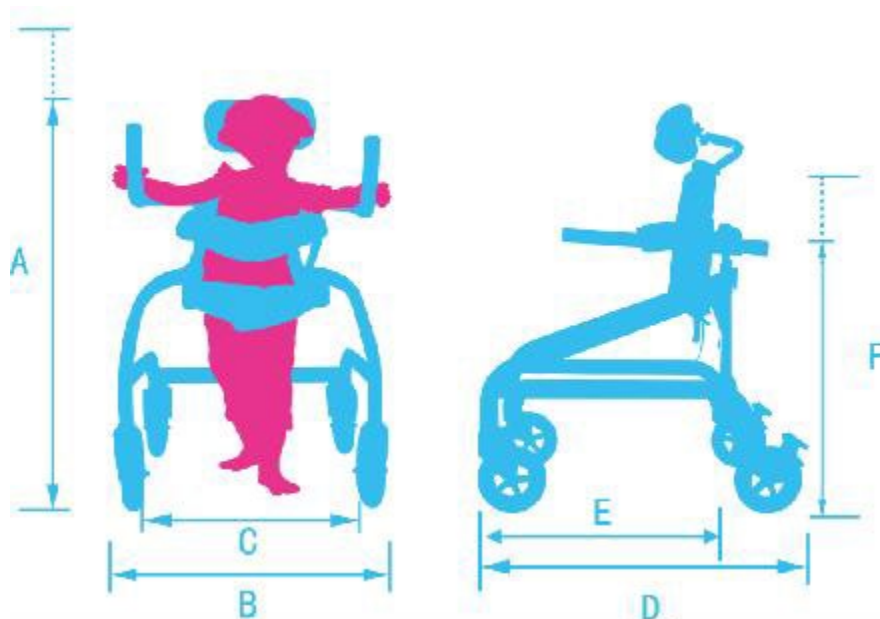GUARANTEE

This product is guaranteed for two years from the date of purchase by the user. IDENTITIES will provide replacement parts for any part that has broken due to a manufacturing, design or manufacturing deficiencies. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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