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IMPORTANT

Please read the product manual carefully before use. Keep the manual for future reference.
The swivel cushion is designed to facilitate user transfers.

ADVANTAGES

- Rotates 360° for easy and effortless movement in any seated direction
- Portable and lightweight (weighs only 1.2 kg!)
- Durable plastic swivel base with non-slip bottom
- Ideal for getting in or out of a vehicle
- Helps prevent back or hip problems
- Comfortable
- Washable cover

SPECIFICATIONS

- Composition: Wooden structure, foam seat, removable and washable polyester cover.
- Dimensions: Diameter 39 cm
- Maximum weight capacity: 135 kg.

INSTALLATION

Remove the product from its packaging. Ensure all components are included and follow the instructions.

The cushion rotates 360° in both directions. It can be used on a car seat, chair, or household/office armchair. Place the cushion directly on the desired surface (seat, chair...). You may sit down.

Caution: The cushion must be properly positioned on the surface. Ensure the surface is stable and poses no risk to the user.

USAGE CONDITIONS AND SAFETY

Use the swivel cushion only for its intended purpose.

Ensure the cushion is properly installed and stable.

Do not modify the product's structure or attempt self-repairs.

The manufacturer is not responsible for damages resulting from improper use or maintenance.

Keep out of reach of children.

MAINTENANCE/SAFETY

Do not use cleaning products containing caustic or abrasive agents.

Troubleshooting:

If the swivel cushion is unstable: The cushion is not suitable for the surface. DO NOT USE THE PRODUCT.

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