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IMPORTANT

Child Safety

Keep the pill organizer out of the reach of children to prevent accidental ingestion.

PRODUCT DESTINATION

Suitable for individuals who take long-term medication, elderly users, people with memory challenges, or those managing daily supplements.

This product is a portable pill organizer with a leather case, designed for convenient medication management. It features 28 compartments labeled MORN, NOON, EVE, and BED, allowing for four daily doses for a week.

Product description:

- Clear Categorization: Each compartment is labeled with a specific time of day for easy medication tracking.
- Enough Capacity: Each compartment provides ample space for multiple pills or supplements.
- Transparent Lids: Easily check medication without opening the compartments, reducing the risk of missing or double doses.
- Portable Leather Case: Protects the pill organizer with a closure, making it convenient for travel and daily use.
- Detachable Daily Compartments: Each day's pill box can be removed separately for on-the-go convenience

USE

1. Filling the Organizer: Open the transparent lids and place the prescribed medications into the corresponding compartments based on the time of day.
2. Closing the Compartments: Ensure all lids are securely closed to prevent pills from spilling.
3. Taking Medication: Open the designated compartment (MORN, NOON, EVE, BED) and take the pills as scheduled.
4. Carrying On-the-Go: If traveling, detach the daily pill box and carry it separately for convenience.
5. Weekly Refills: Refill the organizer at the beginning of each week to ensure uninterrupted medication intake.

CARE

Regularly clean the pill box with water or a damp cloth. Avoid using harsh chemicals.

STORAGE

Keep the pill organizer in a cool, dry place, away from direct sunlight and heat.

Special Medication Storage: Do not store temperature-sensitive medications (e.g., insulin) in the organizer. Follow specific storage instructions for such medications.

FEATURES TECHNICAL

Case material	Leather
Pill organizer dimensions	Length 19 cm x Width 12 cm
Compartment dimensions	4 x 7 cm

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